BM 16

D	Blutdruckmessgerät Gebrauchsanweisung	2-12
GB	Blood pressure monitor Instructions for use	13-22
F	Tensiomètre Mode d'emploi	23-33
E	Tensiómetro Manual de instrucciones	34-44
I	Misuratore di pressione Istruzioni per l'uso	45-55
TR	Tansiyon Ölçme Aleti Kullanım kılavuzu	56-65
RUS	Прибор для измерения кровяного давления в плечевой артерии Инструкция по применению	66-77
PL	Ciśnieniomierz Instrukcja obsługi	78-87
	Electromagnetic Compatibility Information	88-91

Inhalt

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Beurer GmbH
Servicecenter
Lessingstraße 10 b
89231 Neu-Ulm
Germany

Contents

1. Getting to know your instrument.....	13	6. Saving, retrieving and deleting results	20
2. Important information.....	14	7. Cleaning and storing the device and cuff.....	20
3. Unit description	16	8. Rectifying faults	21
4. Prepare measurement	17	9. Specifications.....	21
5. Measuring blood pressure.....	18	10. Mains part.....	22

Included in delivery

- Blood pressure monitor
- Upper arm cuff
- 4 x LR6 AA batteries
- Storage bag
- Instructions for use

Dear Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for the applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Best regards,
Your Beurer Team

1. Getting to know your instrument

Check that the packaging of the Beurer BM16 blood pressure monitor has not been tampered with and make sure that all the required contents are present.

The upper arm blood pressure monitor is used for non-invasive measurement and monitoring of adults' arterial blood pressure. You can use it to measure your blood pressure quickly and easily, storing the results and displaying the progression of readings together with the average.

A warning is issued for anyone suffering from cardiac arrhythmia.

The recorded values are classified and evaluated graphically.

2. Important information

Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and accessories:

	Caution
	Note Note on important information
	Follow instructions for use
	Type BF applied part
	Direct current
	Disposal in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).
	Manufacturer
Storage/Transport 	Permissible storage and transport temperature and humidity
Operating 	Permissible operating temperature and humidity

IP 21	Protected against foreign objects equal to or greater than 12.5 mm in diameter and against vertically falling drops of water
	Serial number
	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical products.

Advice on use

- In order to ensure comparable values, always measure your blood pressure at the same time of day.
- Before every measurement, relax for about five minutes.
- If you want to perform several measurements on the same person, wait five minutes between each measurement.
- Do not take a measurement within 30 minutes after eating, drinking, smoking or exercising.
- Repeat the measurement if you are unsure of the measured value.
- The measurements taken by you are for your information only – they are not a substitute for a medical examination! Discuss the measurements with your doctor, and never base any medical decisions on them (e.g. medicines and their administration)!
- Do not use the blood pressure monitor on newborns or patients with preeclampsia. We recommend consulting a doctor before using the blood pressure monitor during pregnancy.
- This device is not intended for use by people (including children) with restricted physical, sensory or mental skills or

a lack of experience and/or a lack of knowledge, unless they are supervised by a person who has responsibility for their safety or they receive instructions from this person on how to use the device. Supervise children around the device to ensure they do not play with it.

- Cardiovascular diseases may lead to incorrect measurements or have a detrimental effect on measurement accuracy. The same also applies to very low blood pressure, diabetes, circulatory disorders and arrhythmias as well as chills or shaking.
- The blood pressure monitor must not be used in connection with a high-frequency surgical unit.
- Only use the device on people who have the specified upper arm measurement for the device.
- Please note that when inflating, the functions of the limb in question may be impaired.
- During the blood pressure measurement, blood circulation must not be stopped for an unnecessarily long time. If the device malfunctions, remove the cuff from the arm.
- Avoid any mechanical restriction, compression or bending of the cuff line.
- Do not allow sustained pressure in the cuff or frequent measurements. The resulting restriction of the blood flow may cause injury.
- Ensure that the cuff is not placed on an arm in which the arteries or veins are undergoing medical treatment, e.g. intravascular access or therapy, or an arteriovenous (AV) shunt.
- Do not use the cuff on people who have undergone a mastectomy.
- Do not place the cuff over wounds as this may cause further injury.

- Place the cuff on your upper arm only. Do not place the cuff on other parts of the body.
- You can either use the blood pressure monitor with batteries or with a mains part. Please note that data transfer and data storage is only possible when your blood pressure monitor is supplied with power. As soon as the batteries are empty or the mains part is disconnected from the power supply, the blood pressure monitor loses the date and time.
- To conserve the batteries, the monitor switches off automatically if no buttons are pressed for one minute.
- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.

Storage and Care

- The blood pressure monitor is made up of precision electronic components. Accuracy of readings and the instrument's service life depend on careful handling.
 - You should protect the device from impact, moisture, dirt, major temperature fluctuations and direct exposure to the sun's rays.
 - Never drop the device.
 - Do not use near strong electromagnetic fields, i.e. keep it away from any radio systems and mobile phones.
 - Only ever use the cuffs provided with the monitor or original replacement cuffs. Otherwise erroneous results will be recorded.
- If the instrument is not used for any length of time, we recommend removing the batteries.

Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, flush out the affected areas with water and seek medical assistance.
-  **Choking hazard!** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect the batteries from excessive heat.
-  **Risk of explosion!** Never throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.

Repair and disposal

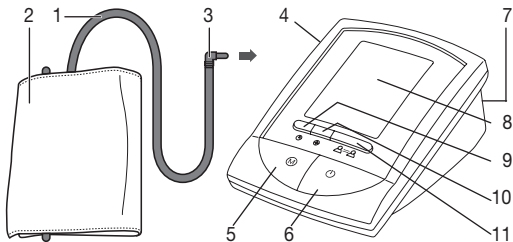
- Batteries do not belong in domestic refuse. Used batteries should be disposed of at the collection points provided.
- Never open the instrument. If these instructions are not heeded, the warranty will be null and void.
- Never attempt to repair the instrument or adjust it yourself. We can no longer guarantee perfect functioning if you do.
- Repairs may only be performed by Beurer Customer Service or authorized dealers. However, always check the batteries and replace them if necessary prior to making any complaint.

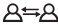
General disposal

- For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.


3. Unit description

1. Cuff tube
2. Cuff
3. Cuff connector
4. Cuff connector port (left side)
5. Memory button **M**
6. ON/OFF button 
7. Mains part port (rear)
8. Display
9. Date/time button 
10. Adjustment button 
11. User selection button 

Icons in the display:

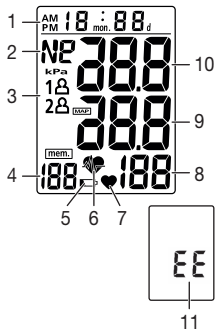1. Date/Time Indicator
2. Classification of measurements
3. Memory Zones
4. Memory Sequence Number
5. Weak Battery Mark 
6. Cardiac arrhythmia icon 
7. Pulse Mark
8. Pulse Rate
9. Diastolic Pressure
10. Systolic Pressure
11. Problem shown on the display

4. Prepare measurement

Inserting battery

- Remove the battery cover from the back of the monitor.
- Insert four AA 1.5V alkaline batteries, making absolutely sure that you insert them with the correct polarity as marked. Never use rechargeable batteries.
- Replace the battery cover carefully.

If the battery change  is continuously illuminated, measurement is no longer possible and you must replace all the batteries. Once batteries have been removed from the device, the time must be reset.

Battery disposal

- The empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.
- The codes below are printed on batteries containing harmful substances:
Pb = Battery contains lead,
Cd = Battery contains cadmium,
Hg = Battery contains mercury.

Setting date and time

It is vital to set date and time. Only in this way is it possible to save and subsequently retrieve your measured values with the right date and time.

Press the  key and  to adjust the month. Then press the  and  in turn to adjust the date, hour, minute and  to confirm the entry.

Time is shown in 12 hour format.

Operation with the mains part

You can also operate this device with a mains part. When doing so, there must not be any batteries in the battery compartment. The mains part can be obtained from specialist retailers or from the service address using order number 071.68.

- To prevent possible damage to the device, the blood pressure monitor must only be used with the mains part described here.
- Insert the mains part into the connection provided for this purpose on the right-hand side of the blood pressure moni-

tor. The mains part must only be connected to the mains voltage that is specified on the type plate.

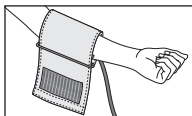
- Then insert the mains plug of the mains part into the mains socket.
- After using the blood pressure monitor, unplug the mains part from the mains socket first and then disconnect it from the blood pressure monitor. As soon as you unplug the mains part, the blood pressure monitor loses the date and time setting but the saved measurements are retained.

5. Measuring blood pressure


Please ensure the device is at room temperature before measuring.

Positioning cuff


Fit the cuff round your bare left upper arm. Blood circulation in the arm should not be restricted by tight clothing or other objects.

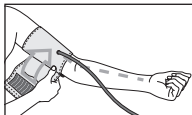The cuff should be placed on the upper arm so that the lower edge is 2 to 3 cm above the bend of the elbow and above the artery. The tube should be in line with the centre of the palm.

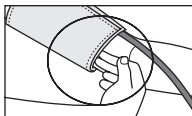
Now place the free end of the cuff snugly, but not too tightly, around the arm, and fix it with the Velcro fas-



tener. The cuff should be fitted tight enough to allow just two fingers to fit beneath the cuff.

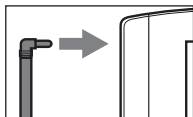Insert the cuff tubing into the socket for the cuff attachment.


Important: The instrument should only be operated with the original cuff. The cuff is suitable for an arm circumference of 24 to 36 cm. A larger cuff for upper arm circumferences from 34 to 46 cm is available from retailers or the service address under order no. 162.383.

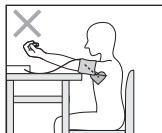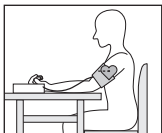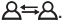Correct posture

- Rest for approx. 5 minutes before each measurement. Otherwise there may be divergences.
- You can perform the measurement either sitting or lying down. Always make sure that the cuff is on a level with your heart.
- To carry out a blood pressure measurement, make sure you are sitting comfortably with your arms and back leaning on something. Do not cross your legs. Place your feet flat on the ground.
- In order not to distort the result, it is important to keep still during the measurement and not talk.

Select memory

You have two memories (50 memory spaces each) in order to save the test results of 2 different people separately, or else save measurements in the morning and evening separately. Select the required memory space by pressing the user selection button .

Measuring blood pressure

- Put on the cuff as described previously and assume the position in which you want to carry out the measurement.
- Press the user selection button  to select a memory zone, and press the  key 2x to start measurement in the chosen memory zone. After checking the display with all digits lighting up, the monitor will automatically inflate. Following the self-test, during which all display elements briefly appear, the measurement begins. During inflation the device already determines measured values used to estimate the required inflation pressure. If this pressure is insufficient, the device automatically inflates by another 40 mmHg (Real Fuzzy Logic).
- Then the pressure in the cuff is slowly released and the pulse is measured.
- When the measurement has been completed, the remaining air pressure is released very quickly. The pulse, the systolic and the diastolic blood pressure are displayed.
- The device requires approx. 5 seconds to store the determined measured value.
- You can interrupt measurement at any time by pressing the ON/OFF button .
- To switch off and release the pressure, press the ON/OFF button  again. If you forget to switch off the device, it switches off automatically after approx. 1 minute.

Wait at least 5 minutes before taking another measurement!

Evaluating results

Cardiac arrhythmia:

This instrument can identify possible cardiac arrhythmia disorders during measurement and if necessary indicates the measurement with the flashing icon .

This may be an indicator for arrhythmia. Arrhythmia is a condition where the heart rhythm is abnormal as a result of defects in the bioelectrical system controlling the heart beat. The symptoms (omitted or premature heart beats, slow or excessively fast heart rate) may be caused, among other things, by heart disease, age, physical predisposition, excessive use of stimulants, stress or lack of sleep. Arrhythmia can only be ascertained through examination by your doctor.

Repeat the measurement if the flashing icon  is displayed after the measurement. Please note that you should rest for 5 minutes between measurements and not talk or move during the measurement. If the icon  appears often, please contact your doctor. Any self-diagnosis and treatment based on the test results may be dangerous. It is vital to follow your doctor's instructions.

Classification of measurements:

The measurements can be classified and evaluated in accordance with the following table.

However, these standard values serve only as a general guideline, as the individual blood pressure varies in different people and different age groups etc.

It is important to consult your doctor regularly for advice. Your doctor will tell you your individual values for normal blood pres-

sure as well as the value above which your blood pressure is classified as dangerous.

The classification in the display shows the range of the measured blood pressure.

If the values for systolic and diastolic pressure are in two different ranges (e.g. systolic in the grade 1 hypertension range and diastolic pressure in the normal range), the classification on the unit always indicates the higher range (grade 1 hypertension in the example described).

Table for classification of blood pressure values (unit of measurement mmHg) for adults:



Range	Systolic	Diastolic	Action
Normal range N	105–119	60–79	Self check
Pre-hypertensive blood pressure range* P	120–139	80–89	Check with doctor
High blood pressure - degree 1	140–159	90–99	Consult your doctor
High blood pressure - degree 2	≥160	≥100	Consult your doctor


(*) Blood pressure range that might transfer to high blood pressure
Adapted from JNC 2003

6. Saving, retrieving and deleting results

The computer automatically saves the blood pressure readings of the last 50 measurements. When the 50 memory locations are exceeded, the current oldest value is deleted.

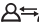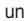You can display the memory as follows:

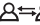- Press the user selection button  to select a memory zone, and press the **M** key to switch from memory to memory with sequence numbers.
- The saved measured values are continuously stored.
- Press the On/Off switch  to turn the unit off.
- If you forget to switch off the computer, it will switch off automatically after 1 minute.

Press the user selection button  to select a memory zone, then press and hold the **M** key for approximately 5-seconds to clear the data in the pre-designated memory zone. The data in both memory zones – zone 1 and zone 2 may be erased if any of the batteries is removed.

7. Cleaning and storing the device and cuff

- Clean the device and cuff carefully using a slightly damp cloth only.
- Do not use any cleaning agents or solvents.
- Under no circumstances hold the device and cuff under water, as this can cause liquid to enter and damage the device and cuff.
- If you store the device and cuff, do not place heavy objects on the device and cuff. Remove the batteries. The cuff line should not be bent sharply.

8. Rectifying faults

Error messages can occur when

- the blood-pressure value is unusually high or low (**EE** appears in the display),
- you move or talk during the measurement (**EE** appears in the display),
- the cuff tube is not properly inserted (**E I** appears in the display),
- inflation takes longer than 20 seconds (**E I** appears in the display),
- the inflation pressure is higher than 300 mmHg (**E2** appears in the display),
- an error occurs when storing the measured values (**E3** appears in the display),
- exceeding measurement range (**Er** appears in the display).

In the about cases, you must repeat the measurement. Make sure that the cuff tube is properly inserted and that you do not move or talk. Put the batteries back in if necessary, or else replace them.

9. Specifications

Model no.	BM 16
Measurement method	Oscillometric, non-invasive blood pressure measurement on the upper arm
Measurement range	Cuff pressure 0–299 mmHg, systolic 40–250 mmHg, diastolic 40–250 mmHg, Pulse 40–199 beats/minute
Display accuracy	Systolic ± 3 mmHg, diastolic ± 3 mmHg, pulse ± 5 % of the value shown

Measurement inaccuracy	Max. permissible standard deviation according to clinical testing: systolic 8 mmHg/ diastolic 8 mmHg
Memory	2 x 50 memory spaces
Dimensions	L 156 mm x W 105 mm x H 74,5 mm
Weight	Approx. 315 g (without batteries)
Cuff size	24 to 36 cm
Permissible operating conditions	+10 °C to +40 °C, 15–85 % relative air humidity (non-condensing)
Permissible storage and transport conditions	-10 °C to +60 °C, 10–90 % relative air humidity, 700–1060 hPa ambient pressure
Power supply	4 x 1,5V  AA batteries
Battery life	For approx. 300 measurements, depending on the blood pressure level and/or pump pressure
Accessories	Instruction for use, 4 x 1.5V AA batteries, storage pouch
Classification	Internal supply, IP21, no AP or APG, continuous operation, type BF applied part

The serial number is located on the device or in the battery compartment.

Technical information is subject to change without notification to allow for updates.

- This unit is in line with European Standard EN 60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility (EMC). Please note that portable

and mobile HF communication systems may interfere with this unit. More details can be requested from the stated Customer Service address or found at the end of the instructions for use.

- This device is in line with the EU Medical Devices Directive 93/42/EEC, the “Medizinproduktegesetz” (German Medical Devices Act) and the standards EN 1060-1 (non-invasive sphygmomanometers, Part 1: General requirements), EN 1060-3 (non-invasive sphygmomanometers, Part 3: Supplementary requirements for electro-mechanical blood pressure measuring systems) and IEC 80601-2-30 (Medical electrical equipment – Part 2–30: Particular requirements for the safety and essential performance of automated non-invasive blood pressure monitors).
- The accuracy of this blood pressure monitor has been carefully checked and developed with regard to a long useful life. If using the device for commercial medical purposes, it must be regularly tested for accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address.

10. Mains part

Model no.	8194049/01
Input	100-240V, 50/60 Hz
Output	12V DC, 600 mA, only in connection with beurer blood pressure monitor
Supplier	Ansmann AG

Protection	This device is double insulated and protected against short circuit and overload by a primary thermal fuse. The green LED lights under normal conditions. Make sure to take the batteries out of the compartment before using the mains part.
	Double insulated/equipment class 2
	Polarity of the the DC voltage connection
Enclosures and Protective Covers	Equipment enclosed to protect against contact with live parts, and with parts which can become live (finger, pin, hook test). The operator shall not contact the patient and the output plug of DC mains part simultaneously.

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Table 1

For all ME EQUIPMENT and ME SYSTEMS

Guidance and manufacture's declaration – electromagnetic emissions

The BM 16 is intended for use in the electromagnetic environment specified below.

The customer or the user of the BM 16 should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The BM 16 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The BM 16 is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

Table 2*Guidance and manufacturer's declaration – electromagnetic immunity*

The BM 16 is intended for use in the electromagnetic environment specified below. The customer or the user of the BM 16 should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV for power supply lines N/A	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV differential mode N/A	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% U_T (>95 % dip in U_T) for 0.5 cycle 40% U_T (60 % dip in U_T) for 5 cycles 70% U_T (30 % dip in U_T) for 25 cycles <5% U_T (>95 % dip in U_T) for 5 s	<5% U_T (>95 % dip in U_T) for 0.5 cycle 40% U_T (60 % dip in U_T) for 5 cycles 70% U_T (30 % dip in U_T) for 25 cycles <5% U_T (>95 % dip in U_T) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the BM 16 requires continued operation during power mains interruptions, it is recommended that the BM 16 be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE: U_T is the a.c. mains voltage prior to application of the test level.

Table 3

Guidance and manufacturer's declaration – electromagnetic immunity

The BM 16 is intended for use in the electromagnetic environment specified below. The customer or the user of the BM 16 should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3V _{rms} 150 kHz to 80 MHz	3V _{rms}	<p>Portable and mobile RF communications equipment should be used no closer to any part of the BM 16, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance:</p> $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P} \text{ 80 MHz to 800 MHz}$
Radiated RF IEC 61000-4-3	3V/m 80 MHz to 2.5 GHz	3V/m	$d = 2.3 \sqrt{P} \text{ 800 MHz to 2.5 GHz}$ <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p>

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the BM 16 is used exceeds the applicable RF compliance level above, the BM 16 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the BM 16.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.

Table 4**Recommended separation distances between portable and mobile RF communications equipment and the BM 16**

The BM 16 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the BM 16 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the BM 16 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.