D	Internet-Blutdruckmessgerät	
	Gebrauchsanweisung	2
GB	Internet blood pressure monitor	
	Instructions for use	14
F	Tensiomètre connecté à Internet	
	Mode d'emploi	26
E	Tensiómetro con acceso a internet	
	Instrucciones de uso	38
I	Misuratore di pressione Internet	
	Istruzioni per l'uso	50
TR	İnternet tansiyon ölçme aleti	
	Kullanım kılavuzu	62

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Dear customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air. Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

With kind regards,
Your Beurer team

1. Getting to know your instrument

The Internet blood pressure monitor is used to carry out non-invasive measurement and monitoring of arterial blood pressure values in human adults.

This allows you to quickly and easily measure your blood pressure; the measurements are automatically transferred to the Beurer Connect website, where you can access them using an Internet browser or phone app (iOS & Android). The recorded values are classified according to the WHO guidelines and evaluated graphically. It enables you to display the development and average of all measured values. You are also warned of possible existing cardiac arrhythmia.

2. Included in delivery

See Description of units and accessories, page 17.

- Upper arm blood pressure monitor
- Cuff
- Wireless box
- Wireless box mains part
- LAN cable
- Storage pouch
- 4 x 1.5V AAA batteries
- Quick start guide
- These instructions for use

System requirement

Broadband router with an LAN connection

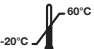
Supported browsers at the time of publication

Windows® Internet Explorer® version 9.0, Mozilla® Firefox® 11.0, Safari 5 for Mac, Google Chrome™ 18.0, Opera for computers 11.0

3. Important notes

Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and accessories:

	Caution		Manufacturer
	Note Note on important information		Temperature limitation
	Follow the instructions for use		Permissible storage air humidity
	Type B applied part		Keep dry
	Direct current	SN	Serial number
	Disposal in accordance with EC Directive 2002/96/EC – WEEE (Waste Electrical and Electronic Equipment).	 0123	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EWG on medical products.

Instructions for application

- Always measure your blood pressure at the same time of day to guarantee comparable values.
- Before every measurement, relax for about five minutes.
- Always wait 5 minutes between two measurements!
- Do not take a measurement within 30 minutes of eating, drinking, smoking or exercising.
- Please repeat the measurement if you are unsure of the measured value.
- The measurements taken by you are for your information only – they are no substitute for a medical examination! Discuss the measurements with your doctor, and never base any medical decisions on them (e.g. medicines and their administration)!
- Do not use the blood pressure monitor on newborns, pregnant women and patients with preeclampsia.
- Erroneous measurements or interference with the measurement accuracy of the unit can arise from diseases of the cardiovascular circulation system, just as from very low blood pressure, diabetes, blood circulation and arrhythmias as well as from chills or shaking.
- The blood pressure monitor must not be used in connection with a high-frequency surgical unit.
- Only use the unit on people that have the specified upper arm measurement for the unit.
- Please note that pumping up the cuff can sometimes lead to impairment of the function of the limb concerned.
- During the blood pressure measurement, the blood circulation must not be stopped for an unnecessarily long time. If the device malfunctions remove the cuff from the arm.
- Avoid any mechanical restriction, compression or sharp bending of the cuff line.
- Do not allow sustained pressure in the cuff or frequent measurements, as the resulting restriction of the blood flow may cause injury.

- Make sure that the cuff is not placed on an arm in which the arteries or veins are undergoing medical treatment, e.g. intravascular or therapy, or an arteriovenous (AV) shunt.
- Do not use the cuff if the patient has undergone a mastectomy.
- The cuff must not be placed over a wound, as this may cause further injury.
- You can either use the blood pressure monitor with batteries or with the mains part. Please note that data transfer and data storage is only possible when your blood pressure monitor is supplied with power. As soon as the batteries are empty or the mains part is disconnected from the power supply, the blood pressure monitor loses the date and time.
- To conserve the batteries, the blood pressure monitor switches off automatically if you do not press any buttons for one minute.
- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.

Instructions for storage and maintenance

- The blood pressure monitor is made from precision and electronic components. The accuracy of the measurements and service life of the device depend on its careful handling:
 - Protect the device from impacts, humidity, dirt, marked temperature fluctuations and direct sunlight.
 - Do not drop the device.
 - Do not use the device in the vicinity of strong electromagnetic fields and keep it away from radio systems or mobile telephones.
 - Only use the cuff included with the delivery or original replacement parts. Otherwise incorrect measurements will be recorded.
- Do not press the buttons before the cuff is placed on the arm.
- We recommend that the batteries be removed if the device will not be used for a prolonged period of time.

Notes on batteries


- Swallowing batteries can be extremely dangerous. Keep the batteries and products out of the reach of small children. If a battery is swallowed, seek medical attention immediately.
- Batteries must not be recharged, reactivated by other means, taken apart, thrown into an open fire or short circuited.
- If the batteries are empty or you do not intend to use the device for longer periods, remove the batteries from the device. This prevents damage which could occur when the batteries expire. Always replace all batteries at the same time.
- Do not use different types of batteries, different brands or batteries with different capacities. For preference, use alkaline batteries.

Instructions for repairs and disposal

- Batteries do not belong in household waste. Please dispose of empty batteries at the collection points intended for this purpose.
- Do not open the device. Failure to comply will invalidate the warranty.
- Do not repair or adjust the device. Proper operation can no longer be guaranteed in this case.
- Repairs may only be carried out by Beurer Customer Services or authorised retailers. Before making a claim, please check the batteries first and replace them if necessary.
- Please dispose of the device in accordance with EC Directive 2002/96/EC – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please contact the appropriate local authorities.

4. Unit description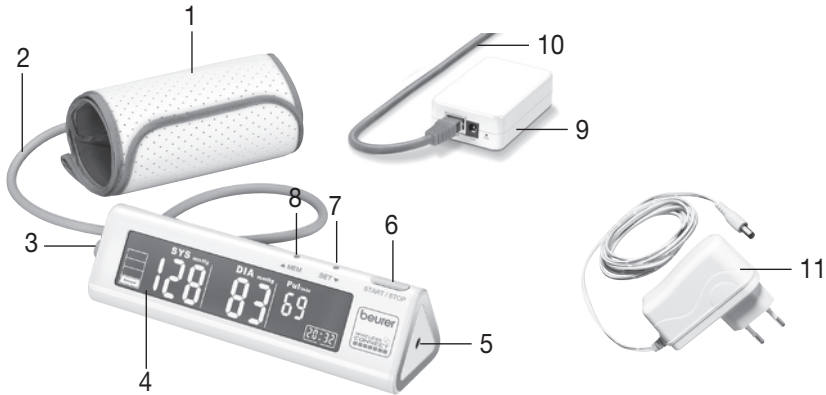

1. Cuff
2. Cuff line
3. Connection for cuff connector (left-hand side)
4. Display
5. Connection for mains part
6. **START/STOP** button
7. **SET ▼** function button
8. **MEM ▲** memory button
9. Wireless box
10. LAN cable
11. Wireless box mains part

Information on the display

Display	Meaning
	Classification of blood pressure measurements
SYS	Systolic pressure
DIA	Diastolic pressure
mmHg	Measurement unit
kPa	Measurement unit (1 kPa = 7.5 mmHg)
▼	Release air
▲	Inflating
♥	Pulse symbol


Display	Meaning
	Cardiac arrhythmia symbol
AM	Hour format (24h/12h)
	Date (month/day)
M	Memory display: memory space number/total memory spaces occupied
	Memory display mode
	Memory display: average value of all memory spaces
	Low battery indicator
	Movement sensor: warning of restless behaviour during measurement

5. Preparing the measurement

To use your new Beurer BM90 Internet blood pressure monitor, you require a broadband router with an LAN connection. To start-up the device, please carry out the following steps in the sequence shown:

- Visit our website: www.beurer-connect.com
- Register
- Enter the wireless box activation code. You will find this on the underside of the wireless box.
- Using the LAN cable connect the wireless box to your router.

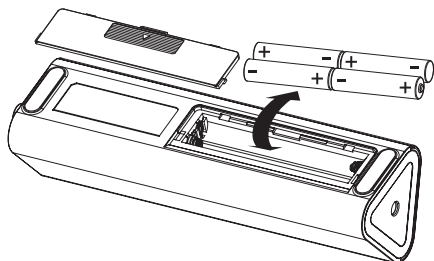- To prevent possible damage to the device, the wireless box must only be used with the mains part included in delivery.
Insert the mains part into the connection provided for this purpose on the top of the wireless box. Then insert the mains plug of the mains part into the mains socket.
 - You can see if the wireless box is ready by the 'traffic light' function: green – ready for use (connection to the server established).
 - Now select the "blood pressure monitor" option on the website and enter the activation code of your blood pressure monitor. You will find this on the underside of the blood pressure monitor.
 - Open the battery compartment lid on the blood pressure monitor.
 - Insert four 1.5 V AAA (alkaline type LR 03) batteries. Make sure that the batteries are inserted the correct way round.
- Do not use rechargeable batteries.

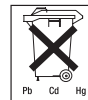
- Subsequently close the battery compartment lid carefully.
- Carry out a measurement and assign the result to your profile.

Replacing the batteries

If the low battery indicator  and  are displayed, you can no longer perform any measurements and must replace the batteries. Once the batteries have been removed from the device, the current date and time are deleted. The saved measurements are retained.

Used batteries should not be disposed of in normal household waste. Dispose of them via your electronics retailer or your local recycling point. You are legally obligated to do so.

-  The codes below are printed on batteries containing harmful substances:
Pb: battery contains lead,
Cd: battery contains cadmium,
Hg: battery contains mercury.

Setting the date and time

- If there is an existing connection to the server, the date and time will be automatically retrieved by the server.
- If there is no connection to the server, you will be sent directly to the date/time settings mode.
It is essential to set the date and time. Otherwise, when you reconnect to the server, your measurements will not be transferred, stored and later retrieved with the correct date and time.

-  If you press and hold the  **MEM** button, you can set the values more quickly.

- When switched off, press the **SET ▼** button for approx. three seconds. The year display will start to flash. Set the year with the **▲ MEM** button and confirm with the **SET ▼** button.
- Then set the month, day, time format (24h/12h), hour, minutes and measurement unit (mmHg/kPa) and confirm each setting with the **SET ▼** button.
- After you have performed all the settings, **0 000** appears on the display and the device automatically switches off.

Operating the blood pressure monitor with a mains part


You can also operate this blood pressure monitor with a mains part. When doing so, there must not be any batteries in the battery compartment. The mains part can be obtained from specialist retailers or from the service address using order number 071.50.

- To prevent possible damage to the device, the blood pressure monitor must only be used with the mains part described here.
- Insert the mains part into the connection provided for this purpose on the right-hand side of the blood pressure monitor.
The mains part must only be connected to the mains voltage that is specified on the type plate.
- Then insert the mains plug of the mains part into the mains socket.
- After using the blood pressure monitor, unplug the mains part from the mains socket first and then disconnect it from the blood pressure monitor. As soon as you unplug the mains part, the blood pressure monitor loses the date and time setting but the saved measurements are retained.

6. Measuring blood pressure


Ensure the device is at room temperature before measuring.

Attach the cuff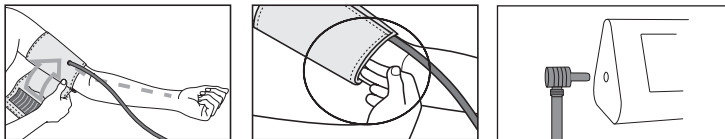

 The unit may only be operated with the original cuff. The cuff is for an arm circumference of 22 to 42 cm.

Adopt the correct posture

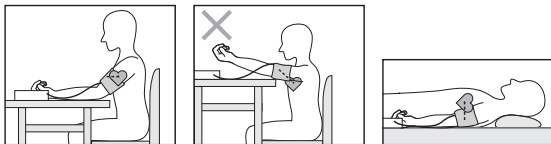- Before every measurement, relax for about five minutes. Otherwise deviations can occur.
- You can perform the measurement while sitting or lying. Make sure that the the cuff is at heart level.

- To carry out a blood pressure measurement, make sure you are sitting comfortably, with your arms and back leaning on something. Do not cross your legs. Place your feet flat on the ground.
- To avoid falsifying the measurement, it is important to remain still during the measurement and not to speak!

Performing the blood pressure measurement

- As described before, attach the cuff and adopt the posture in which you want to perform the measurement.
- Start the blood pressure monitor with the **START/STOP** button. • After the self test, during which all display elements are displayed, measurement begins. The cuff pumps itself up automatically. The measurement is taken during the pump operation.

 Measuring can be cancelled at any time by pressing the **START/STOP** button.

- As soon as a pulse is detected, the pulse symbol flashes .
- When the measurement has been taken, the pressure dissipates very quickly. The pulse, the systolic blood pressure and diastolic blood pressure are shown on the display.
- The measurements are automatically transferred to your profile. This is shown on the display by a rolling .
- The successful transfer is signalled when **Send** is shown on the display.

 If the transfer fails, **STORED** is shown on the display. In this event, the measurements are stored on the blood pressure monitor and will be transferred once the server connection is re-established.

- If the blood pressure measurement has not been performed properly, **E** appears on the display. Observe the chapter on error messages/troubleshooting in these instructions for use and repeat the measurement.
- In addition to the transfer of the measurements to your Beurer Connect profile, the device has 60 memory spaces available. The measurement is automatically stored.
- To switch off the device, press the **START/STOP** button. If you forget to turn off the device, it will switch off automatically after approx. 1 minute.

Wait at least 5 minutes before taking another measurement!

7. Evaluating results

Cardiac arrhythmia:

This unit can identify potential disruption of the heart rhythm when measuring and if necessary, indicates this after the measurement with the symbol . This can be an indicator for arrhythmia. Arrhythmia is an illness in which the heart rhythm is abnormal because of flaws in the bioelectrical system that regulates the heartbeat. The symptoms (skipped or premature heart beats, pulse being slow or too fast) can be caused by factors such as heart disease, age, physical make-up, excess stimulants, stress or lack of sleep. Arrhythmia can only be determined through an examination by your doctor. If the symbol  is shown on the display after the measurement has been taken, it should be repeated. Please ensure that you rest for 5 minutes beforehand and do not speak or move during the measurement. If the symbol  appears frequently, please consult your doctor. Self-diagnosis and treatment based on the measurements can be dangerous. Always follow your GP's instructions.

Table for the classification of blood pressure values (measurement unit, mmHg) for adults:

Display	Category	Systole	Diastole	Action
Normal	Normal range	< 140	< 90	Self-monitoring
Mild	Grade 1: mild hypertension	140 - 159	90 - 99	regular monitoring by doctor
Moderate	Grade 2: moderate hypertension	160 - 179	100 - 109	seek medical attention
Severe	Grade 3: severe hypertension	> = 180	> = 110	seek medical attention

8. Saving, displaying and deleting measurements on the device

Saving measurements

- The results of every successful measurement are automatically stored together with the date and time. If there are more than 60 measurements, the oldest measurements are lost.

Displaying measurements

- When switched off, press the **▲ MEM** button. Firstly, the average value of all values stored will be displayed. This is denoted by  in the display.
- If you press the memory button, **MEM ▲** again, further individual measurements are displayed – beginning with the value measured most recently. The individual values are denoted by the memory space numbers 1 to 60. The date of each measurement is displayed after approx. 4 seconds and after about a further 4 seconds the time is also displayed.

 By pressing the **▲ MEM** button, the most recent measurements are displayed; by pressing the **SET ▼** button, the oldest are displayed.

- Turn off the device using the **START/STOP** button or it will turn off automatically after approx. 1 minute.

Deleting measured values from the device

- To clear the memory, press the **▲ MEM** button for approx. 3 seconds when switched off.
-  flashes on the display; confirm using the **SET ▼** button. When the values have been deleted,  is shown on the display and the device turns off automatically.

9. Error message/troubleshooting

If the blood pressure measurement has not been performed properly,  appears on the display.

Error messages may appear if:

- : the cuff is fastened too loosely,
- : the cuff is fastened too tightly,
- : the pump pressure is too high,
- : you move or speak during the measurement,
- : there is a system or unit error.

In such cases, repeat the measurement. Ensure that the cuff line is correctly attached and that you do not move or speak. If necessary, reinsert or replace the batteries.

10. Cleaning and storing the unit

- Clean your blood pressure monitor carefully using a slightly damp cloth only.
- Do not use any cleaning agents or solvents.
- Under no circumstances should you hold the unit under water, as this can cause liquid to enter and damage the unit.
- When you store the unit, no heavy objects should be placed on top of it. Remove the batteries. The cuff line should not be bent sharply.

11. Technical specifications

Model no.	BM 90
Measurement method	Oscillometric, non-invasive blood pressure measurement on the upper arm
Measurement range	Cuff pressure 0–300 mmHg, systolic 30–280 mmHg, diastolic 30–280 mmHg, pulse 40–199 beats per minute
Display accuracy	systolic ± 3 mmHg/diastolic ± 3 mmHg/ pulse $\pm 5\%$ of the value shown
Measurement inaccuracy	max. permissible standard deviation according to clinical testing: systolic 8 mmHg/diastolic 8 mmHg
Memory	60 memory spaces
Dimensions	L 140 mm x W 60 mm x H 56 mm
Weight	Approximately 471 g (without batteries)
Cuff size	22 to 42 cm
Permissible operating conditions	+5°C to +40°C, $\leq 80\%$ relative humidity (non-condensing)
Permissible storage conditions	-20°C to +60°C, < 10–93% relative humidity Ambient pressure: 800–1050 hPa
Power supply	6V  (4x 1.5V AAA)
Battery life	For approx. 120 measurements, depending on levels of blood pressure and pump pressure
Radio frequency	RF 868MHz
Protection class	Internal supply, IPX0, no AP or APG, continuous operation
Classification	Application part type B

Mains part for wireless box

Model No.	AMS47-0501000FV
Input	100–240V~, 50/60Hz, 0,2A
Output	5V  1,0A, only in connection with Beurer wireless connect box
Manufacturer	Amigo (Global Yeou Diann Electric Industrial Co., Ltd.)
	This device is double insulated / equipment class 2
Housing and protective cover	Equipment enclosed to protect against contact with live parts, and with parts which can become live (finger, pin, hook test).

Mains part for blood pressure monitor

Model No.	UE 08 WCP-060100SPA
Input	100–240 V~, 50–60 Hz
Output	6V DC  , 400 mA, only in conjunction with the Beurer BM 90 blood pressure monitor
Manufacturer	Fuhua Electronic Co., Ltd.
Protection	The device is double protected and has a primary-side cutout switch which disconnects the device from the mains in case of malfunction. Ensure that you have removed the batteries from the battery compartment before you use the mains part.
	Insulated/protection class 2
Housing and protective cover	The housing of the mains part protects users from touching live parts or parts that could be live (for example with finger, needle, checking hook). The user may not touch the patient and the output connector of the AC mains part at the same time.

Technical information is subject to change without notification to allow for updates.

- We hereby guarantee that this product complies with the European R&TTE Directive 1999/5/EC. Please contact the specified service address to obtain further information, such as the CE Declaration of Conformity.
- This device complies with European Standard EN 60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility. Please note that portable and mobile HF communication systems may interfere with this unit. More details can be requested from the stated Customer Service address or found at the end of the instructions for use.

- This device is in line with the EU Medical Devices Directive 93/42/EC, the „Medizinproduktegesetz“ (German Medical Devices Act) and the standards EN 1060-1 (non-invasive sphygmomanometers, Part 1: General requirements), EN 1060-3 (non-invasive sphygmomanometers, Part3: Supplementary requirements for electro-mechanical blood pressure measuring systems) and IEC 80601-2-30 (Medical electrical equipment – Part 2-30: Particular requirements for basic safety and essential performance of automated non-invasive sphygmomanometers).
- The accuracy of this blood pressure monitor has been carefully checked and developed with regard to a long useful life.
If using the device for commercial medical purposes, it must be regularly tested for accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address.

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ELECTROMAGNETIC COMPATIBILITY INFORMATION

Table 1
For all EQUIPMENT and SYSTEMS

Guidance and manufacturer's declaration - electromagnetic emissions		
The BM90 is intended for use in the electromagnetic environment specified below. The customer of the user of the BM90 should assure that it is used in such and environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The BM90 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The BM90 is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	

Table 2
For all ME EQUIPMENT and ME SYSTEMS

Guidance and manufacturer's declaration - electromagnetic immunity			
The BM90 is intended for use in the electromagnetic environment specified below. The customer or the user of BM90 should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floor are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines	±2kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV line(s) to line(s)	±1 kV differential mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% U_T (>95% dip in U_T) for 0.5 cycle	<5% U_T (>95% dip in U_T) for 0.5 cycle	Mains power quality should be that of a typical commercial or hospital environment. If the user of the BM90 requires continued operation during power mains interruptions, it is recommended that the BM90 be powered from an uninterruptible power supply or a battery.
	40% U_T (60% dip in U_T) for 5 cycles	40% U_T (60% dip in U_T) for 5 cycles	
	70% U_T (30% dip in U_T) for 25 cycles	70% U_T (30% dip in U_T) for 25 cycles	
	<5% U_T (>95% dip in U_T) for 5 sec	<5% U_T (>95% dip in U_T) for 5 sec	
Power frequency (50Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE U_T is the a.c. mains voltage prior to application of the test level.			

Table 3
For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING

Guidance and manufacturer's declaration - electromagnetic immunity			
The BM90 is intended for use in the electromagnetic environment specified below. The customer or the user should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
<p>Conducted RF IEC 61000-4-6</p> <p>Radiated RF IEC 61000-4-3</p>	<p>3 V_{rms} 150 kHz to 80 MHz</p> <p>3 V/m 80 MHz to 2.5 GHz</p>	<p>3 V_{rms}</p> <p>3 V/m</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the BM90, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> <p>$d = 1.167 \sqrt{P}$ $d = 1.167 \sqrt{P}$ 80 MHz to 800 MHz $d = 2.333 \sqrt{P}$ 800 MHz to 2.5 GHz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b Interference may occur in the vicinity of equipment marked with the following symbol:</p> <p align="center">  </p>
<p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.</p> <p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the BM90 is used exceeds the applicable RF compliance level above, the BM90 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the BM90.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Table 4
For ME EQUIPMENT or ME SYSTEM that are not LIFE-SUPPORTING

Recommended separation distances between portable and mobile RF communications equipment and the BM90.

The BM90 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the BM90 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the BM90 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1.167 \sqrt{P}$	80 MHz to 800 MHz $d = 1.167 \sqrt{P}$	800 MHz to 2,5 GHz $d = 2.333 \sqrt{P}$
0,01	0.167	0.167	0.233
0,1	0.369	0.369	0.738
1	1.167	1.167	2.333
10	3.690	3.690	7.388
100	11.67	11.67	23.330

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.