# bewell<sup>®</sup> connect

MyTensio

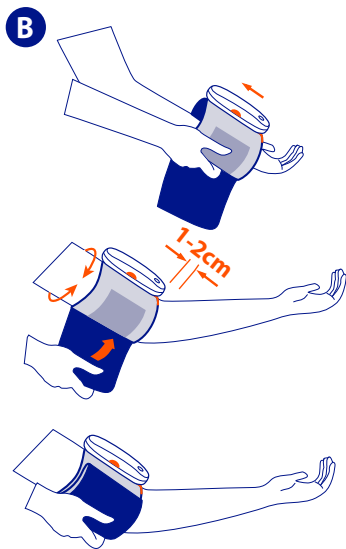
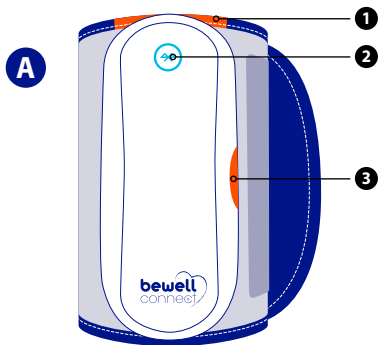
CE  
0197

BW-BA1

By Visionmed

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Log onto our website to find out how to install and use the BewellConnect® application  
[www.bewell-connect.com/install](http://www.bewell-connect.com/install)

*The manufacturer keeps the right to modify without any preliminary opinion technical specifications of the product.*

Dear customer,

Thank you for buying the automatic blood pressure armband BewellConnect®. We hope you enjoy using them and we recommend you read the instructions carefully to measure your blood pressure accurately.

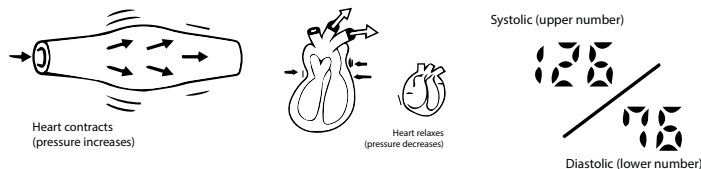
The easy-to-use appliance enables you to measure your blood pressure anytime, anywhere. In a matter of seconds, the maximum and minimum pressure and heart rate are measured and transferred by Bluetooth 4.0 using the oscillometric blood pressure measurement method.

*For further information about your blood pressure, PLEASE CONTACT YOUR DOCTOR.*

### 1. NOTES ON SAFETY

- Use this appliance only for the use for which it is designed as described in this manual.
- This appliance may be used for personal use at home. Do not use the unit for other purpose.
- Use this appliance in an ambient temperature range of 10 to 40°C.
- Do not expose this appliance to extreme temperature conditions > 60°C ou < -20°C.
- Do not use this appliance at a relative humidity of more than 85%.
- This appliance must always be placed in a clean and dry place.
- Do not expose this appliance to sunlight or water.
- Do not expose this appliance to electric shocks.
- Do not use this appliance outdoors.
- Never drop the appliance.
- Follow the maintenance instructions specified in this manual.
- Do not attempt to open the device. In case of problems, contact your dealer.
- Do not leave this appliance within the reach of children.
- The measurement results are given for information. If in doubt about your results, please contact your doctor.
- This product should not encourage self-medication or adaptation of your treatment.
- Discontinue use of the device in case of anomalies or malfunction.
- This electrical medical equipment requires specific precautions regarding electromagnetic compatibility. It must be installed and used according to the electromagnetic information.
- Generally, portable and mobile RF communications equipment don't affect medical electrical equipment, but it can't be used in the strong electromagnetic interference environment: TV, microwave oven...
- This appliance is not designed for use by people (including children) with reduced physical, emotional or mental abilities or those lacking experience or knowledge unless used in the presence of someone responsible for their security, under surveillance or following instructions about how to use the appliance.


## 2. INFORMATION

Blood pressure is the force exerted on the walls of your blood vessels as blood flows through them. Your heart is like a pump. When it contracts, or beats, it sends a surge of blood through the blood vessels and pressure increases. This is called your systolic pressure. When your heart relaxes between beats, your blood pressure decreases. This is called your diastolic pressure. When a doctor takes your blood pressure, he or she measures both your systolic and diastolic pressures and records them as numbers. For example, if your blood pressure reading is 126/76 (126 over 76), your systolic is 126 and your diastolic is 76. The numbers are calculated in millimetres of mercury and recorded as 126/76 mm Hg.

These two numbers provide important information about your health. The more difficult it is for your blood to flow through your blood vessels, the higher both numbers will be. When blood pressure is consistently above normal it is called hypertension (High blood pressure). People with a condition that causes circulatory problems (diabetes, kidney disease, arteriosclerosis, or poor peripheral circulation) may get lower readings with upper arm type monitor than with a blood pressure monitor that is used on the upper arm. Please consult your doctor to determine if your upper arm blood pressure accurately reflects your actual blood pressure.

### FLUCTUATION WITHIN A DAY (EXAMPLE: 35-YEAR OLD MALE)

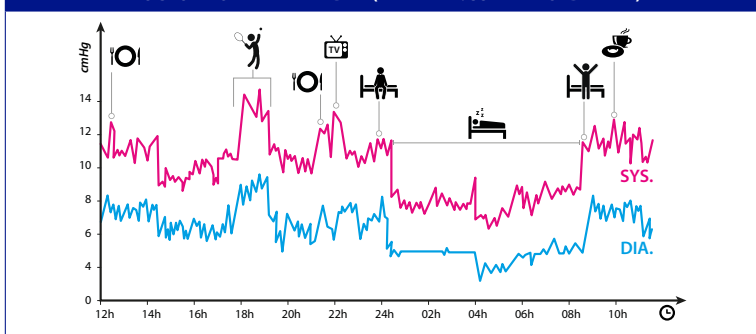Factors that may cause the blood pressure to fluctuate: breathing, exercise, mental stress, worries, environmental and/or temperature change, eating, talking, taking a bath, drinking alcohol, smoking, etc...

*Note : One or two readings will not provide a true indication of your normal blood pressure. It is very important to take regular, daily measurements and to keep accurate records. In partnership with your doctor, an accurate record of your blood pressure over a period of time can be a valuable aid in diagnosing and preventing potential health problems.*

The World Health Organisation (WHO) developed the following Blood Pressure Classification. This classification, however, is only a general guideline because blood pressure varies from person to person according to age, weight, and health status.

CONSULT YOUR DOCTOR TO DETERMINE YOUR NORMAL BLOOD PRESSURE.

### CATEGORIES FOR BLOOD PRESSURE LEVELS IN ADULTS (IN MMHG\*)

| Category             | Systolic (Top number) | Diastolic (Bottom number) | Color indication |
|----------------------|-----------------------|---------------------------|------------------|
| Optimal              | less than 120         | less than 80              | green            |
| Normal               | less than 130         | less than 85              | green            |
| Prehypertension      | 130 - 139             | 85 - 89                   | green            |
| Hypertension stage 1 | 140 - 159             | 90 - 99                   | yellow           |
| Hypertension stage 2 | 160 - 179             | 100 - 109                 | orange           |
| Hypertension stage 3 | higher than 180       | higher than 110           | red              |

*Note : There is not an universally accepted definition of hypotension (low blood pressure), but a systolic pressure below 99 mmHg is usually regarded as hypotension.*

*\* millimeters of mercury*

## 3. FEATURES

- Oscillometric method
- Pressure sensor inserted in silicon
- Automatic blood pressure and pulse measurement
- Heart arrhythmia detection
- Easy to use (one button to take measurements)

- Easy-to-adjust stiff armband
- Automatic switch-off
- Data transfer to Bluetooth 4.0 compatible mobiles/tablets.

## 4. USE

### 4-1. DESCRIPTION

View diagram A page 2

- ① Orange Marker
- ② Bluetooth LED
- ③ ON/START button

### 4-2. PRECAUTIONS BEFORE USE

- Do not confuse self-monitoring with self-diagnoses. Blood pressure measurements should only be interpreted by a health professional who is familiar with your medical history.
- If you are taking medication, consult with your physician to determine the most appropriate time to measure your blood pressure. NEVER change a prescribed medication without first consulting with your physician.
- This device is designed for adults to use if this device is used on a child patient, old people or toddlers, consult with your physician.
- This product should not encourage self-medication or adaptation of your treatment.
- If there is overmuch pressure or you feel uncomfortable, please press ON/START button immediately for quick deflation.
- Do not use this unit in a moving vehicle; this may result in erroneous measurement.
- Blood pressure measurements determined by this monitor are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard Institute, Electronic or automated sphygmomanometers.

### 4-3. SUGGESTIONS BEFORE MEASUREMENT

- The first time you use the appliance, please charge up the battery using the USB port provided. Connect the USB cable provided to your computer to charge up the appliance. When the appliance is fully charged, please unplug the USB cable.
- Avoid eating, smoking or exercising 30 minutes before taking measurements. Relax for 15 minutes before taking blood pressure.
- Stress increases blood pressure. Avoid taking measurements during stressful periods.
- With the exception of contraindications (fistula, ganglion), we recommend taking blood pressure on your left arm (right handed) or right arm (left handed). Please speak to your doctor if you have any questions.
- You must take blood pressure in a calm environment and you must be relaxed and seated with your arm on a table.
- Remain stationary and do not speak whilst taking blood pressure.

## 4-4. SELF BLOOD PRESSURE MONITORING

- MAM (Microlife Average Mode) : MAM technology automatically takes at least three measurements in a matter of seconds. Repetition is essential when self-monitoring blood pressure as isolated measurements are often unreliable for a number of reasons due to huge variations in blood pressure and its susceptibility to factors such as : recent effort, stress and even breathing. Taking 3 measurements in a row and the average provides more reliable home measurements by a non-professional. This method comes recommended by the SFHTA (French Hypertension Society).
- The rule of three : measurements recommended whilst seated, at rest, 3 days in a row :  
In the morning before taking medication, 3 measurements in a row (MAM)  
In the evening after taking medication, 3 measurements in a row (MAM)  
An isolated measurement may not be significant especially in terms of blood pressure self-monitoring.
- Blood pressure self-measurement : the surveillance targets for a known subject with high blood pressure self-measuring at home are different and lower than those of the WHO (World Health Organisation sets the hypertension grade). These targets depend on age and can vary depending on your health.  
General study : SBP\* < 135 mmHg and DBP\* < 85 mmHg  
Age > years old : SBP\* < 150 mmHg  
Chronic kidney disease : SBP\* < 130 mmHg and DBP\* < 80 mmHg  
\* SBP : Systolic Blood Pressure - DBP: Diastolic Blood Pressure

*The aforementioned figures are provided for reference only and need to be rechecked by the doctor. It is important to check your targets in the event of serious disease or pregnancy.*

### 4-5. INSTRUCTIONS

See diagram B page 2

*Do not take measurements over clothing. Remove jewellery from the arm you are measuring before taking measurement (watch, ring, bracelet etc.).*

- Sit down on a chair with your feet flat on the floor (do not cross your legs).
- Ensure your mobile or tablet with the BewellConnect® app is within reach.
- Put the blood pressure monitor around the arm with the orange marker at the bottom.
- It must be 2-3cm from elbow crease. The orange marker must lie on the inside arm's artery.
- Close the velcro (not too tight).
- Press ON/START to turn the blood pressure monitor on. The blue LED lights up.
- Put your arm on the table, palm up so that the blood pressure monitor is level with the heart.
- Remain seated and calm for two minutes before taking measurement.

*Note : For more reliable results, take measurements on the same arm.*

#### 4-6. DOWNLOAD APPLICATION

The following information may be altered.

Download the BewellConnect® app on your mobile or tablet :

- App store or Google Play

- scan the QR Code on the side of the box

Then click on the MyTensio icon.

#### 4-7. USER GUIDE

Launch the BewellConnect® app and click on MyTensio.

- Press START on the screen (app) to begin measurement.

- The armband inflates automatically. 3 blood pressure measurements are repeated automatically.

- The armband inflates and deflates three times in a row based on the MyTensio app procedure.

Do not remove the armband before the 3 blood pressure measurements. The measurements are automatically sent to the app via Bluetooth 4.0.

- Once the measurements have been taken, the average of the 3 is displayed on the app screen.

- Press the ON/START to turn it off. The unit automatically turns off after approximately 1 minute.

**Notes :**

*- The measurement will be too low if the armband is higher than the heart or too high if it is lower than the heart.*

*- If you want to stop the blood pressure measurement at any point, press the ON/START to turn it off.*

*- Speak to your doctor if you have any questions about using the blood pressure monitor.*

#### 4-8. MEASUREMENT RESULTS

The app automatically evaluates the MAM (Microlife Average Mode) and uses WHO colour grading to provide the result.

#### 5. HOW TO MAINTAIN AND STORE THE UNIT

- Use a dry soft cloth to clean the unit. If necessary, use a cloth lightly dampened with tap water.

- Do not use alcohol, benzene, thinner or other harsh chemicals to clean the device or cuff.

- Before using please wash your hand. Do not wash or wet the cuff.

- Avoid placing the unit where direct sunlight, high humidity or dust.

- Roll up the pressure armband and store in its case.

#### 6. HOW TO PREVENT A MALFUNCTION

- Do not drop the device and avoid sudden jars or shocks.

- Do not insert other objects into any holes.

- Do not attempt to disassemble the unit.

- Do not put the unit in water.

- The monitor might not meet its performance specifications or cause safety hazard if stored or used outside the specified temperature and humidity ranges in specifications.

#### 7. SPECIFICATIONS

|                          |  |
|--------------------------|--|
| Product name             | BewellConnect® MyTensio  |
| Model                    | BW-BA1   |
| Classification           | Classe IIa   |
| Measuring method         | Oscillometric System   |
| Dimensions               | 55 x 23 x 135 mm - Cuff 22-36 cm   |
| Weight                   | 175g   |
| Measuring range          | Cuff pressure : 0~299 mmHg<br>Pulse : 40~180 beat/min                                |
| Accuracy                 | Pressure : ± 3mmHg<br>Pulse : ± 5%   |
| Inflation                | Electric pump inflation  |
| Deflation                | Pressure release valve   |
| Function                 | IHB (Irregular HeartBeat detector) :Irregular heartbeat detection                    |
| Working Temp. & humidity | Temp : 10~40°C<br>Humidity : 15~90%RH  |
| Storage Temp. & humidity | Temp : -20~60°C<br>Humidity : 10~95%RH<br>Pressure : 80~105Kpa                       |
| Technology               | Bluetooth 4.0  |
| Bluetooth LED            | Appliance on : Lit LED<br>Low battery : Flashing LED<br>Charging : Lit LED           |
| Battery                  | Li-ion Rechargeable battery (650mAh)   |
| Battery life             | ± 250 measurements   |
| USB                      | USB 2.0 port   |
| Voltage                  | 3.7V   |
| Supplied with            | 22~36cm armband, instruction manual, quick user guide, storage case, 800mm USB cable |

## 8. TROUBLESHOOTING

If you have one of the following problems, please refer to this breakdown service guide to help resolve the problem. If the problem persists, please contact our customer service.

| PROBLEM   | POSSIBLE CAUSE   | SOLUTION  |
|---|--|---|
| Abnormal results                                      | The bracelet is not positioned correctly or not tight enough             | <i>Put the bracelet on correctly and try again</i>                                      |
|   | Incorrect position during measurement                                    | <i>Reread "Instructions" and retry</i>  |
|   | You spoke, moved or were agitated, excited or nervous during measurement | <i>Try again when you have calmed down. Do not speak or move during test.</i>           |
|   | Arrhythmia   | <i>The appliance is not suitable for people suffering from serious heart arrhythmia</i> |
| The blue Bluetooth LED is not lighting up or flashing | Low battery  | <i>Charge up appliance with USB cable</i>   |
| No data transfer to your mobile/tablet                | Bluetooth connection deactivated   | <i>Activate bluetooth on your mobile/tablet</i>   |
|   | Incompatible mobile or tablet  | <i>Check your mobile or tablet has Bluetooth 4.0</i>                                    |

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**EN:** Visiomed\* will repair or replace this product free of charge in the case of defective parts or manufacturing defects, in accordance with the conditions mentioned below as follows:

**DURATION:** 24 MONTHS RETURN TO WORKSHOP

**LIMITS AND EXCLUSIONS:** This guarantee concerns only the original final purchaser. A purchase invoice, or another proof of purchase, with this guarantee card will be required to obtain an after-sales service, in accordance with this guarantee. This guarantee card will not be extended to another person only the original final purchaser. This guarantee becomes void if the serial numbers on the product are modified, replaced, illegible, absent, or if repair has been carried out by a service not approved, including the user.

This guarantee covers only the defects of the material or parts, occurring during normal use of the product. It does not cover the damage caused during the transport of the apparatus, causes due to repairs being carried out by the distributor, by any modifications undertaken, any connection of equipment not approved by Visiomed\*, or causes contrary to those written in the user manual or notice. Moreover, the present guarantee does not cover damage due to falls, bad handling, bad installations, damage by fire, floods, lightning, or any other natural disaster. This guarantee does not cover the packing of the material, the accessories, the defects caused by commercial exposure of the product, show room, sale space, demonstration etc... Normal maintenance, cleaning and the replacement of parts where wear is normal, are not covered by the terms of this guarantee. Visiomed\* and its representatives and agents will not in any case be held responsible for any damage and consecutive damages due to the mishandling of this product. This guarantee is the only valid one at Visiomed\*, any other guarantee (commercial guarantee) except this one will not be taken into account.

**IMPORTANT:** During the guarantee period if you are dissatisfied with the repairs of this product, please contact the Visiomed\* customer service.

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**EN:** Disposal of Waste Equipment by Users in Private Households in the European Union.

The symbol on the product or on its packaging indicates that this product must not be disposed of with your other household waste. Instead, it is your responsibility to dispose of your waste equipment by handing it over to a designated collection point for the recycling of waste electrical and electronic equipment. The separate collection and recycling of your waste equipment at the time of disposal will help to conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment. For more information about where you can drop off your waste equipment for recycling, please contact your local city office or your household waste disposal service.

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**EN:** BewellConnect<sup>®</sup> is a brand registered by Visiomed Group SA. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registered design. The logos and trademarks are the property of their respective owners.

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 Made in P. R. C.  
Nexmed Technology Co., LTD.  
2<sup>nd</sup> Floor of N°.1 building, Jia An  
Technological Industrial Park  
67 District, Bao An  
518101 Shenzhen - China

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© 06/2015 BW-BA1
Visiomed SAS France