boso Carat synchro

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D 2

Manual

GB 20

Mode d'emploi

F 38

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I 56

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E 74

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
Table of contents

Overview of device	21
Contents of package	21
Quick start guide	22
Introduction	23
Using the device for the first time	24
 General information on measuring blood pressure	24
Preparing to measure blood pressure	25
Information on cuff size	25
Fitting the cuffs	26
Measuring blood pressure	28
Interrupting a measurement	29
Results display	30
Error message	32
Cleaning and disinfection	33
Warranty	33
Customer service / Disposal	34
Technical data	35
Metrology test instructions (only for trained specialists)	36

Overview of device

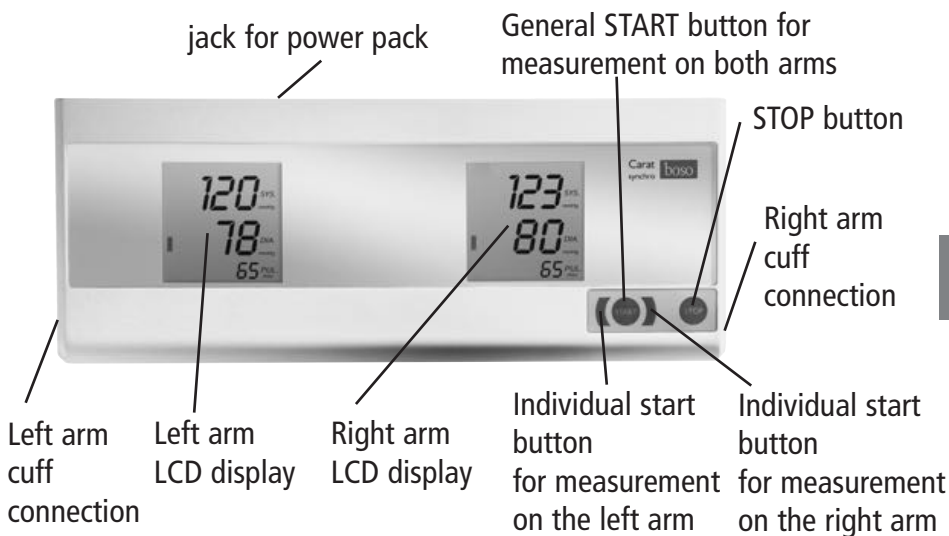Device ready for use

Measurement being performed

see page 28

Dysrhythmia during measurement

see pages 30, 32

Patient moved during measurement

see pages 30, 32

Contents of package

1 **boso Carat synchro**
blood pressure instrument

1 DC 6 volt
power pack

1 user manual

2 universal cuffs
CA 04

Quick start guide

- ➔ Connect power pack
- ➔ Connect cuffs to instrument (p. 25)
- ➔ Fit cuffs to patient (p. 26)
-  ➔ Start performing a measurement on both arms by pressing the general START button. (p. 28)

Systole, diastole and pulse results are displayed after the end of the measurement (p. 30)

 This quick start guide is not a substitute for the detailed information on the operation and safety of your blood pressure instrument contained in the manual.

Therefore, please ensure that you read the manual.

Introduction

Dear customer,

Thank you for purchasing this boso blood pressure measuring system. The boso brand stands for top quality and accuracy, and is the choice of the professionals: 77% of German doctors currently use boso blood pressure instruments.

This device has undergone our strict quality control procedures and will be a reliable partner in monitoring your patients' blood pressure.

 Please read this manual carefully before using the system for the first time, because blood pressure measurements can only be taken properly if the system is used correctly.

The "➡" symbol in this manual indicates action to be taken by the user.

Important information is highlighted by the  symbol.

Please contact your specialist supplier or the manufacturer for help in preparing for first use, using the device or maintenance (contact details are given on the inside back cover of this manual).

Notify the manufacturer immediately of any unexpected operational status or any event which has or could have caused damage to health.

If you sell the system, please include this manual.

In accordance with the Medicinal Products Operator Decree, metrology tests (see page 36) must be conducted at regular intervals when the system is used in a medical context.

The boso Carat synchro blood pressure measuring system is suitable for patients with an upper arm circumference between 16 and 48 cm (see p. 25 for cuff sizes).

It is not suitable for infants. The boso Carat synchro system allows you to measure blood pressure on both arms at the same time, and therefore to find out in which arm blood pressure is higher, and the difference between the sides.

Using the device for the first time

➔ Plug the power pack supplied into the socket on the back of the instrument.

Only use the boso power pack (**order no. 410-7-156**).

This power pack has been set to the correct voltage, stabilised and correctly polarised. Power packs from alternative sources can damage the electronics and result in loss of the manufacturer's warranty. They can also cause fire.

GB

General information on measuring blood pressure

1. Dysrhythmia can impair the accuracy of the system or lead to incorrect results (see page 30).
2. Incorrect results may be produced for patients with a weak pulse (e.g. patients with pacemakers). The blood pressure instrument does not affect the pacemaker.
3. Operation in the vicinity of strong electromagnetic fields (e.g. radiation devices, mobile phones) can cause malfunction. Medical electrical equipment is subject to specific precautions with regard to electromagnetic compatibility (EMC).

More information will be provided by the manufacturer on request (contact details are given on the inside back cover of this manual).

Preparing to measure blood pressure

Information on cuff size

⚠ The system may only be used with the following cuffs. These must be selected in accordance with the upper arm circumference printed on them.

Type	Arm circumference	Item number
CA01	22 – 32 cm	143-4-764
CA02	32 – 48 cm	143-4-756
CA03	16 – 22 cm	141-4-771
CA04	22 – 42 cm	143-4-765*

* 2 universal cuffs are supplied as standard.

Connect the cuffs to the system by pushing the plugs into the sockets on the right and left sides of the system until they are firmly in place. (see figure 1)

figure 1

Preparing to measure blood pressure

Before the measurement, the patient should:

- ➔ rest for 5 minutes

During the measurement, the patient should:

- ➔ sit in a relaxed position
- ➔ not cross his/her legs
- ➔ place his/her feet flat on the floor
- ➔ have his/her back and arms supported
- ➔ not move or speak

GB

Fitting the cuffs

The measurement must be performed on the bare upper arm.

! Alf the patient is wearing a tight-sleeved top, make sure that rolling up the sleeves does not constrict the arms. It may be necessary for the patient to remove the top before the measurement is taken.

- ➔ Open the cuff to form a ring shape and slip it over the upper arm until the lower edge of the cuff reaches a point about 2-3 cm above the elbow. The cuff must be positioned so that the marking lies on the artery (see figure 2).

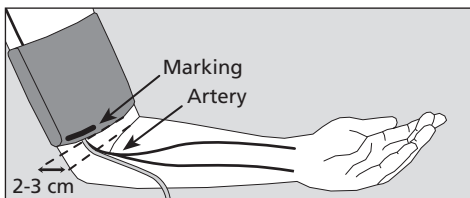
figure 2

! The cuff must not be placed on top of wounds, as this could lead to further injuries.

! Ensure that the cuff is not placed on an arm of which the arteries or veins are undergoing or have undergone medical treatment, e.g. a shunt.

Preparing to measure blood pressure

⚠ In the case of women who have had a mastectomy, do not place the cuff on the arm on the side on which the operation took place.

⚠ Other medical devices being used on the same arm at the same time may malfunction during measurement.

⚠ The cuff must not be too tight; you should still be able to fit a couple of fingers between the arm and the cuff.

⚠ The metal bracket must **never** lie on top of the artery as this could lead to incorrect results.

➔ Pass the cuff through the metal bracket, pull on the end to tighten, and wrap it around the upper arm. Press the Velcro gently to secure.

➔ Ask the patient to place his/her arms in the cuffs on the table, slightly bent and in a relaxed position, with the cuffs at heart height.

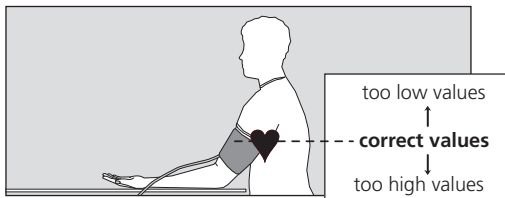
figure 3

⚠ Make sure that the air tubes are not kinked during measurement. The resulting blood congestion could cause injury.

⚠ The process of taking blood pressure must not stop the circulation for an unnecessarily long time (more than 2 minutes). If the system malfunctions, remove the cuffs from the arms.

Measuring blood pressure

- ➔ Press the general START button to start measuring blood pressure in both arms at the same time.

 From now on the patient should keep his/her arms completely still and not speak.

GB

- ➔ To measure blood pressure in the right or left arm only, press the appropriate individual start button.

All the display elements of the LCD display will light up briefly to check that they are working properly (see figure 4).

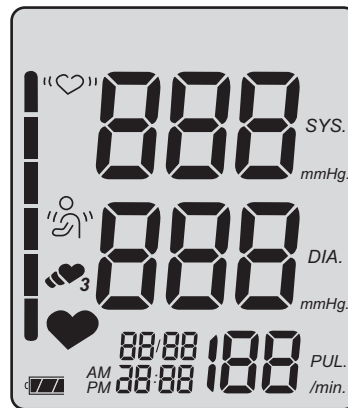
figure 4

The pumps start to inflate the cuff to the required pressure.
Increasing cuff pressure is displayed.

When the **required pressure** has been reached, the pumps switch off and air is slowly released from the cuffs.

Measuring blood pressure

Interrupting a measurement:

A measurement can be interrupted at any time by pressing the STOP button. This automatically deflates the cuffs.

Falling cuff pressure and the flashing  - symbol appear during measurement (see figure 5).

figure 5

If the  symbol appears during measurement, please hold the arm still.

Results display

The integrated valves automatically open to rapidly deflate the cuffs once the measurement is complete.

The blood pressure values measured systole, diastole, pulse) are displayed (see figure 6).

GB

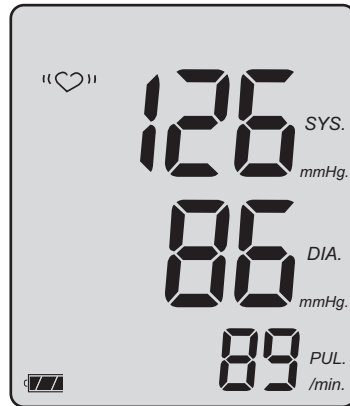
figure 6

Incorrect measurements are indicated by the "Err" display. (see page 45)

You are advised to repeat the measurement if the "♡" symbol appears, keeping the arms completely still. If this symbol appears frequently while the arms are still, this may indicate dysrhythmia.

⚠ It is not possible to detect the difference between sides when the symbol "♡" is displayed.

The symbol "👤" indicates that the arm moved during measurement. You are advised to take another measurement.

Blood pressure is a dynamic parameter and can be affected by the patient's position, i.e. whether he or she is sitting, standing or lying down, by movement before or during the measurement, and by his or her physical state (stress, illness, etc.).

➡ Repeat the measurement if the result is obviously wrong.

Press the appropriate START button again to repeat a measurement.

Results display

Leave at least two minutes between two measurements.

Excessively frequent measurements can cause injury as a result of impaired blood flow.

➔ Remove the cuffs from the patient's arms if you do not want to perform another measurement.

Error message

If problems occur during measurement, an error message is displayed instead of the result.

Meaning of error messages:

Meaning of error messages:	Action:
  Cuff pressure fluctuates too much while the measurement is being taken No measurable pulsations Systole/diastole difference too small: Systole/diastole ≤ 10 mmHg	Keep arm still Check cuff position (see page 27) and repeat measurement. Check cuff position (see page 27) and repeat measurement.
 Error during inflation	Cuff may be too loose, tighten cuff (see page 27)
 If the "♥" symbol appears frequently while the arms are still, this may indicate dysrhythmia.	You are advised to repeat the measurement, keeping the arms completely still.
 The "👤" symbol indicates that the arm moved during measurement.	You are advised to repeat the measurement, keeping the arms completely still.

Cleaning and disinfection

Cleaning the device:

Use a soft, dry cloth to clean the device

Cleaning the cuffs:

Small spots on the cuffs can be removed carefully using ordinary washing-up liquid.

Disinfection:

We recommend using antiseptic liquid disinfectant (Schülke & Mayr) to disinfect the device by wiping it, leaving the disinfectant on for at least 5 minutes. We recommend a disinfectant spray to disinfect the velcro on the cuffs. Ensure that the cuffs are regularly cleaned and disinfected, especially if the system is being used by different patients.

Warranty

We provide a two-year manufacturer's warranty on this product from the date of purchase. The date of purchase must be shown by the invoice. Defects due to material or manufacturing shortcomings are repaired free of charge within the warranty period. If parts are replaced under warranty, the warranty period is extended only for those parts, not for the entire device.

The warranty does not cover damage due to wear and tear (e.g. of the cuffs), transport damage or any damage caused by incorrect use (e.g. failure to comply with the instructions for use) or by intervention by unauthorised parties. The warranty does not give you any claim for damages against us. This does not affect the purchaser's right to make a claim on the grounds of defects as set out in § 437 of the German Civil Code.

If you wish to make use of the warranty, you must send the device together with the original receipt to:

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Customer service / Disposal

Customer service

Work under warranty and repairs must be carried out by trained and approved staff. The device must not be altered without the manufacturer's consent. Pack the device carefully and send it to your approved specialist dealer or directly to the following address, making sure that you have paid for postage in full:

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Disposal

The measuring system must not be discarded with household waste.

Once it has reached the end of its useful life it must be returned to the manufacturer for disposal.

It may not be disposed of via local authority collection facilities for used electrical equipment.

Technical data

Principle of measurement:	Oscillometric
Measurement range:	40 to 280 mmHg, 30 to 190 pulses/min.
Cuff pressure:	0 to 300 mmHg
Display:	LCD
Operating conditions:	Ambient temperature +10°C to +40°C rel. air humidity 15 to 85%
Storage conditions:	Ambient temperature -10°C to +60°C rel. air humidity 15 to 85%
Power supply:	Power pack DC 6 V, order no. 410-7-156
Weight:	1,3 kg
Dimensions (WxHxD):	355 mm x 82 mm x 145 mm
Classification:	Protection class II (□) type BF (⚡)
Protection against foreign objects:	IP20
Clinical test (DIN 58130):	Accuracy of measurement complies with the requirements of EN 1060 part
maximum cuff pressure measurement error:	± 3 mmHg (max. difference left-right: ± 1 mmHg)
maximum pulse display measurement error:	± 5 %
Applicable standards:	EN 1060, part 1: "Non-invasive sphygmomanometers - General requirements" and part 3: Supplementary requirements for electro-mechanical blood pressure measuring systems".

Metrology test instructions (only for trained specialists)

Metrology tests must be carried out at least once every two years by the following bodies or individuals:

- the manufacturer
- the authority responsible for metrology
- individuals meeting the requirements of the Medicinal Products Operator Decree.

GB

A) Function test

Function tests of the measuring system can only be performed on a human subject or a suitable simulator.

B) Test of pressure circuit tightness and pressure display deviation N.B.

- a) if pressure exceeds 320 mmHg in measuring mode, the quick-release valve responds and opens the pressure circuit. If pressure exceeds 320 mmHg in test mode, the results display flashes.
- b) A pump ball must also be introduced into the pressure circuit for the tests.

Test

- 1.) Disconnect the power pack.
- 2.) Press and hold both individual start buttons and connect the power pack.
- 3.) Release the individual start buttons.
The system is now in test mode, and the current pressure is displayed in the SYS and DIA fields.
- 4.) Carry out the pressure display deviation test and pressure circuit tightness test in the usual way, taking care to ensure that the cuff set-up time is at least 30 seconds.
Maximum difference left-right of pressure indication: ± 1 mmHg.
- 5.) Press the STOP button to switch off the measuring system.

C) Safeguard

As a safeguard, the upper and lower parts of the housing can be joined with a safety seal; as an alternative, the hole for the housing connection screw on the base of the system can be sealed.

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BOSCH + SOHN GmbH u. Co. KG