

EASYMAX[®]

MU Self-Monitoring Blood Glucose System

User's Manual

(Individual Version)



**Please read this User's Manual thoroughly before using
your blood glucose meter**

Dear EASYMAX[®] MU SMBG System Owner,

Thank you for using the **EASYMAX[®] MU** Self-Monitoring Blood Glucose (SMBG) System. We designed this system to be dependable, easy-to-use, compact, lightweight and portable to help you monitor your blood glucose on a regular basis.

Please read this manual thoroughly before you begin testing. This manual provides you and your diabetes care team with important information and step-by-step direction to use the **EASYMAX[®] MU** Self-Monitoring Blood Glucose System. To start testing quickly, you can also refer to the Quick Reference Guide.

Thanks again for choosing the **EASYMAX[®] MU** SMBG.

Intended Use

The **EASYMAX[®] MU Self Monitoring Blood Glucose Test System** is intended for the quantitative measurement of glucose in venous whole blood or fresh capillary whole blood from fingertips, palm and forearm. Testing is done outside the body (In Vitro diagnostic use). It is indicated for self-testing (over the counter [OTC]) by persons with diabetes, or in clinical settings by healthcare professionals, as an aid to monitor the effectiveness of diabetes control.

Standard Accessories

Your new **EASYMAX[®] MU** Blood Glucose meter and accessories work together to measure the amount of glucose in your blood. The system includes:

- **EASYMAX[®] MU Blood Glucose Meter**
- **Alkaline Battery (2 ct.)**
- **Glucose Test Strips (10 pcs)**
- **Lancets (10 pcs)**
- **Lancing device**
- **AST Lancing Device Cap**
- **User's Manual**
- **Quick Reference Guide**
- **Test Strip Instructions**
- **Self-Test Log Book**
- **Normal Control Solution**
- **Glucose Control Solution Instructions**
- **Carrying Case**

Optional Accessories

- **Low Control Solution**
- **High Control Solution**



1. Normal control solution is included with the system.
2. Low control solution and High control solution are available. For purchase, please contact your healthcare provider.

Why is it so important to test blood glucose regularly?

Testing your blood glucose regularly can make a big difference in how you manage your diabetes every day. We have made this SMBG system as simple as possible to help you use it regularly. Your meter is easy to use, and you can adjust the lancing device for your comfort.

Do you need help?

If you have questions or need assistance, please contact your healthcare provider.



Although the EASYMAX[®] MU SMBG System is easy to use, you may need to consult with your healthcare professional (this may be your doctor, pharmacist or diabetes nurse educator) for instructions on how to use the system. Only the correct use of the system will ensure accurate results.

Important Information about Your New Meter

- **EASYMAX[®] MU** blood glucose meter is designed and approved for testing fresh capillary whole blood samples from your fingertips, palm and forearm. The meter is for in vitro diagnostic use ONLY (for testing outside the body). It should not be used to diagnose diabetes.
- **EASYMAX[®] MU** blood glucose meter can only be used with **EASYMAX[®] MU** Blood Glucose Test Strips. Other test strips will give inaccurate results.
- Testing is not valid for neonatal blood specimens.
- Do not disassemble the meter as this may cause damage to the components resulting in incorrect readings. Disassembling the meter will also void the warranty.
- Always keep the meter clean and store it in a safe place. Protect the meter from direct sunlight to ensure a longer lifespan.
- You should not store the meter and test strips in a car, bathroom, or refrigerator.
- Keep the meter, test strips and lancing device away from children and pets.
- You should not test critically ill patients with home-use blood glucose meters.
- Incorrect results may occur when performing the test. If you believe you are not feeling well, please contact your healthcare professional.
- Remove batteries if the meter will not be used for one month or more.

- Please dispose device according to the local rule of the disposition of electronic device / accessory waste.
- Warning for potential biohazard: Healthcare professionals using this system on multiple patients should be aware that all products or objects that come in contact with human blood, even after cleaning, should be handled as if capable of transmitting a viral disease.



- Consult with your healthcare professional before testing on your palm or forearm.
- Do not touch the strips with wet hands.
- Do not use expired strips (the expiration date is shown on the bottle.)
- Do not bend, cut or twist the strips.
- Altitude up to 3,048 meters above sea level has no effect on readings.

Health-Related Information


















- If you are experiencing dehydration, frequent urination, low blood pressure, shock or hyperosmolar hyperglycemic nonketotic coma (HHNKC), you may get a test result that is lower than what your blood glucose really is. If you think you are dehydrated, call your doctor right away.
- If you have followed the steps in the user's manual, but still have symptoms that do not seem to match your test results, or if you have questions, please contact your healthcare provider.
- Please read your test strip instructions carefully for additional health-related information.





Warning for potential biohazard









Healthcare professionals using this system on multiple patients should handle all products or objects in contact with human blood carefully to avoid transmitting viral disease, even after cleaning.

Explanation of Symbols

	Consult instructions for use		Caution
	Batch code		Do not reuse
	In vitro diagnostic medical device		1.5V(AAA) x 2 batteries only
	Use by		Temperature limitation
	Manufacturer		Catalogue number
	Serial number		Control
	Sufficient for		Authorized representative in the European Community
	Blood glucose test result in mg/dL		Blood glucose test result in mmol/L
	Green Dot / Duales System Deutschland GmbH (DSD)		

	<p>Separate collection for WEEE- Waste of electrical and electronic equipment</p>
	<p>This product meets the requirements of Directive 98/79/EC in vitro diagnostic medical devices</p>

Explanation of Symbols (continued)

	<p>Discard 6 months (180 days) after opening</p>		<p>Large LCD screen</p>
	<p>Hypo alarm</p>		<p>Keep away from sunlight</p>
	<p>5-seconds result</p>		<p>0.6 μ L blood volume</p>
	<p>Alternate site testing</p>		<p>Human-factor flow</p>

Explanation of Meter Symbols









00:00	Date (on the left side)	00:00	Time (on the right side)
M	Memory	888	Record/Average display
AM	AM (Before Noon)	PM	PM (After Noon)
000	Result		Apply Blood
	Control Solution		Temperature
	Application of blood or control solution	mmol/L mg/dL	Unit
	Insert a strip		Battery

Table of Contents

Chapter 1: Understanding Your Meter	13
The EASYMAX[®] MU Blood Glucose Meter	13
The EASYMAX[®] MU Accessories.....	14
Inserting Batteries.....	15
Setting The Time and Date.....	16
Using EASYMAX[®] MU Blood Glucose Test Strips	17
Chapter 2: Control Solution Testing	18
Why Run a Control Solution Test	18
About The Control Solutions	19
Running a Control Solution Test	20
Understanding Control Solution Test Results	22
Chapter 3: Testing Your Blood Glucose	24
Using the Lancing Device	24
Inserting a Lancet into the Lancing Device	25
Running a Blood Glucose Test with Blood from Your Fingertips.....	27
Alternate Site Testing (AST).....	29
Running a Blood Glucose Test with Blood from Your Forearm	31
Running a Blood Glucose Test with Blood from Your Palm	32
Discarding Used Lancets	33

Understanding Your Test Results.....	34
Unusual Test Results.....	34
Symptoms Of High Or Low Blood Glucose.....	35
Comparing Your Meter Result to Lab Results	36
Chapter 4: Meter Memory, Setup	37
Memory, Storing Test Results	37
Viewing and Deleting Test Results	38
Running With Your Computer	39
Chapter 5: Maintenance and Troubleshooting	40
Inserting Batteries	40
Cleaning Your Meter	41
Cleaning Your Lancing Device	41
Maintenance and Testing	42
Screen Messages and Troubleshooting	43
Chapter 6: Technical Information	47
Specifications	47
Limitations.....	48
Device Information	50
Warranty	51

Chapter 1: Understanding Your Meter

The EASYMAX[®] MU Blood Glucose Meter

Test Strip Slot-
Insert test strip here.

Display-
Shows results, messages, and results stored in memory.

Power Button-
Press to turn on/off the meter, and to confirm the setting.

Left (←) Button-
Press to enter memory, adjust setting, and scroll through results.

Strip Ejector-
Push the Strip Ejector to remove the strip.


Battery Door-
Flip open the battery door by pushing the tab in the direction of the arrow and pulling the door up.

Right (→) Button-
Press to enter memories, adjust setting, and scroll through results.



The EASYMAX[®] MU Accessories

Blood Glucose Test Strip

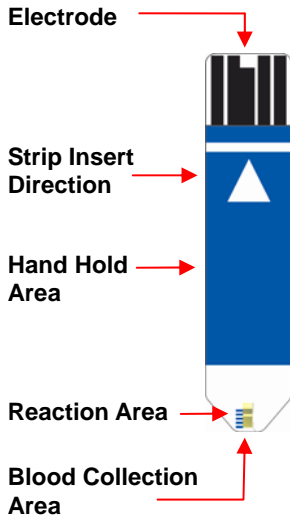

Test Strip Bottle Control Solution Bottle



Expiration Date

Lancing Device

Sliding barrel-

Pull on until it clicks and then release to enable lancing device.

Sleeve-
Hand Hold Area

Hub-

Unscrew or Recap the Cap.

Trigger button-

Press the trigger button to activate the lancing device

AST Lancing Device Cap-


Use this transparency cap for AST testing

Carrier-

Insert the lancet into the carrier

Adjustable tip-

Select the desired penetration depth

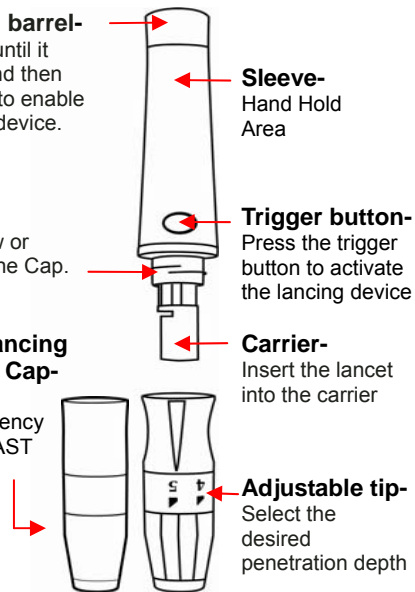


Inserting Batteries







1. Open the battery door on the back of the meter by pushing the tab in the direction of the arrow and pulling the door up.
2. Insert two batteries. The meter will beep to confirm the batteries are inserted correctly.
3. Put the battery door back in place and snap it closed.

Setting The Time and Date

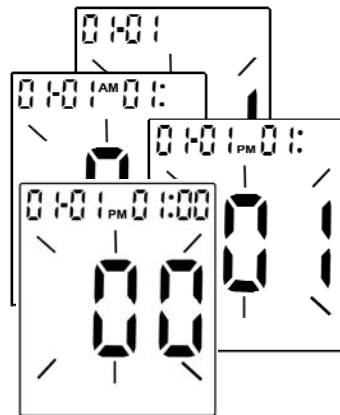Setting the current time and date in your meter is important if you use the meter memory.

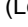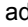

1. Press  (Power Button) and hold the button until the meter turns on.
2. The last 2-digits of the year flashes at the top of the display. Press  (Left Button) or  (Right Button) to adjust the year and press  (Power Button) to confirm the setting.
3. Repeat step 2 to set the date and time. The flashing field is the one you are currently setting.

Using EASYMAX[®] MU Blood Glucose Test Strips

- Use only with EASYMAX[®] MU Blood Glucose Meters.
- Keep the test strips in their original bottle.
- After you take a test strip out of the bottle, tightly close the bottle immediately. This keeps the test strips dry.
- Use the test strip within 3 minutes after taking it out of the bottle.
- The strip is for single use only. Do not reuse it.
- Record the date you open the test strip bottle. Be sure to check the expiration date on the test strip bottle. The test strip is good for six months from the date the bottle is opened or until the expiration date on the bottle, whichever comes first.
- Store the test strip bottle and your meter in a cool dry place.
- Store the test strips between 2°C ~30°C (36°F - 86°F). Do not freeze.
- Do not apply blood or control solution to the test strip before you insert it into the meter.
- Do not touch the test strip with wet hands. Do not bend, cut, or twist the test strips.

Chapter 2: Control Solution Testing

Why Run a Control Solution Test

We recommend that you run the **EASYMAX[®] MU** control test because it lets you know that your meter and test strips are working properly to give reliable results. You should run the control solution tests when:

- You use the **EASYMAX[®] MU** Blood Glucose Meter for the first time.
- You open a new bottle of test strips.
- You think the meter or test strips may be working incorrectly.
- You drop the meter.
- You have repeated a test and the test results are still lower or higher than expected.
- You are practicing the test procedure.




Professional users are instructed to follow federal, state, and local guidelines.



About The Control Solutions

- Use with **EASYMAX[®] MU** test strips.
- Write the date you opened the control solution bottle on the bottle label. The control solutions are good for three months from the date the bottle is opened or until the expiration date on the bottle, whichever comes first.
- Do not use a control solution that is past the expiration date.
- Control solutions can stain clothing. If you spill it, wash your clothes with soap and water.
- Close the bottle tightly after every use.
- Left over control solution should not be added back into the control bottle.
- Store control solution at room temperature, between 2°C~30°C (36°F - 86°F). Do not freeze.
- If you would like to purchase **EASYMAX[®] MU** Control Solutions, please contact your healthcare provider.

Running a Control Solution Test

You need the meter, a test strip, and control solution.


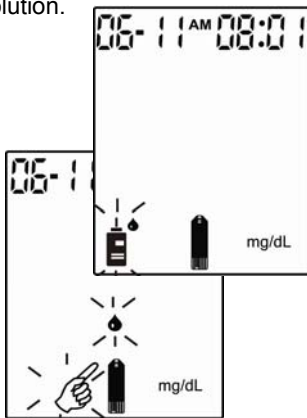


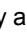1. Put a test strip into the meter in the direction of the arrow.
2. The meter turns on automatically and the icon of  shows itself. Press **←** (Left Button) to select the testing mode of Control Solution and the icon of  flashes.

3. Place the meter on a flat surface, like a table.
4. Remove the control solution bottle cap and wipe the tip of the bottle with a tissue.
5. Squeeze the bottle until a tiny drop forms at the tip of the bottle and place solution on top of bottle cap.



Setting the meter in control solution mode will prevent the control solution reading from being added to your memory and averages.

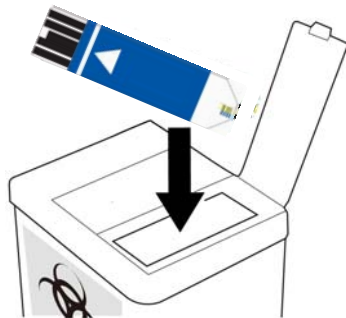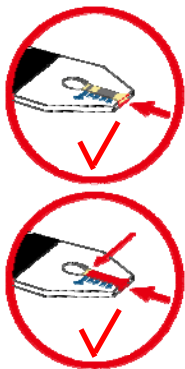

6. Touch the drop to the blood collection area at the tip of the test strip. Do not put control solution on the top of test strip. The meter starts to count down from 5 seconds and will show the results.
7. Do not remove the test strip until you confirm that the reading falls within the range printed on the test strip vial.
8. Push the Strip Ejector to eject the test strip.
9. Dispose properly.
10. Please check if the result on the display is the same as audible result.



Be aware that there are Normal and High ranges listed on your test strip bottle. The Normal and High refer to the three different control solutions that are available. Please be sure that you are looking at the correct range.

Understanding Control Solution Test Results

The label on your test strip bottle shows the acceptable ranges for the Control Solutions. The result you get should be inside the acceptable range for the appropriate control solution level. Make sure you compare the result to the correct level of control.

When the control solution result is inside the range on the test strip bottle, your test strips and your meter are working properly.

If your control solution result is not inside the acceptable range (printed on your test strip bottle), here are some things you can do to solve the problem:



In the Control Solution mode, control Solution values will NOT be included in the average memory.

Troubleshooting Check

- ✓ Was the test strip exposed to open air for a long period of time?
- ✓ Does test strip cap close tightly? Or was test strip cap left open?
- ✓ Is the meter functioning well?
- ✓ Is the control solution expired or contaminated?
- ✓ Were test strips and control solutions stored in cool, dry places?
- ✓ Did you follow the testing steps properly?

Action

If yes, repeat the control test with properly stored strips.

If the cap was not tight, or the bottle was left uncapped, open a new bottle of test strips. Do not reuse the strips from the affected bottle.

You can use the control solutions to verify the meter's functions.

If yes, replace with a new control solution to check the performance of SMBG system.

If no, repeat the control test with properly stored strips or control solutions.

Read Chapter 2 "Control Solution Testing" and test again. Stop using the meter if you continue to obtain the inaccurate results.

Chapter 3: Testing Your Blood Glucose

Using the Lancing Device


- The best depth setting is the lowest number that draws enough blood for a test. Try different settings to find the one that's right for you.
- Please do not share your lancing device with anyone. And always use a new, sterile lancet. Lancets are for one time use only.




Used test strips and lancets are considered bio-hazardous waste in accordance with local regulations and should be handled as if capable of transmitting infection. Users may discuss methods for disposing used test strips and lancets with their doctor.

Inserting a Lancet into the Lancing Device


You must first load the lancet into the lancing device to get it ready for use.

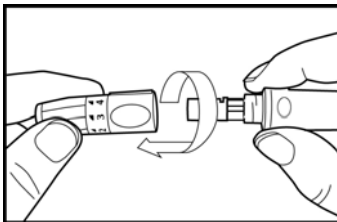

1. Unscrew the Cap.

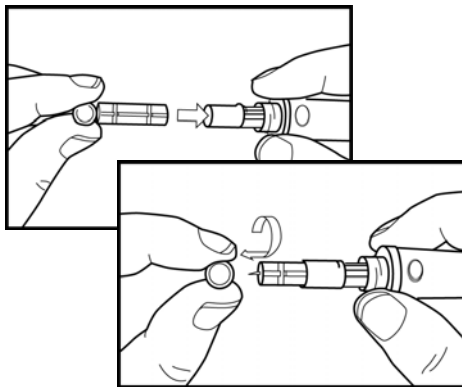


2. Insert the lancet into the lancing device firmly then twist off the protective cover.

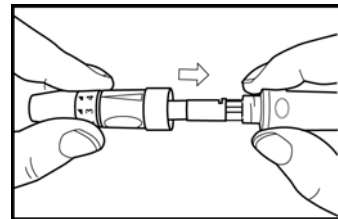


3. Recap the front cap.




Lancets are for use only and a new, sterile lancet should be used each time you perform a test.

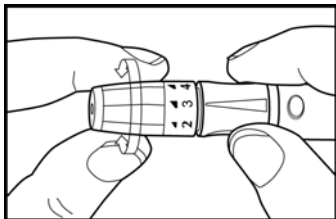

4. Select the desired penetration depth.

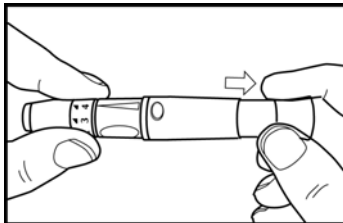

5. Pull on the sliding barrel of the lancing device until it clicks and then release.
Now the lancing device is ready.
Do not prick your finger until your meter and strip are prepared.

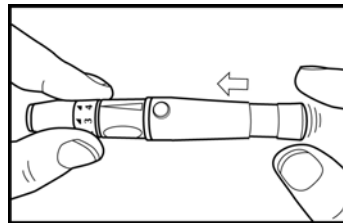


6. Set the lancing device aside until later in the test.





1. Select 1-2 for soft or thin skin, 3 for average, and 4-5 for thick or calloused skin.
2. Lancing device and lancets are not to be shared between users. Sharing lancing devices and lancets may transmit blood borne pathogens, such as viral hepatitis.

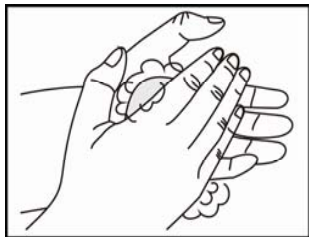
Running A Blood Glucose Test With Blood From Your Fingertips



1. Wash your hands with soap and warm water. Rinse and dry thoroughly.




2. Put a test strip into the meter in the direction of the arrow. The meter turns on automatically. The icon shows itself.
3. When the icons of  and  flashes on the display, the testing mode of blood drop is starting.

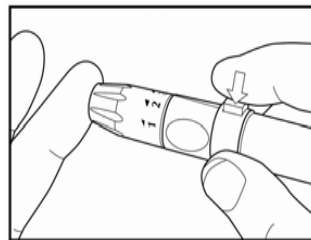


4. Place the lancing device against the pad of your finger. Press the trigger button to activate the lancing device.

You can also turn on the meter by inserting a test strip.

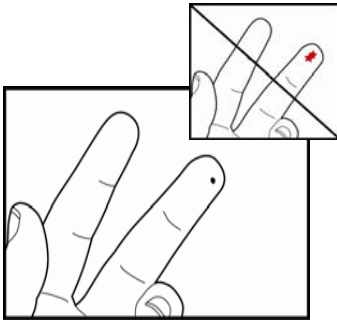

Apply blood to the edge of the test strip.

Do not apply blood on top of the test strip.

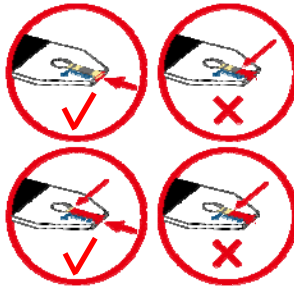

Be sure to get enough blood on strip to make it to the confirmation window.

Not enough blood on strip to get a test result.



- Gently squeeze and/or massage your fingertip until a round drop of blood on your fingertip.



- Touch the blood drop at the tip of the transparent window of the test strip. **Do not put blood on top of the strip.** Be sure to get enough blood on the strip's reaction zone. Otherwise, an inaccurate reading may result.

- The meter starts to count down from 5 seconds and then displays the test result.
- Push the Strip Ejector to eject the test strip & dispose of both strip & lancet properly. Remove the test strip and the meter turns off automatically.

Please make sure to apply blood when the blood drop appears on the display.

Alternative Site Testing (AST)

Understanding Alternative Site Testing

What is AST?

Besides the fingertip, you can test your palm or forearm.

What is the advantage of AST?

You have the option of testing other places on your body besides the fingertip.

Consult your health care professional before you begin using the palm or forearm for testing. Blood glucose test results obtained from your palm or forearm may differ significantly from fingertips samples.

We strongly recommend that you:

Do AST ONLY in the following intervals:

- In a pre-meal or fasting state (more than 2 hours since the last meal).
- Two hours or more after taking insulin.
- Two hours or more after exercise.

Do NOT use AST if:

- You think your blood glucose is low.
- You are unaware of hypoglycemia.
- Your AST results do not match the way you feel.
- You are testing for hyperglycemia.
- Your routine glucose results are often fluctuating.

Fingertip test only:

- If sick
- If blood glucose is low
- After exercising
- Two hours or less after eating
- When you have just taken insulin
- After injecting rapid-acting insulin (two hours or less)

AST Results:


- If the blood glucose test result from the alternate site test does not match how you feel, do a fingertip test to confirm the result again.
- Do NOT change your treatment just because of an alternative site result, do a fingertip test to confirm the result.
- If you often do not notice when your blood glucose is low, do a fingertip test.

Caution:



- Talk with your healthcare professional before you test with your palm or forearm.
- Do NOT ignore symptoms of high or low blood glucose.
- Fingertips samples are able to show the rapid change of glucose faster than forearm samples.
- Do NOT change your treatment just because of a result.

Running a Blood Glucose Test with Blood from Your Forearm



Please use the clear cap with the lancing device for AST testing.

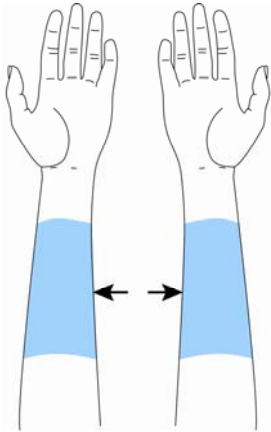

1. This graphic shows where the meter cleared for alternative site testing.


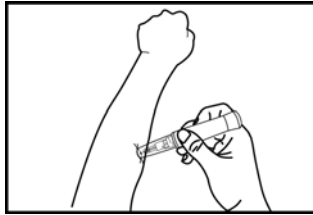
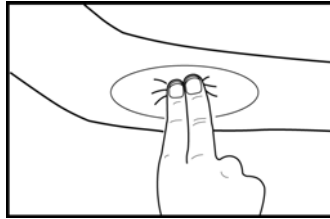


2. Massage the puncture area of forearm for a few seconds.
3. Press and hold the device with a clear adjustable tip against the forearm.

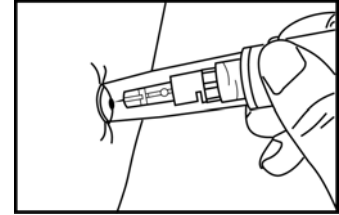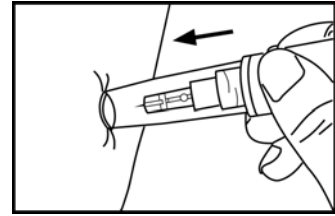

4. Press the trigger button to activate the lancing device.
5. Hold the device against forearm and increase pressure until the blood sample size is sufficient.

Running A Blood Glucose Test With Blood From Your Palm

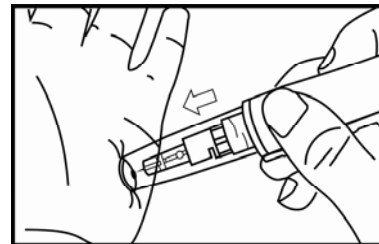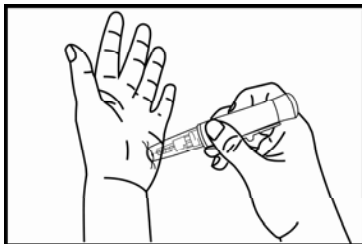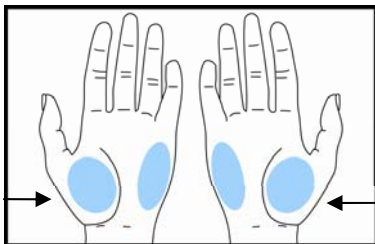


1. Massage the puncture area of palm for a few seconds.
2. Press and hold the device with a clear adjustable tip against the palm.
3. Press the trigger button to activate the lancing device.
4. Hold the device against palm and increase pressure until the blood sample size is sufficient.




Check with your healthcare professional before testing sites other than the fingertip.


Discarding Used Lancets

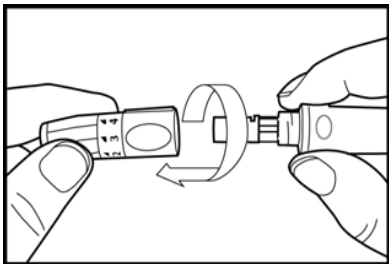

1. Unscrew and remove the cap.

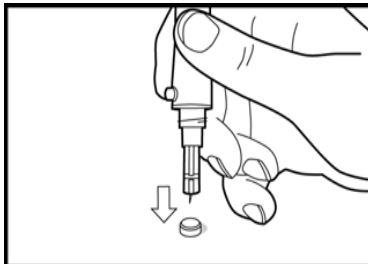

2. Without touching the used lancet, stick the lancet tip into its protective cover.

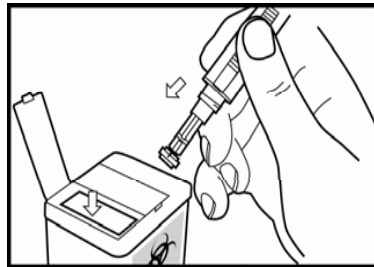

3. Pointing the lancing device toward a container for sharp or biohazard material, slide the ejection button up to release the covered lancet into the container.

Understanding Your Test Results

The **EASYMAX[®] MU** Blood Glucose test strips are whole-blood referenced and calibrated for easier comparison to lab results. The American Diabetes Association recommends a post-meal glucose level of less than 180 mg/dL (10 mmol/L) and a pre-meal glucose of 70–130 mg/dL (3.88-7.22 mmol/L)*.

* Reference: American Diabetes Association. Standards of medical care in diabetes-Table 10. Diabetes care. 2011; Vol. 34, Suppl. 1, S21.

Unusual Test Results

If your test result does not match the way you feel, please follow these steps:

1. Run a control test, Chapter 2, "Control Solution Testing."
2. Repeat a blood glucose test, Chapter 3, "Testing Your Blood glucose."
3. If your test results still do not reflect the way you feel, call your doctor immediately.



1. **Extremely high humidity may affect the test results. A relative humidity greater than 90% may cause inaccurate results.**
2. **Hematocrit below 20% may cause higher results. Hematocrit above 60% may cause lower results.**
3. **Some studies have shown that electromagnetic fields may affect results. Do not test near an operating microwave oven.**

Symptoms Of High Or Low Blood Glucose

Being aware of the symptoms of high or low blood glucose can help you understand your test results and decide what to do if they seem unusual. Here are the most common symptoms:

Greater than 240 mg/dL (13.33 mmol/L)

What It Means:

The test result is higher than reference normal range. (70-130 mg/dL or 3.89-7.22 mmol/L)

Symptoms:

Fatigue, increased appetite or thirst, frequent urination, blurred vision, headache, general aching, or vomiting.

What to Do:

- If you are experiencing any of these symptoms, test your blood glucose.
- If the result displayed is greater than 240 mg/dL (13.33 mmol/L) and you have symptoms of high blood glucose, contact your healthcare professional instantly.
- If the result does not match how you feel, follow the steps under "Unusual Test Results."

Below 60 mg/dL (3.33 mmol/L)

What It Means:

The test result is lower than reference normal range. (70-130 mg/dL or 3.89-7.22 mmol/L)

Symptoms:

Sweating, trembling, blurred vision, rapid heartbeat, tingling, or numbness around mouth or fingertips.

What to Do:

- If you are experiencing any of these symptoms, test your blood glucose.
- If the result displayed is below 60 mg/dL (3.33 mmol/L) and you have symptoms of low blood glucose, contact your healthcare professional instantly.
- If the result does not match how you feel, follow the steps under "Unusual Test Results."

Comparing Your Meter Result To A Lab Result

A common question is how the blood glucose results on your meter compare to the lab results. Your blood glucose can change quickly, especially after eating, taking medication, or exercising. If you test yourself in the morning, then go to the doctor's office for a blood glucose test. The results will probably not match, even if you are fasting. This is typically not a problem with your meter, it just means that time has elapsed and your blood glucose has changed.

If you want to compare your meter result to the lab result, you must be fasting. Bring your meter to the doctor's office, and test yourself by fingertips within five minutes of having blood drawn from your arm by a healthcare professional. Keep in mind that the lab could use different technology than **EASYMAX[®] MU** blood glucose meter, and that blood glucose meters for self testing generally read somewhat lower or higher than the lab result.

For accuracy and precision data and for important information on limitations, see the instructions that come with your test strips.

Chapter 4: Meter Memory, Setup


Memory, Storing Test Results


Your meter stores a maximum of 480 test results with the time and date of the test. You can review them at any time. When the memory is full, the oldest result is dropped as the latest is added, so it is very important to have the correct time and date set in the meter.









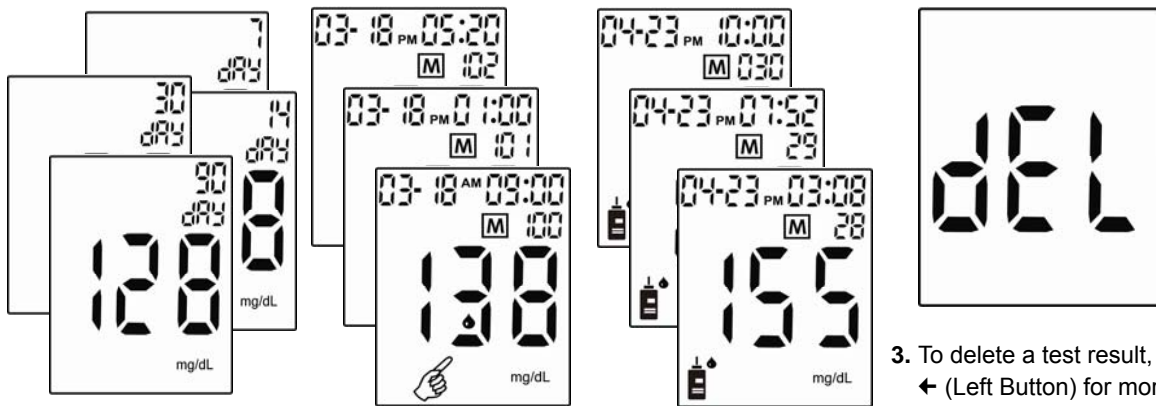


1. **Do not change your therapy based on one individual result in memory.**
2. **The memory is not lost when you replace the battery. You do need to check that the time and date are still correct. See Section "Setting the time and date" in Chapter 1.**
3. **Once 480 results are in memory, adding a new result causes the oldest one to be deleted.**
4. **In the Control Solution mode, control Solution values will NOT be included in the average memory.**

Viewing and Deleting Test Results

Each review requires the user to go back to the main screen (testing mode) by pressing the  (Power Button).



1. Press  (Left Button) or  (Right Button) to view averages of 7-/14-/30-/90-day test results first and then all results. Press  (Power Button) back to the testing mode.
2. Press  (Left Button) for more than 2 seconds to view the control solution records in order from 30 to 1. Press  (Power Button) back to the testing mode.
3. To delete a test result, press  (Left Button) for more than 2 seconds and display shows “dEL”, press  (Power Button) to confirm deletion.
4. Press  (Left Button) or  (Right Button) to keep reviewing the results.

Running With Your Computer

To Transfer data, the meter can be turned on or turned off.



1. Use the USB cable to connect to the meter and your computer. The display shows "PC".




2. The meter starts to transfer data to the "GlucoManager™" software.



3. When all data are uploaded from the meter to the PC, the meter shows "OK" and beeps.


Chapter 5: Maintenance and Troubleshooting


Inserting Batteries

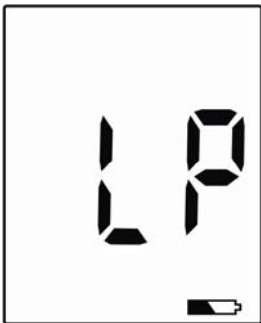

The meter uses two alkaline 1.5V (AAA) batteries. Batteries will normally last for more than 2000 tests. Other types of 1.5V (AAA) batteries are also acceptable, but the capacity of test times may differ. Insert the batteries when you first use the meter or replace with new batteries when the “LP” (Low Power) message and the low battery symbol appear on the display.

The meter will not turn on the first time batteries are inserted.

Please press and hold  (Power Button) or insert the test strip to turn your meter on.

The meter will turn off automatically. Or you can press and hold  (Power Button) to turn your meter off.

Low battery symbol



1. The meter won't delete earlier records after you replace batteries.
2. You should reset the time and date again after you replace the batteries.
3. 1.5V (AAA) x 2 batteries are available at most stores. You may take the old batteries with you for replacement.
4. Remove batteries when you will not be using the meter for one month or more.

Cleaning Your Meter

Caring for your **EASYMAX[®] MU** SMBG system does not require special cleaning. Please keep the meter free of dirt, dust, bloodstain, and water stains. Follow these guidelines carefully to help you get the best performance possible:

Do:

- Make sure the meter is turned off.
- Gently wipe the meter's surface with a soft cloth slightly dampened with ethanol (70~75%).

Do Not:

- Get any moisture in the test strip slot.
- Spray any cleaning solution directly onto the meter.
- Put the meter under water or liquid.
- Pour liquid into the meter.


Cleaning Your Lancing Device

- To clean the lancing device, wipe it with a soft cloth dampened with water and mild detergent. **DO NOT** place the entire device under water.
- To disinfect the cap after cleaning, place it in 70%-75% rubbing alcohol for 10 minutes at least once a week. Allow the cap to air-dry after disinfecting.

Maintenance and Testing



Your meter needs little or no maintenance with normal use. It automatically tests its own systems every time you turn it on and lets you know if something is wrong. (See "Screen Messages" and what to do about them.)

To make sure the display is working properly, turn off the meter. Press and hold  (Power Button) to see the complete display. All the indicators should be clear and look exactly like the picture to the left. If not, please contact your healthcare provider.

Screen Messages and Troubleshooting

Never make treatment decisions based on an error message. If you have any concerns, please contact your healthcare provider.

Message

What it means?

What to do?



Humidified / Used strips

The meter has detected a problem with the test strip.

Repeat the test with a new strip.

Refer to pages 27-28 for information on sample application.



Low power

The meter batteries do not have enough power to perform a test.

Replace the new batteries.

Message

What it means?

What to do?



System error

There may be a problem with the meter.

Replace the batteries first.


Refer to pages 15 and 40. If this error message appears again, please contact your local dealer.



Memory Error

Replace the batteries first.

If **ERROR 005** appears again, please contact your local dealer.

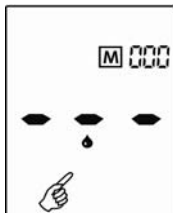

No result in memory.


The test proceeds incompletely. The meter was unable to recall this result.

You can still perform a blood glucose test and get an accurate test result.

Message

What it means?

What to do?


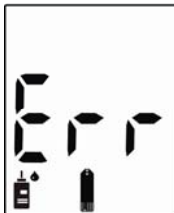


Volume detector error

The volume of blood or control solution is NOT enough.

Replace with a new strip.



If **Err** appears again, please contact your local dealer.

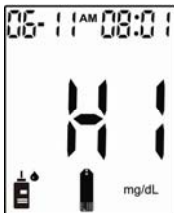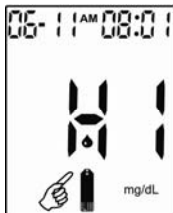

Test result is higher than 630 mg/dL (35.0 mmol/L).

Re-check your glucose level.

If the result is HI again, obtain and follow instructions from your healthcare professional without delay.

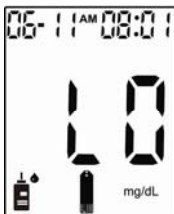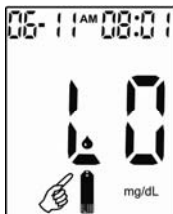

The test result is lower than 20 mg/dL (1.1 mmol/L).

This may require immediate treatment according to your healthcare professional's recommendations. Although this message could be due to a test error, it is safer to treat first and then do another test.

Message

What it means?

What to do?



The “HF” and thermometer icon appears. Temperature is too high, outside the required range of 10°C - 40°C (50°F - 104°F). This alerts users that an incorrect result may occur if the test continues.

Relocate the meter to a location with temperature between 10°C - 40°C (50°F - 104°F).



The “LF” and thermometer icon appears. Temperature is too low, outside the required range of 10°C - 40°C (50°F - 104°F). This alerts users that an incorrect result may occur if the test continues.

Relocate the meter to a location with temperature between 10°C - 40°C (50°F - 104°F).

Chapter 6: Technical Information

Specifications

Brand name	EASYMAX[®] MU Blood Glucose Meter	
Range	20~630 mg/dL (1.1~35.0 mmol/L)	
Response time	5 seconds	
Memory sets	480 test results	
Operating condition	Temp.	10°C - 40°C (50°F - 104°F)
	Relative Humidity	R.H. ≤ 90%
Storage and transportation condition	Temp.	-20°C - 50°C (-4F - 122°F)
	Relative Humidity	R.H. ≤ 90%
Blood sample	0.6 µL	
	Fresh blood from fingertips, palm, or forearm	
Hematocrit (Hct)	20~60%	
Power	2 Alkaline 1.5V (AAA)	
Battery life	Over 2000 tests	
Display dimension	1.4 x 1.7 inches (35.0 x 43.0 mm)	
Device dimension H × W × D	3.7 x 2.0 x 0.8 inches (94 x 50 x 19.5 mm)	
Weight	1.68 oz. without batteries (47.7±1 gram)	
Principles	Electrochemical biosensor technology	
Software via USB	GlucoManager [™]	

Limitations

The test strips are used for fresh capillary whole blood samples.

1. DO NOT use serum or plasma sample.
2. DO NOT use anticoagulant NaF or potassium oxalate for venous sample preparation.
3. DO NOT use neonate blood sample.
4. Extreme humidity may affect the results. A relative humidity greater than 90 % may cause incorrect results.
5. The system should be used at temperatures between 10°C and 40°C (50°F and 104°F). Outside this range, the system may get incorrect results.
6. DO NOT reuse the test strips. The test strips are for single use only.
7. Hematocrit: The hematocrit between 20% and 60% will not affect the results. Hematocrit below 20% may cause higher results. Hematocrit above 60% may cause lower results.
8. Altitude up to 3,048 meters above sea level has no effect on readings.

Healthcare Professionals – Please note these additional Limitations

1. If the patient has the following conditions, the result may fail:
 - ◆ Severe dehydration
 - ◆ Severe hypotension (low blood pressure)
 - ◆ Shock
 - ◆ A state of hyperglycemic-hyperosmolar state (with or without ketosis)
2. Lipemic samples: Cholesterol level up to 500 mg/dL (12.92 mmol/L) and triglycerides up to 3,000 mg/dL (33.6mmol/L) do not affect the results. Grossly lipemic patient samples have not been tested and are not recommended for testing with **EASYMAX[®] MU** Glucose Meter.

- Critically ill patients should not be tested with **EASYMAX[®] MU** glucose meters.
- DO NOT use during xylose absorption testing. Xylose in the blood will interfere Self-Monitoring Blood Glucose System.
- Interfering Substances depend on the concentration. The below substances up to the test concentration will not affect the test results.

Bias Concentrations of the interference tested	Glucose Level	80 mg/dL (4.4 mmol/L)	250 mg/dL (13.9 mmol/L)	500 mg/dL (27.8 mmol/L)
Ascorbic Acid, 4 mg/dL (0.26mmol/L)		10.89%	-1.76%	4.55%
Ibuprofen, 50 mg/dL (2.43mmol/L)		3.10%	2.88%	4.62%
L-Dopa, 1.8 mg/dL (0.09mmol/L)		10.59%	7.91%	4.90%
Sodium Salicylate, 50 mg/dL (3.12mmol/L)		-2.59%	9.42%	-0.84%
Tetracycline, 1.5 mg/dL (0.03mmol/L)		-5.32%	3.81%	3.20%
Tolbutamide, 100 mg/dL (3.7mmol/L)		-2.60%	12.30%	0.89%
Bilirubin-unconjugated, 2.4 mg/dL (0.04mmol/L)		-2.52%	4.05%	-0.23%
Uric acid, 8 mg/dL (0.48mmol/L)		2.71%	9.55%	-1.75%
Xylose, 4 mg/dL (0.27mmol/L)		-5.12%	-1.64%	-4.44%

Device Information

EASYMAX[®] MU SMBG System,
EASYMAX[®] MU Blood Glucose Test Strips,
EASYMAX[®] MU Blood Glucose Meter,
EASYMAX[®] MU Normal Control Solution,
EASYMAX[®] MU High Control Solution,
EASYMAX[®] MU Low Control Solution



Obelis s.a.

Boulevard Général Wahis 53, B-1030 Brussels, Belgium
TEL : (32) 2.732.59 54 FAX : (32) 2.732.60 03
E-mail: mail@obelis.net



Manufacturer:



EPS BIO TECHNOLOGY CORP.

No.8, R&D RD. III, Hsinchu Science Park, Hsinchu, Taiwan 30077
TEL : +886-3-6686868 FAX : +886-3-6686866
E-mail : info@epsbio.com.tw Website : <http://www.epsbio.com>

Warranty

EPS warrants the original purchaser for a period of 3 years from the date of purchase. This means during the warranty period if the Self-Monitoring Blood Glucose System does not work for any reason (other than obvious abuse), EPS will replace it with a new system or an equivalent product free of charge.

Lancing Device & Lancet

Meets the requirements of MDD 93/42/EEC



Manufacturer:

STERILANCE MEDICAL (SUZHOU) INC.

68# LiTangHe RD, XiangCheng, Suzhou, China 215131

Tel: +86 512 65799308 Fax: +86 512 67217663