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DEAR GAMMA SEMI-AUTOMATIC BLOOD PRESSURE MONITOR SEMI PLUS OWNER

Thank you for choosing our Semi-automatic Blood Pressure Monitor **Gamma**® designed on the basis of the most advanced technologies. The device you are holding now is your reliable assistant. You can measure your blood pressure with this monitor as accurately and precisely as an experienced doctor or a skilled nurse would do it.

We are sure that having appraised worthily the high quality and reliability of this device you will become a regular user of the products of Trademark **Gamma**®.

Before starting to use this device please study the instruction book carefully. The instruction book offers all information you need to correctly measure your blood pressure and pulse.

Regular blood pressure control allows for reliable and easy health monitoring and for effective combating possible complications caused by blood pressure abnormalities.

ATTENTION

This blood pressure monitor is designed to carry out self-control over blood pressure but NOT to make self-diagnosis of hypertension/hypotension. Please DO NOT diagnose by yourselves basing on the measurement results obtained with the blood pressure monitor. Please DO NOT execute self-treatment of high/low blood pressure and DO NOT change the methods prescribed without consulting your doctor.

Type BF applied part.


Read the instructions carefully before using this device.

TABLE OF CONTENTS

IMPORTANT INFORMATION ON BLOOD PRESSURE AND ITS MEASUREMENT.....	6
What Is Blood Pressure?.....	6
Which Values Are Normal?.....	7
Blood Pressure Level Indicator.....	9
Irregular Heartbeat Detection.....	9
GETTING READY FOR MEASUREMENT.....	10
Safety Precautions.....	10
Description of the Blood Pressure Monitor.....	11
Insert or Replace Batteries.....	12
Cuff Connection.....	13
Setting the Date and Time.....	13
MEASUREMENT PROCEDURE.....	14
Fitting the Cuff.....	14
Taking a Single Measurement.....	15
Reading Measurement Results.....	16
Discontinuing a Measurement.....	16
MEMORY FUNCTION.....	17
Viewing the Stored Values.....	17
Delete Memories.....	17
ERROR MESSAGES/TROUBLESHOOTING.....	18
CARE AND MAINTENANCE.....	19
SYMBOL INFORMATION.....	20
TECHNICAL SPECIFICATIONS.....	20
WARRANTY.....	21

IMPORTANT INFORMATION ON BLOOD PRESSURE AND ITS MEASUREMENT

What Is Blood Pressure?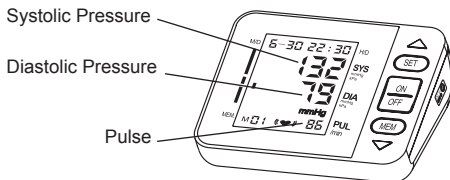

Blood pressure is the pressure that your blood extends to the vascular walls. Blood pressure is necessary to provide for constant blood flow inside the body. Thanks to it the cells get oxygen that provides for their normal functioning. The heart performs the function of a «pump», sending blood to the blood vessels. Each heart beat creates a certain level of the blood pressure.


There are 2 kinds of blood pressure: a systolic (upper) one, which corresponds to the heartbeat pushing blood into the arteriae; and a diastolic (lower) one, which means the blood pressure between two heartbeats.

Blood pressure is subject to fluctuations during the day even in healthy people. The fluctuations are influenced by a number of factors - time of day, person's condition, physical or mental activity, environment, etc.

Day-Night Fluctuations of the Blood Pressure

An increase of blood pressure increases the burden onto the heart, affects blood vessels making their walls thick and less elastic.

One of the features of the hypertension is its ability to remain unnoticed for the patient at its early stages. That's why the self control of the blood pressure is so important. With the illness progressing, headaches and regular dizziness appear, the sight declines, the functioning of vitalses (encephalon, heart, kidneys, blood vessels) breaks down. Without special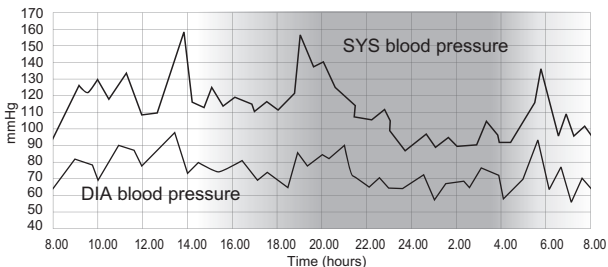

treatment the complications of hypertension might be kidney damages, breast-pang, paralytic stroke, aphasia, dementia, heart attack and stroke.

Which Values are Normal?

Monitoring your routine blood pressure trend helps you to know your body condition. Human blood pressure naturally increases after reaching middle age.

This symptom is a result of continuous ageing of the blood vessels. Further causes include obesity, lack of exercise and cholesterol (LDL) adhering to the blood vessels, diabetes. Rising blood pressure accelerates hardening of the arteries, and the body becomes more susceptible to apoplexy and coronary infarction.

Definitions and Classification of blood pressure levels according to 2007 ESH-ESC Practice Guidelines for the Management of Arterial Hypertension: Data in mmHg.

Category	Systolic (mmHg)		Diastolic (mmHg)
Optimal	< 120	and	< 80
Normal*3	120-129	and/or	80-84
High normal	130-139	and/or	85-89
Grade 1 hypertension	140-159	and/or	90-99

Grade 2 hypertension	160-179	and/or	100-109
Grade 3 hypertension	≥ 180	and/or	≥ 110
Isolated systolic hypertension	≥ 140	and	< 90

Isolated systolic hypertension should be graded (1, 2, 3) according to systolic blood pressure values in the ranges indicated, provided that diastolic values are < 90 mmHg.

Source: The European Society of Hypertension and European Society of Cardiology Task Force Members. 2007 ESH-ESC Practice Guidelines for the Management of Arterial Hypertension. J Hypertens 2007; 25:1751-1762.

- The diagnosis of hypertension requires from the patient to combine medical treatment prescribed by the doctor and mode of life correction.
- People with normal pressure and high normal pressure are recommended to carry out self-control of their tension in order to timely take measures to decrease the blood pressure level down to the optimal one without using any medications.
- For people more than 50 years old high level of systolic blood pressure (higher than 140 mmHg) is more crucial than diastolic pressure.
- Even with blood pressure being normal, people run the bigger risk of hypertension development with advancing age.

ATTENTION

If you have normal results of blood pressure measured under calm conditions but your results are excessively high when measured under the conditions of physical or mental exhaustion, this might be a sign of so called brittle (that is unstable) hypertension. If you suspect that, please consult your doctor.

When measured correctly, if diastolic blood pressure is more than 120 mmHg, it is necessary to call the doctor immediately.

Blood Pressure Level Indicator

Blood pressure level indicator is located along the left side in the display. The classification corresponds to ESH (European Society of Hypertension) ranges described in the table of the section «Which values are normal?». After the measurement there appears the bars in the left part of the display: green zone - optimal blood pressure, yellow - elevated, orange - too high, red - dangerously high. This function helps you to self-orient in the measurement results.

Irregular Heartbeat Detection

This function indicates allorhythmic heartbeating. If the symbol of IHD (♥) appears on the display that means that certain abnormality in heart beat frequency was detected during the measurement.

In this case, the result may deviate from your normal blood pressure – repeat the measurement. In most cases, this is no cause for concern. However, if the symbol (♥) appears on a regular basis (e.g. several times a week with measurements taken daily) we advise you to tell your doctor. Please show your doctor the following explanation:

Information for the doctor on frequent appearance of the Irregular Heartbeat Detector.

This instrument is an oscillometric blood pressure monitor that also analyses pulse frequency during measurement. The instrument is clinically tested. The symbol (♥) is displayed after the measurement, if pulse irregularities occur during measurement. If the symbol (♥) appears more frequently (e.g. several times per week on measurements performed daily) we recommend the patient to seek medical advice. The instrument does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

GETTING READY FOR MEASUREMENT

Safety Precautions


1. Use this device **ONLY** for the intended use described in this manual.
2. Do **NOT** use accessories which are not specified by the manufacturer.
3. Do **NOT** use the device if it is not working properly or damaged.
4. Do **NOT** use under any circumstances on newborns.
5. This device does **NOT** serve as a cure for any symptoms or diseases. The data measured are for reference only. Always consult your doctor to have the results interpreted.
6. Keep the equipment and its flexible cord away from hot surfaces.
7. Do **NOT** apply the cuff to areas other than the place directed.
8. Proper maintenance and periodically calibration are essential to the longevity of your device. If you are concerned about your accuracy of measurement, please contact local customer service for help.
9. This system is intended for individuals age 18 or above.
10. Do not use it for diagnosis of hypertension or testing on newborns, babies, young children or persons who cannot express their consent.
11. If you have been diagnosed with a severe arrhythmia or irregular heartbeat, atrial or ventricular premature beats or atrial fibrillation, measurements made with this instrument should only be evaluated after consultation with the doctor or your healthcare professionals.

KEEP THIS MANUAL DURING THE WHOLE LIFETIME OF THE DEVICE

Components of Your Blood Pressure Monitor Kit

The blood pressure monitor kit includes Semi-Automatic Blood Pressure Monitor Model Semi Plus, size cuff 22-42 cm, 4 batteries (AA), User's Manual, Warranty Card, Bag, Kit Box.

Description of the Blood Pressure Monitor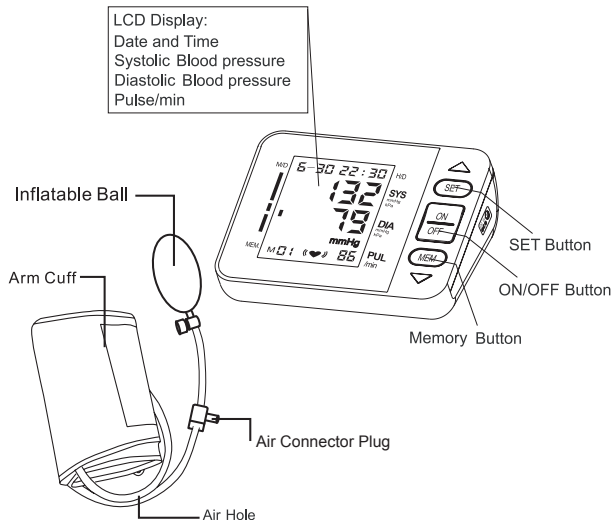

Cuff:

Semi Plus offers a wide range rigid cuff Comfort measuring size: 22~42 cm, suitable for most of arm circumferences.

Accessory:


Manual


Bag

SYMBOLS ON DISPLAY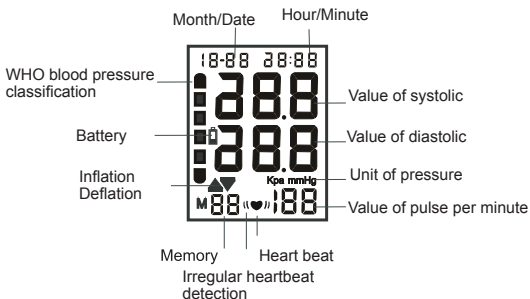

Insert or Replace Batteries

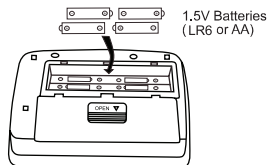1. Remove the battery cover.
2. Insert new batteries into the battery compartment as shown, taking care that the polarities (+) and (-) are correct.
3. Close the battery cover. Use only LR6, AA batteries.

Disposal of old batteries to the authorized collecting party subject to the regulation of each individual territory.

CAUTION


- Insert the batteries as shown in the battery compartment.
- When  (LOW BATTERY mark) blinks in the display, replace all batteries with new ones. Do not mix old and new batteries. It may shorten the battery life, or cause the device to malfunction.
-  (LOW BATTERY mark) does not appear when the batteries run out.

- Battery life varies with the ambient temperature and may be shorter at low temperatures.
- Remove the batteries if the device is not to be used for a long time.
- The batteries may leak and cause a malfunction.
- Use the specified batteries only. The batteries provided with the device are for testing monitor performance and may have a shorter life.

NOTE: To prolong the batteries' life the device switches off automatically if no button is pressed for 1 minute. Otherwise you can switch it off by pressing the ON/OFF button.


Cuff Connection

Insert the cuff tube into the opening provided on the side of the device as shown in the picture.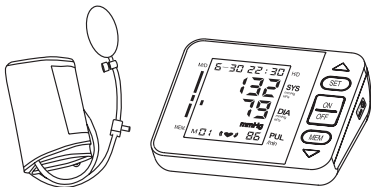

Setting the Date and Time

1. Press and hold "SET" key until the year number displays and flashes on LCD to enter setting mode.
2. Press "MEM" key to adjust the year, then press "SET" key again to save your setting and enter the month setting mode.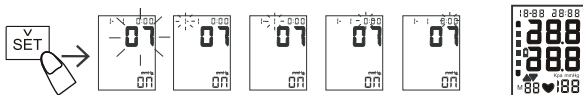

3. Press “MEM” key to adjust the month. Follow the same steps to adjust date/hour/minute until setting completed.

NOTE: If the meter is idle for 1.5 minutes during the setting mode, it will switch off automatically.


MEASUREMENT PROCEDURE

ATTENTION

- Find time to relax by sitting in a quiet atmosphere for some time before measurement.
- Efforts by the patient to support the arm can increase the blood pressure. Make sure you are in a comfortable, relaxed position and do not activate any muscles in the arm during measurement.
- Always measure on the same arm (normally left).
- Remove any garment that fits closely to your upper arm. Do not roll the sleeve since it can squeeze your hand and this can lead to false results.
- Use only clinically approved original cuff.
- If you want to follow the results of your blood pressure measurements, always perform measurements at the same time of day, since blood pressure changes during the course of the day.
- Measurements should be done after a 5 minute rest to ensure accuracy.

Fitting the Cuff

1. Plug the cuff connector to the cuff socket on the device left side.
2. Stretch your left (right) arm in front of you with your palm facing up. Slide and place the cuff onto your arm to let the air tube and artery mark region toward the lower arm.

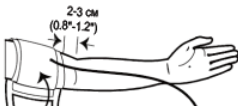Wrap and tighten the cuff above your elbow. The artery mark on the edge of the cuff should be approx. 0.8" to 1.2" (2 cm to 3 cm) above your elbow and above the artery. Align the tube over the main arteries

on the inside.

3. Leave a little free space between the arm and the cuff: you should be able to fit two fingers between them. Remove all clothing covering or constricting the measurement arm.
4. Press the hook material firmly against the pile material. The top and bottom edges of the cuff should be tightened evenly around your upper arm.

Taking a Single Measurement

ATTENTION

1. Sit down for at least 10 minutes before measuring.
2. Place your elbow on a flat surface. Relax your hand with the palm facing up.
3. Make sure the cuff is about the same height as the location of your heart.
4. Remain still and do not talk or move during the measurement.

WARNING:

If the cuff is relatively lower (higher) than the heart, the obtained blood pressure value could be higher (lower) than the actual value.

Always apply the pressure cuff before turning on the monitor.

1. Press the ON/OFF button. The device will turn on and the display for 2 seconds to highlight all the characters, then there will be «00».


2. Take a rubber blower to free hand and inflate the cuff to a pressure greater than 40 mm Hg the value of your expected systolic pressure. If you do not know your systolic pressure, inflate the cuff to 160-180 mm Hg. The pressure in the cuff is shown on the display.

3. When you reach the desired pressure, place the pear and sit quietly, not moving. The measurement process began.

If the measurement is not performed, and the display shows «▲», the cuff pressure was insufficient to perform the measurement. In this case it is necessary to pump up the pressure in the cuff to a higher value, at least 20 mm Hg. Art. higher than the previous.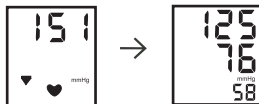

Reading Measurement Results

The measured systolic and diastolic blood pressure values, as well as the pulse are now displayed. The appearance of this symbol (♥) signifies that an irregular heartbeat was detected. This indicator is only a caution. It is important for you to be relaxed, remain still and do not talk during measurements.

NOTE: We recommend contacting your physician if you see this indicator frequently.

Discontinuing a Measurement


If for any reason you must stop measurement (for example, due to poor health), open the air release valve on the blower. The cuff pressure decreases. In this case, the display may appear symbol error «Err». Switch off the appliance. The device then immediately lowers the cuff pressure automatically.

MEMORY FUNCTION

Viewing the Stored Values

The device stores 90 most recent blood pressure test results along with respective dates and times in the monitor memory for each user.

When the monitor is turned off, press «MEM» button, displays average of the three most recent measurements. To see the next page (old measurements) press «SET» then press «MEM» button to display the next latest measurement.

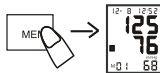Delete Memories

To delete memory, hold «MEM» button for five seconds, the LCD will display «no» - this means that all the memory has been deleted.

TROUBLESHOOTING

If you have trouble in using the unit please check the following points first.

ERROR DISPLAY	POSSIBLE CAUSE	HOW TO CORRECT
Nothing is displayed. When you push the POWER button or  battery icon flash	No battery installation	Insert batteries
	Battery worn out	Replace with new batteries
	The polarities of batteries placed wrongly	Insert battery in the correct polarities
E0	Testing pressure with Clothes	Take off the clothes and test it gain
	The marker of arm cuff  is not in alignment with the arm's artery.	Align the marker of arm cuff with the arm's artery, and measure again.
E1: Can't normally increase pressure	Check your cuff for air leakage	Replace cuff with new one
E3: Inflate pressure too high		Remeasure or send back to dealer to recalibrate
E2 E4: Hand shaking while measurement	Hand or body shaking while measurement	Keep static position and measure again
Battery icon on	Battery low power	Replace battery and measure again

The systolic pressure Value or diastolic Pressure value too high	1.The cuff was held lower than your heart	Keep correct position and measure again
	2.The cuff was not attached properly	
	3.You moved your body or spoke during measurement	
The systolic pressure value or diastolic pressure value too low	1.The cuff was held higher than your heart	
	2.You moved your body or spoke during measurement	

CARE AND MAINTENANCE

To avoid the monitor attracting dirt, dust or other contaminants, wash and dry your hands thoroughly before use.

Cleaning

- To clean the monitor exterior, wipe it with a cloth moistened with tap water or a mild cleaning agent, then dry the device with a soft dry cloth. Do NOT flush with water.
- Do NOT use organic solvents to clean the monitor.
- Do NOT wash the pressure cuff.
- Do NOT iron the pressure cuff.

Monitor Storage

- Storage condition: -20°C to 55°C (-4°F to 131°F), below 95% relative humidity.
- Always store or transport the monitor in its original storage case.
- Avoid dropping or heavy impact.
- Avoid direct sunlight and high humidity.

SYMBOL INFORMATION

SYMBOL	REFERENT
	Consult instructions for use
	Manufacturer
	Serial number
	Caution, consult accompanying documents
	Dispose of in accordance with the requirements of your country
	Type BF Equipment
	Class II
	CE mark

SPECIFICATIONS

Measuring Method	Oscillometric Measurement
Indication	Digital LCD display
Measuring Range:	Pressure: (30~280) mmHg Pulse: (40~199) Beat/min
Accuracy:	Static Pressure: ± 3 mmHg Pulse: $\pm 5\%$
Memory:	90 Memories
Power supply:	4x1.5V Batteries (LR6 or AA)

Operating condition:	+5°C~+40°C. 30% RH~80% RH Atmospheric pressure: 80 kPa~106 kPa
Storage condition:	-20°C~+55°C. 10% RH~93% RH Atmospheric pressure: 50 kPa~106 kPa
Dimensions:	Approx: 140(W) X 100(H) X 50(D) mm
Weight:	Approx: 340 g, excluding batteries
Classification	Type BF
Cuff Size	22-42 cm

* Specifications may be changed without notice in the event of improvement being made.

This device has been tested to meet the electrical and safety requirements of:

IEC/EN 60601-1,
IEC/EN 60601-1-2

Reference to Standards:

- EN 1060-1/-3, NIBP-requirements
- IEC60601-1 General requirement for safety
- IEC60601-1-2 Requirements for EMC
- EN1060-4, NIBP clinical investigation
- AAMI/ANSI/IEC 80601-2-30, ANSI/AAMI/ISO 81060-2, NIBP requirements

WARRANTY

Your Semi-automatic Blood Pressure Monitor Model Semi Plus is warranted for 5 years from date of purchase. Warranty for the cuff is 1 year from the date of purchase. The warranty does not apply to damage caused by improper handling, accidents, not following the operating instructions or self-maintained alterations made to the device. The warranty is only valid upon presentation of the warranty card which was correctly filled in and sealed.

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LOTUS GLOBAL Co., Ltd.