NISSEI Digital Blood Pressure Monitor DS-1873

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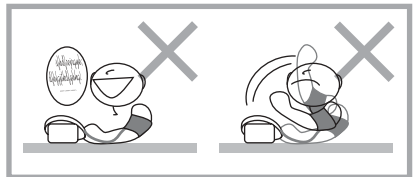
DIA
mmHg

PUL
/min

AVERAGE \bar{x} 30 number of total stored data

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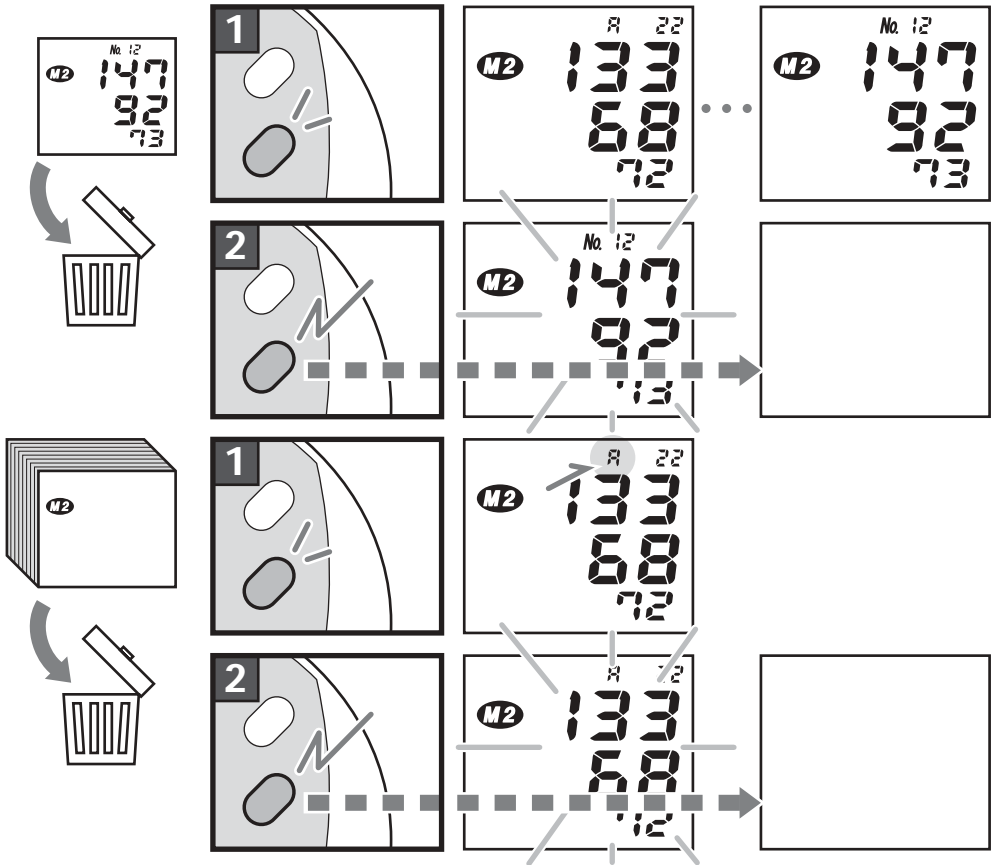
LÖSCHEN DER GESPEICHERTEN DATEN p.21
CANCELLAZIONE DEI DATI MEMORIZZATI p.37

BASIC PRODUCT INFORMATION

This manual is intended to assist the user in the safe and efficient operation of the Model DS-1873 Digital Blood Pressure Monitor. The product must be used in accordance with the procedures contained in this manual and must not be used for purposes other than those described herein. It is important to read and understand the entire manual. In particular, please read carefully and become familiar with the section entitled "TIPS ON TAKING YOUR BLOOD PRESSURE".

The user is cautioned that the system may fail to yield specified measurement accuracy if operated or stored in temperature or humidity conditions outside the limits stated in the specifications section of this manual. Also, an inaccurate reading may result if this instrument is used on a child's arm. Consult your physician if you wish to take a child's blood pressure.

Indications for Use

The DS-1873 system is intended for the non invasive measurement of systolic and diastolic blood pressure and determination of heart rate in adult patients, i.e., age 15 and above. The product is recommended for use by patients with labile blood pressure or known hypertension in a home care environment as an adjunct to medical management. The cuff will accommodate an upper arm circumference range of approximately 22 to 32 centimeters. Pressure is measured over a range of 0 to 300 mmHg and heart rate over a range of 40 to 160 beats/minute.

Method of Measurement

This product employs the oscillometric method for measurement of blood pressure and heart rate. The cuff is connected to the main unit and wrapped around the arm. As ON/OFF BUTTON is pressed, the system starts automatic pressurization. Circuits within the cuff sense the small oscillations in pressure against the cuff produced by the expansion and contraction of the arteries in the arm in response to each heart beat. The amplitude of each pressure waves is measured, converted to millimeters of mercury, and displayed on the LCD as a digital value. A memory circuit stores the 30 most recent measurement results for comparison.

SPECIFICATIONS

Model	: DS-1873	Memory	: 30 measurement results x 2, computation of the average of stored data
Operating Principle	: Oscillometric method	Operating environment	: +10°C to +40°C, 85% Relative humidity or below
Indicator	: 13 digits LCD display	Storage environment	: -5°C to +50°C, 85% Relative humidity or below
Pressure indicating Range	: 0 to 300 mmHg (cuff pressure)	Cuff	: Coverage arm circumference : 22 to 32 cm
Measuring Range	: 50 to 250 mmHg (systolic) 40 to 180 mmHg (diastolic) 40 to 160 pulse/min. (pulse rate)	Main Unit	: Weight : Approx. 370 g (without batteries) Size : 163 x 180 x 83 mm (W x D x H) when CUFF HOLDER attached
Accuracy	: ± 3 mmHg (cuff pressure) $\pm 5\%$ of reading (pulse rate)	Key to symbols	: Type BF equipment 
Inflation	: Automatic inflation	Important	: Read operating instructions 
Deflation	: Electric control valve		
Exhaust	: Automatic exhaust valve		
Power Supply	: 4 pcs. 1.5 volt "AA" (R6P/LR6) type batteries or AC adapter		
Power Consumption	: 4W (max.)		Specifications are subject to change without notice due to improvements in performance.

This device complies with EMC (IEC60601-1-2:1993), EN1060-1 and EN1060-3.

PRECAUTIONS FOR USE

- Do not use cuffs other than the original cuff included with this product.
- Do not use this instrument without consultation with your doctor if you are under dialysis therapy or on anticoagulants, antiplatelets or steroids. Use of this instrument under such conditions could cause internal bleeding.
- To avoid any possibility of accidental strangulation, keep this unit away from children and do not drape AIR HOSE ⑩ around your neck.
- For specific information on your blood pressure, contact your physician. Never make any judgment on measurement results on your own.
- Use of this device in areas near portable phones, microwave ovens or other devices with strong electromagnetic field may cause malfunctions.

TIPS ON TAKING YOUR BLOOD PRESSURE

Blood pressure is a measurement of the force exerted by the heart in pumping the blood through the arteries and the resistance by the veins in this flow.

Blood pressure changes easily as influenced by mental and physical factors and is never constant.

In general, blood pressure is highest during the working hours and gradually decreases during the afternoon and evening hours. It is low during sleep and slowly increases at a relatively fast rate after arising from bed.

Causes for Changes in Blood Pressure

- Body movement
- Conversation
- Mental Tension
- Emotions
- Meals
- Drinking Alcohol
- Smoking
- Recent Urination or Bowel Movement
- Temperature
- Changes in the surroundings such as movement or noise, etc.

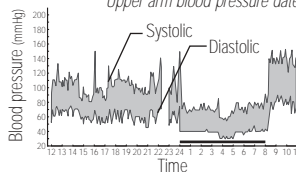
Blood pressure measured at home tends to be lower than when measured in a hospital, clinic or doctor's office.

When blood pressure is measured in a hospital, it could be higher than when measured at home. This is because you are tense at the hospital and relaxed at home. It is important to know your stable normal blood pressure at home.

Let a qualified physician interpret your blood pressure readings.

Depending on your age, weight and general condition, blood pressure can be slightly different. Consult with your doctor on determining what blood pressure is normal for you.

Blood pressure changes shown below.
Upper arm blood pressure date.

180~	(severe)	Hypertension				
160-179	(moderate)					
140-159	(mild)					
130-139	High Normal					
120-129	Normal					
-120	Optimal					
SYS mmHg		80	85	90	100	110
DIA		80	84	89	99	109

Blood pressure classification by WHO
(1999)

- Rest for approximately five minutes and take your blood pressure while relaxed in a quiet environment.
- Take blood pressure measurement with the correct posture and do not move nor speak during measurement.
- Avoid exercise, eating, drinking alcohol, smoking and other activities that affect your blood pressure right before a measurement.
- Take your blood pressure every day at the same time.
- The ambient temperature should be approximately 20°C when you take your blood pressure.

INSTALLING BATTERIES AND SETTING THE CLOCK see fig. 1

1 Turn the unit over and remove CUFF HOLDER ⑦ from the unit.

2 Open the battery cover.

Never forcefully remove CUFF HOLDER or the battery cover.

3 Install four “AA” type batteries into the compartment as indicated. Make sure that the polarities correspond to the (+) and (-) marks.

4 Close the battery compartment cover.

Year flashes on the display.

Clock can be set only after the batteries are reinstalled. Adjust the clock so that measurement results are stored with correct date and time.

5 Use M1 ③ to increase the flashing number and M2 ④ to decrease the number.

6 Press ON/OFF BUTTON ② to confirm and move to next step.

Day flashes.

7 Adjust day with M1 and M2 buttons.

8 Press ON/OFF BUTTON and month flashes.

9 Adjust month with M1 and M2 buttons.

CAUTION!

Do not use rechargeable batteries.

 **battery replacement indication**

Replace all the batteries when the battery replacement indication appears on the display or nothing is displayed even if ON/OFF BUTTON ② is depressed.

Use alkaline batteries to increase battery life. Use of regular Carbon-Zinc batteries will necessitate more frequent replacement. *The enclosed batteries are for monitoring, and their life may be shorter than that of commercial batteries.*

Since neither the device nor batteries are household waste, follow your local recycling rules and dispose of them at appropriate collection sites.

Operation with an AC adapter
AC ADAPTER JACK ⑥ is located on the right side of the unit. **Do not use any adapters other than the one specified below.**

Specifications of the AC adapter to be selected are:
 Output voltage : 6 VDC $\pm 5\%$
 Input voltage range : Standard voltage $\pm 15\%$
 Maximum electric load : 600mA or higher

INSTALLING BATTERIES AND SETTING THE CLOCK

10 Press ON/OFF BUTTON and hour flashes.

The clock operates in 24-hour mode.

11 Adjust with M1 and M2 buttons.

12 Press ON/OFF BUTTON and minute flashes.

13 Adjust with M1 and M2 buttons.

14 Press ON/OFF BUTTON.

The clock is set and the unit is turned off.

The clock is displayed while the unit is turned off.

Output plug polarity	: Center Minus	
Output plug shape	: External diameter	5.50±0.1 mm
	: Internal diameter	2.10±0.1 mm
	: Length	9.5±0.3 mm

Remove the batteries when operating the unit with AC adapter for extended period of time.

When batteries are removed or AC adapter is unplugged, the clock returns to the default value. *Stored measurement results remain.

CORRECT MEASURING POSTURE

see fig.2  

Sit at the table and let the table support your arm as you take the blood pressure measurement.

Make sure that the measurement location on the upper arm is at approximately the same height as the heart, and that the forearm is extended naturally on the table and does not move.

You may lie on your back and take the measurement. Look at the ceiling, stay calm, and do not move your neck or body during the measurement. Again, make sure that the measurement location on the arm is at approximately the same height as the heart.

WRAPPING THE CUFF

see fig.3  

1 With the tube of the cuff positioned in the direction of the fingertips, place it on your left arm.

2 Wrap the cuff around the arm with the edge of the cuff approximately 2~3 cm above the elbow.

3 Press the surface of the cuff to make sure that it is attached securely.

When wrapping the cuff, wrap it loosely enough around the arm so that two fingers can be placed between the cuff and the arm. If the cuff is wrapped more tightly or loosely than this, inaccurate blood pressure readings may result.

If you are wearing a shirt that might restrict circulation in your upper arm or you roll your sleeve up over the upper arm, the blood flow will be restricted, preventing accurate measurement.

**Take deep breaths and relax.
Do not move, chat or strain your arm or hand
during measurement.**

- 1 Insert TUBE PLUG ⑪ into AIR CONNECTOR ⑤.**
- 2 Press ON/OFF BUTTON ②.**

Inflation mark "▲" flashes and the monitor starts automatic inflation. Inflation stops and pressure starts to descend. The pulse is indicated by heart mark "♥".

**Maximum and minimum blood pressures
and pulse rate are displayed.**

Deflation mark "▼" appears and the unit automatically exhausts the air from the cuff.

- 3 Press either MEMORY BUTTON M1 ③ or M2 ④.**

The displayed memory bank number changes as the button is pressed.

The result is stored in the selected bank.

The result is stored in the bank which is selected when the unit is turned off. *When a measurement resulted in an error, it will not be stored.*

- 4 Press ON/OFF BUTTON to turn off the power.**

If you forget to turn off the unit, it will be automatically turned off after 3 minutes.

Stopping Measurement

Press ON/OFF BUTTON ② and the monitor will stop inflation, discharge air rapidly, and then turn off.

Automatic Repressurization

If the pressure applied is judged insufficient in the early phase of measurement or if movement of arm or hand occurs during measurement, the unit will inflate again to a level about 40 mmHg higher. Automatic Repressurization is repeated until a measurement is made. However, *this does not indicate a problem.*

To manually control pressurization, hold down ON/OFF BUTTON immediately after the start of inflation and release it when the pressure reaches the level 50 mmHg higher than the expected maximum systolic pressure. If the pressure value has exceeded 180 mmHg, the inflation will stop when the key is released. Pressure can be increased to approximately 300 mmHg.

**Do not excute repeated
measurements for congestion of
blood could result in false
measurement. Let your arm rest for
at least 5 minutes.**

Measured result is automatically stored in either M1 or M2. Make sure which memory bank number is displayed before turning off the unit. Each of two memory banks can store up to 30 results and their average. When the number of measurements exceeds 30, the oldest data will be deleted to record new data. You may decide which bank to store your measurement results to avoid data mixture with someone else's, or may use two banks to save data measured in the morning and evening separately.

Recalling measured data

- 1 Press MEMORY BUTTON M1 ③ or M2 ④ to see stored data.**

MEMORY FUNCTIONS

The average of the stored result in the selected bank is displayed.

The latest result stored is displayed when there is only a single result and the clock display remains when there are no results stored.

The indication at the top of the display alternately changes from memory data number to date and to time.

The memory data is displayed for approximately 30 seconds. Approximately 30 seconds after MEMORY BUTTON is released, the apparatus will turn off.

2 Press MEMORY BUTTON to move to next data.

The memory data number 1 is the latest among the stored data in the selected memory bank.

3 Every depression of the key switches among the memory data.

As MEMORY BUTTON is pressed, the memory data number increases; the bigger the number, the older the result.

4 Press ON/OFF BUTTON ② and turn off the unit.

DELETING STORED DATA

see fig. 6

**Deleting a single data****1 Show the data to be deleted following the steps in MEMORY FUNCTIONS.****2 Press and hold down MEMORY BUTTON M1 ③ or M2 ④ whichever the data is stored.**

The displayed data starts flashing.

3 Hold down the button until the display disappears.**Deleting all data in a bank****1 Press MEMORY BUTTON M1 ③ or M2 ④ and show the average of the bank to be deleted.****2 Press and hold down selected MEMORY BUTTON.**

The displayed data starts flashing.

3 Hold down the button until the display disappears.

CARE AND MAINTENANCE

Because the unit includes precision parts, care should be taken to avoid extreme temperature variations, humidity, shock, dust, and direct sunlight. Do not drop or strike the unit. Make sure not to expose the unit to moisture. **This unit is not water resistant.**

Use only a soft, dry cloth to clean the unit. Do not use gasoline, paint thinner, or other strong solvents on the unit. Since the cuff may absorb perspiration and other fluids, inspect it for stain and discoloration after each use. When cleaning the cuff, do not machine wash or scrub it. Use a synthetic detergent and gently rub the surface. Make sure fluid never gets in AIR HOSE ⑩. Air dry thoroughly.

When storing the unit, do not place heavy objects on it and do not coil AIR HOSE too tightly. When the unit has been stored at a temperature below the freezing point, keep it for at least 1 hour in a warm place before using it. Remove the batteries if the instrument is to be stored for an extended period of time. **Keep**

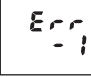
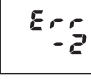
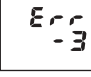
the batteries out of reach of children.

We suggest that you have your blood pressure monitor checked every 2 years. This operation may only be performed by the manufacturer or by firms authorized by the manufacturer.

Do not inflate the cuff when it is not wrapped around your arm.

Do not disassemble or modify the unit.

ERROR DISPLAYS AND TROUBLESHOOTING

ERROR SYMOL / SYMPTOM	CAUSE / CHECK POINT	REMEDY
OVER-PRESSURIZATION 	The cuff was inflated to the maximum pressure because of movement of body etc.	Do not move during measurement.
MEASUREMENT ERROR 	Measurement could not be made because of moving or talking during measurement.	Remain still and quiet during measurement.
INFLATION ERROR 	TUBE PLUG ① is not correctly inserted.	Reinsert TUBE PLUG ① and make sure that it is securely inserted.
DEFLATION ERROR 	Movement or talking was detected during measurement.	Turn off the power and make another measurement while being quiet and still.
IMPROPER OPERATION 	ON/OFF BUTTON ② was accidentally pressed during battery replacement.	Press ON/OFF BUTTON ② to turn off the power once and press it again to start a measurement.
No display when ON/OFF BUTTON ② is pressed.	Have the batteries polarities been positioned incorrectly? Have the batteries run down? Are the battery terminals clean?	Reinsert the batteries in the correct position. Replace them with new ones. Clean the battery terminals with a dry cloth.

ERROR DISPLAYS AND TROUBLESHOOTING

ERROR SYMOL / SYMPTOM	CAUSE / CHECK POINT	REMEDY
Display does not change when MEMORY BUTTON (③ or ④)s pressed.	Is there any measured result stored in the memory? Are you pressing the right MEMORY BUTTON? Have the batteries run down?	Nothing is displayed when there is no measured data stored. Check the memory bank number. Replace them with new ones.
Cannot make great number of measurements.	Are you using alkaline batteries?	Use alkaline batteries to increase battery life.
Cannot complete the measurement.	Have the batteries run down?	Replace all four batteries with new ones.
Blood pressure is different each time. The reading is extremely low (or high).	Are you measuring with correct posture? Blood pressure readings constantly vary with time of measurement and nervous condition.	Measure with the correct posture. See TIPS ON TAKING YOUR BLOOD PRESSURE.
Pulse rate is too low (or too high).	Did you move during the measurement? Did you make measurement right after exercise?	Remain still during the measurement. Measure again after resting for more than 5 minutes.
The power is automatically turned off.	Have you left the instrument untouched after the measurement?	This does not indicate a problem. The instrument automatically turns off 3 minutes after a measurement or 30 seconds after memory display.
Pressurization stops once and starts again.	Repressurization starts in order to make accurate measuring. Did you move your body during the measurement?	See MEASUREMENT PROCEDURES. Remain still and continue the measurement.

If you cannot get correct measurement with the methods above, do not tamper with the internal mechanism. Contact your dealer. If the unit malfunctions, please return it to the dealer or an authorized service representatives for service according to the warranty. Service data are provided to qualified dealers by the manufacturer.

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WARRANTY

This equipment is guaranteed for the period of 2 years after the date of purchase against manufacturing defects when returned along with the proof of date of purchase to the dealer from whom the purchase was made. During this period, the unit will be repaired or replaced free of charge if the failure is attributable to faulty design or manufacture. This warranty does not cover damage or malfunctions caused by improper handling or use contrary to the instructions in this manual. Please contact your dealer for additional warranty provisions which may remain effective after the manufacturer's warranty period has expired.

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NIHON SEIMITSU SOKKI CO., LTD. 2508-13 Komochi Kitagunma Gunma Japan
JPI Trading GmbH, Holzbrücke 7, 20459 Hamburg, Germany  040-82 22 80 510