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PART NAMES AND PRODUCT COMPONENTS (→fig.1)

(d) AIR CONNECTOR (e) BATTERY COMPARTMENT (f) METAL RING (g) AIR HOSE CUFF (h) AIR PLUG POUCH

AA (LR6) BATTERIES INSTRUCTIONS

DISPLAYS (→fig.2) 1 PULSE RATE MARK

MEMORY BANK SYMBOL RATTERY REPLACEMENT INDICATION BODY MOTION SYMBOL (5) IRREGULAR PULSE RHYTHM SYMBOL (3) INFLATION SYMBOL **DEFLATION SYMBOI** SYSTOLIC **O** PULSE RATE DIASTOLIC

GENERAL INFORMATION

This manual is intended to assist you in the safe and efficient operation of BLOOD PRESSURE MONITOR DS-A10. The product must be used in accordance with the procedures contained in this manual and must not be used for purposes other than those described herein. It is important to read and understand the entire manual. In particular, please read carefully and become familiar with the section entitled "TIPS ON TAKING YOUR BLOOD PRESSURE".

INDICATIONS FOR USE

This product is intended for noninvasive measurement of systolic and diastolic blood pressure, determination of pulse rate and calculation of pulse pressure in adults in a home healthcare environment. The product is not designed for neonatal use. Please consult with your doctor or physician to use this product to take blood pressure of child or person in pregnancy or under pre-eclamptic

METHOD OF MEASUREMENT

This product employs the oscillometric method for measurement of blood pressure and pulse rate. The cuff is connected to the main unit and wrapped around the arm. Circuits within the cuff sense the small oscillations in pressure against the cuff produced by the expansion and contraction of the arteries in the arm in response to each heart beat. The amplitude of each pressure waves is measured, converted to nillimeters of mercury, and displayed on the LCD as a digital value.

PRODUCT SPECIFICATIONS : DS-A10 : Coverage arm circumference; 22 to 32 : 9 digits liquid crystal display : Weight; Approx. 250 g (without batteries) Size; 116.5 x 122.2 x 69.9 mm (W x D x H) : 3 to 300 mmHg (cuff pressure Protection against electric shock Measuring Range: 50 to 250 mmHg (systolic), 40 to 180 mmHg : Internally powered equipment/Class II (diastolic), 40 to 180 bpm (pulse rate) equipment, Type BF applied part : ±3 mmHg (cuff pressure), ±5% of : Internally powered equipment/Class II reading (pulse rate) : Automatic inflation Protection class IP : Four 1.5 volt LR6 (AA alkaline) batteries particles with a diameter of more than or AC adaptor ADP-W5 series 12.5 mm, no protection against water. Power Consumption: 4W (max.) Key to symbols Adaptor; AC100-240V, 50/60Hz, 0.12A : Class II equipment Unit; DC6V, 500mA, with designated AC : The used electrical and electronic products are not household waste : 700hPa to 1060hPa Follow your national/local recycling rules to dispose of them properly. In the Operating Condition: +10°C to +40°C, 15% to 85% RH EU countries, please refer to waste management symbol(s) marked on the : -20°C to +60°C, 10% to 85% RH

This device complies with EN1060-1:1995+A2:2009 Non-invasive sphygmomanometers Part 1: General requirements, EN1060-3:1997+A2:2009 Non-invasive sphygmomanometers Part 3: Supplementary requirements for electro-mechanical blood pressure measuring system and EMC (IEC60601-1-2:2007). *Accuracy is guaranteed with the measured values that are within the measuring range.

PRECAUTIONS AND CONTRAINDICATIONS • Do not use this product without consultation with your doctor if you are under dialysis therapy or on

Specifications are subject to change without notice due to improvements in performance

anticoagulants, antiplatelets or steroids. Use of this instrument under such conditions could cause • For specific information on your blood pressure, contact your physician. Never make any judgment on

your own regarding measurement results. . Do not use this product along with implantable and wearable medical electrical equipment such as pacemaker, defibrillator, or electrocardiographic monitor. This product is not also intended to be used with HF surgical equipment.

. Do not use this product in an explosive environment such as near flammable anesthetics or inside • The system may fail to yield specified measurement accuracy if operated or stored in temperature or

humidity conditions outside the limits stated in the specifications section of this manual • Do not use cuffs or accessories other than those specified by the manufacturer. Otherwise, correct measurement readings cannot be obtained. Use of this product in areas near mobile phones, microwave ovens or other devices with strong

electromagnetic field may cause malfunctions. Maintain a minimum distance of 3 m from such devices

when using this unit. . Do not apply the cuff over wounded arm, arm under an intravascular access or therapy or an arterio-venous shunt, or arm on the side of a mastectomy. Otherwise injury may be resulted. • Make sure that inflation of the cuff is not causing prolonged impairment of blood circulation. Also, be

cautious about temporary loss of the functions of any other medical equipment if any monitoring . To avoid harmful injury due to interfered blood flow from cuff inflation.

conducted properly and prolonged, and • To avoid any possibility of accidental strangulation, keep this product away from children and do not drape

- Make sure that AIR HOSE is not kinking before measurement. Otherwise, cuff inflation may not be

AIR HOSE around your neck. . The unit contains small parts which could be swallowed by children. It should therefore be kept out of the

 Because the product includes precision parts, avoid extreme temperature variations, humidity, shock dust, and direct sunlight. Do not drop or strike the product. Make sure not to expose it to moisture. This product is not water resistant.

. Do not press the display and operation keys or place the unit with display face down . Do not take out batteries or unplug the AC adaptor when the unit is turned on. Make sure to switch off

the unit before removing batteries or AC adaptor. . Do not touch the output plug of AC adaptor during measure

. Do not disassemble or modify the product. . Do not inflate the cuff when it is not wrapped around your arm.

TIPS ON TAKING YOUR BLOOD PRESSURE

Blood pressure is the force exerted by the heart in pumping the blood through the arteries and the resistance by the veins in this flow Blood pressure varies all the time, influenced by mental and physical factors, and is neve

In general, blood pressure is highest during the working hours and gradually decreases during the afternoon and evening hours. It is low during sleep and increases at a relatively fast rate after arising

Causes for Changes in Blood Pressure Mental Tension Body movement Conversation Drinking Alcohol

 Recent Urination or Bowel Movement Room temperature Changes in the surroundings such as movement or noise, etc. Blood pressure measured at home tends to be lower than when measured in a hospital, clinic or doctor's

This is because you are tense at the hospital and relaxed at home. It is important to know your stable normal blood pressure at home. Let a qualified physician interpret your blood pressure readings

 $\label{lem:consult} \textbf{Depending on your age, weight and general condition, blood pressure can be slightly different. Consult}$ with your doctor on determining what blood pressure is normal for you. Before taking blood pressure, rest for approximately five minutes and take your blood

Measure blood pressure using the correct posture and do not move or speak during

Avoid exercise, eating, drinking alcohol, smoking and other activities that affect your blood

Take your blood pressure at the same time every day.

The ambient temperature should be approximately 20°C when you take your blood pressure. eading may vary slightly, depending on posture, whether sitting or lying.

INSTALLING BATTERIES (→fig.3) 1. OPENING THE BATTERY COMPARTMENT

ove the cover of the battery compartment on the underside of the unit. 2. INSERTING RATTERIES

Install four "AA" type batteries into the compartment, observing proper polarity (marking in battery

Close the battery compartment by clipping the battery cover back into the unit. 4. OPERATION WITH AC ADAPTOR

Connect the designated AC adaptor to the AC ADAPTOR JACK located at the right side of the unit. It is suggested to take out the batteries when using the adapte

CONNECTING THE CUFF (→fig.4) Connect the AIR PLUG to the AIR CONNECTOR at the left side of the unit.

Accurate measurement is not possible if the AIR PLUG is unplugged or loose. Make sure that the AIR PLUG

WRAPPING THE CUFF (→fig.5) 1. Place the cuff on your left arm with the air hose positioned toward your hand.

2. Wrap the cuff around your arm with the edge of the cuff approximately 2~3 cm above the elbow. AIR HOSE should be on the palm side of the arm. 3. Press the surface of the cuff to make sure that the hook & loop fastens securely.

When wrapping the cuff, wrap it loosely enough around the arm so that two fingers can be placed between the cuff and the arm. If the cuff is wrapped more tightly or loosely than this,

If you are wearing a shirt that might restrict circulation in your upper arm or you roll your sleeve up over the upper arm, the blood flow will be restricted, preventing accurate

Sit at the table and let the table support your arm as you take the blood pressure m Make sure that the measurement location on the upper arm is at approximately the same height as the heart, and that the forearm is extended naturally on the table and does not move. You may lie on your back and take the measurement. Look at the ceiling, stay calm, and do not move your

CORRECT MEASURING POSTURE (→fig.6)

approximately the same height as the heart. Measured data may vary slightly depending on the posture during measurement. If the cuff is ower (higher) than the heart, the measured reading tends to become larger (smaller). MAKING A MEASUREMENT (→fig.7)

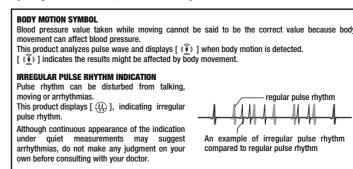
1. After the full display appears and disappears, deflation mark flashes and the calibration is adjusted to the 2. If measurement has been done after installing batteries.

To stop measurement Press START/STOP BUTTON and inflation stops, air is exhausted, and DATE TEHNICE A DISPOZITIVULUI

the previous reading is displayed while deflation mark 3. Automatic inflation starts and inflation mark flashes. 4. Inflation mark disappears and measurement begins. 5. Heart mark flashes as pulse is detected.

The unit automatically exhausts the air from the cuff as the measurement is complete Blood pressures and pulse rate are displayed. The reading is automatically saved in the bank

Press START/STOP BUTTON to turn off the power If you forget to turn off the unit, it will be automatically turned off after 3 minutes.

Do not execute repeated measurements for congestion of blood could result in false ement. Let vour arm rest for at least 5 minutes

CARE AND MAINTENANCE

Keep the product clean. Inspect its cleanliness after use. To clean, use only a soft dry cloth. Do not use gasoline, paint thinner, or other strong solvents. Since the cuff may absorb perspiration and other fluids,

When cleaning the cuff, use a synthetic detergent and gently rub the surface. Air dry thoroughly. Make sure fluid never gets in AIR HOSE. Do not machine wash or scrub it. When storing the product, do not place heavy objects on it. Do not fold or bend the cuff forcibly. To

nect the cuff from the main unit, do not pull AIR HOSE but hold AIR PLUG and unplug it. Do not coil AIR HOSE too tightly. When the product has been stored at a temperature below the freezing point, keep it for at least 1 hour in a warm place before using. Remove the batteries if the product is to be stored for an extended period of time. Keep the batteries out of reach

We suggest that you have your blood pressure monitor checked every 2 years. This operation may only be performed by the manufacturer or by firms authorized by the manufacturer.

TROUBLESHOOTING

Measurement is not taken with correct posture or the cuff is wrapped incorrectly

The reading was effected from moving or talking. Remain still and quiet during measuremen Measurement was taken right after exercise or other activities influencing the reading. Measure again after resting for more than 5 minutes. See TIPS ON TAKING YOUR BLOOD PRESSURE. sured values vary all the time

Mental and physical condition or measurement condition is influencing blood pressure. Make ent under the same condition red value is different from that taken at a hospital.

Mental condition such as nervousness at a hospital or relaxed feeling at home is influencing blood pressure. Take records of blood pressure taken at home and consult with your doctor. Blood pressure is changing 24 hours a day in relation with mental and physical condit

drinking, smoking, exercising or bathing will alter blood pressure. Blood pressure will be also influenced by mental condition such as feeling tensed or relieved and health condition It is important to know the tendency of blood pressure by making measurement regularly. Make your own rules including conditions and time to make measurement and take blood pressure everyday.

Over pressure: blood pressure could not be taken due to moving or speaking although the cuff was inflated to the maximum pressure. Do not move or speak during measuremen [Err -1] is displayed Blood pressure could not be taken due to moving or speaking. Do not move or speak during

[Err -2] is displayed The cuff is not securely connected to the unit. Secure the air connected

The cuff is not wrapped appropriately. Reconfirm how to put the cuff on your arm. [(a)] is displaye Batteries are weak, Replace all batteries with new ones

Batteries are depleted. Replace all batteries with new ones. Battery terminals are not clean. Clean the terminals with dry cloth.

Touching the keys with tip of finger or nail is touching the key. Touch the key with the plane of your [0 0] is displayed

"START/STOP" KEY was being touched when batteries are being installed or when AC adaptor is being plugged in. Turn off the unit once with "START/STOP" KEY.

Put the cuff with its hook and loop fastener side down. Put the cuff end through the metal ring. Fold back the cuff at the metal ring.

If you cannot get correct measurement with the methods above, contact your dealer. Do not disassemble or tamper with the internal mechanism.

his equipment is guaranteed for the period of 2 years after the date of purchase against manufacturing defects when returned along with the proof of date of purchase to the dealer from whom the purchase was made. During this period, the unit will be repaired or replaced free of charge if the failure is attributable to faulty design or manufacture. This warranty does not cover damage or malfunctions caused by imprope handling or use contrary to the instructions in this manual. Please contact your dealer for additional warranty provisions which may remain effective after the manufacturer's warranty period has expired.

NIHON SEIMITSU SOKKI CO., LTD. 2508-13 Nakago Shibukawa Gunma 377-0293 Japan EC-Representative: Nissei Healthcare (UK) Ltd. Henfield, BN5 9SJ UK http://www.nissei-kk.co.jp/english

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JEDNOSTKA ELEKTRONICZNA D PRZYCISK «WŁĄCZ/WYŁĄCZ» © GNIAZDO DO PRZYŁACZENIA ZASILACZA @ GNIAZDO PRZEWODU POWIETRZA @ KOMORA BATERII METALOWY PIERŚCIEŃ FIKSUJACY PRZEWÓD POWIETRZA ⊕ ZŁACZE PRZEWODU POWIETRZA MANKIET
Nissei Healthcare (UK) Ltd. Henfield, BN5 9SJ UK web site: http://www.nissei-kk.co.jp/english/