Norditalia

IT MANUALE D'USO

BP-500

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TABLE OF CONTENTS

Introduction	3
Symbols Used	3
Important Safety Notes	4
Appliance Description	5
Irregular Heartbeat (IHB) Indicator	7
Irregular Heartbeat Symbol (IHB)	7
Arrhythmia	7
Before Taking A Measurement	8
Battery Installation / Replacement	9
Settings	10
1. Memory area / user selection	10
2. Date and time	10
3. Save Settings	10
Proper Way of Measurement	11
Take a Measurement	11
Classification of Blood Pressure	12
Use the Memory Function	13
Last 3 Tests Average	13
To read the memory values	13
To delete the memory value	14
About Blood Pressure	14
Blood Circulation	14
Health and Blood Pressure	14
Hypertension	15
Hypotension	15
Exceptional Situations	16
Error Indicators	16
Fault Conditions	16
Care	17
Maintenance	17
Specifications	18
Warranty Conditions and Customer Care	19

Introduction

Thank you for purchasing Norditalia Fully Automatic Wrist Digital Blood Pressure Monitor. The appliance is based onto the oscillometric principle for blood pressure measurement. Please read this instruction manual thoroughly before using the appliance. For specific information about your own blood pressure, consult your doctor.

Symbols Used

SYMBOLS USED IN THIS INSTRUCTION MANUAL AND ONTO THE APPLIANCE	
	WARNING: read carefully the instruction manual.
	Type BF applied part.
	Manufacturing Date
	Compliant to european directive 93/42/EEC.
	Manufacturer for 93/42/EEC directive.
	Lot number.
	Dispose of the appliance and its accessories in compliance with the regulations in force. Do not dispose of this appliance in the environment!
	Keep dry


Important Safety Notes

- Read all the instructions carefully before using the appliance.
- Do not use the appliance for any purpose other than measuring blood pressure on adult. Do not use onto neonatal baby.
- Always consult your doctor for specific information about your blood pressure. Self-diagnosis and treatment using measured results may be dangerous.
- Never change your therapy without consulting your doctor.
- Do not measure your blood pressure after bathing, drinking alcohol, smoking, eating or physical exercise.
- Consult your doctor during pregnancy, arrhythmia and arteriosclerosis. People with severe blood flow problems, or blood disorders, should consult a doctor before using the appliance. Cuff inflation can cause internal bleeding.
- Do not leave the appliance unattended with infants or persons who cannot express their consent.
- If battery fluid should get on your skin or eyes, immediately rinse with plenty of clean water. Consult a doctor immediately.
- Use only alkaline batteries with this appliance. Respect the polarity. Remove immediately depleted batteries. Do not use old and new batteries together. Remove batteries if the appliance will not be used for three months or more.
- Do not disassemble or attempt to repair the appliance or components.
- Do not shock or drop the appliance.
- Do not wet the appliance and accessories including cuff.
- Do not inflate the wrist cuff when it is not wrapped around your wrist.
- Do not use a mobile phone, or other devices that emit electromagnetic fields, near the appliance. This may result in incorrect operation of the appliance.
- Do not use the appliance where flammable gas (such as anesthetic gas, oxygen or hydrogen) or flammable liquid (such as alcohol) are present.


Appliance Description

Body & Cuff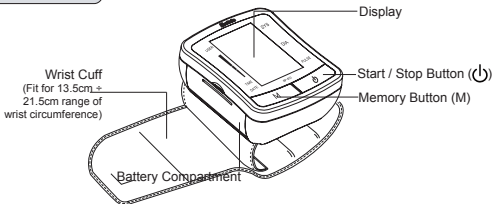

To guarantee the performances, functionalities and safety of the appliance, only use Norditalia original accessories and spare parts.

Display BP-500

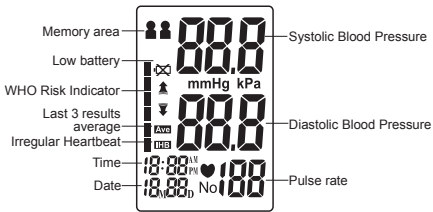Accessories

2 x AAA Alkaline Batteries

Storage Case

Irregular Heartbeat (IHB) Indicator

Irregular Heartbeat Symbol (IHB)

The appliance can detect irregular heartbeats.

When the appliance detects an irregular rhythm two or more times during the measurement, the irregular heartbeat Symbol will appear on the display with the measurement values.

An irregular heartbeat rhythm is defined as a rhythm that varies by more than 25% from the average rhythm detected during the measurement.

If the Irregular Heartbeat Symbol displays often with your measurement results, we recommend you consult your doctor.

Arrhythmia

Arrhythmia is a condition where the heartbeat rhythm is abnormal because of defects in the electrical stimulation of the heart.

Typical symptoms are: skipped heartbeats, premature contraction, too rapid pulse (tachycardia - atrial fibrillation) or too slow pulse (bradycardia).

A frequent appearance of the irregular heartbeat symbol could indicate arrhythmia but this can only be determined by your doctor through a specific test.

Self-diagnosis and self treatment are always dangerous, consult your doctor.

Before Taking A Measurement


1. Avoid eating, drinking alcohol or coffee, smoking, exercising and bathing for at least 30 minutes before taking a measurement.
2. Stress raises blood pressure. Avoid taking measurements during stressful times.
3. The cuff can be applied to your left or right wrist. Always measure your blood pressure on the same wrist. If there are high differences between the two wrists ask your doctor which wrist you should use.
4. Measurements should be taken in a quiet place.
5. Position the appliance at heart level throughout the measurement.
6. Remain still and do not talk during the measurement.
7. A single measurement does not provide an accurate indication of your true blood pressure. You need to take several readings over a period of time. Usually it is suggested to take 3 measurements and calculate the average. Please note the appliance can calculate automatically the average of last 3 measurements (L3). Please see "Use the memory function".
8. Your blood pressure varies during the day. Try to measure your blood pressure at the same time each day for consistency (for example within one hour after waking up).
9. It is normal for two measurements close in time to give even very different values.
10. Wait 2 or 3 minutes between measurements. The wait time allows the arteries to return to normal conditions.
11. Measurement taken by your doctor can be different from the measurements you get at home as the situation is different.

Battery Installation / Replacement

Remove the battery cover

Install two "AAA / LR03" size batteries so the + (positive) and - (negative) polarities match the polarities indicated in the battery compartment..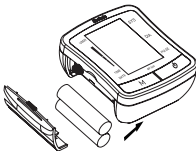

Replace the battery cover.
NOTE: stored measurements will not be deleted during battery replacement


Settings

With power off, press "⏻" button about 3 seconds to activate system setting.

1. Memory area / user selection

The device can store 120 memories divided in 2 different memory areas (1 and 2). Memory areas could be used for storing the measurements of 2 different users or a single user could decide to keep 2 different groups of measurements separated (ex: summer/winter, before/after therapy, etc...)

Press "M" button to choose the memory area

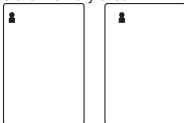2. Date and time

Press "⏻" button again to set the Time/Date mode. Set the month first by adjusting the "M" button. Press "⏻" button again to confirm current month. Continue setting the day, hour and minute in the same fashion.

3. Save Settings


While in any setting mode, press "⏻" button about 3 seconds to turn the unit off. All information will be saved.

Proper Way of Measurement

1. Remove all clothes from your wrist allowing the cuff to fit directly on the skin.
2. Wrap the cuff tightly around your wrist by using the Velcro strip.

3. Sit with your feet flat on the floor. The wrist have to be at heart level during the measurement.
4. Relax your arm, keep the palm upward and the fingers curving naturally.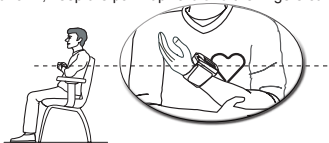

Take a Measurement

1. Press the "⏻" button. All display symbols appear.
2. Measurement starts while inflating. The hearbeat symbol flashes once a pulse is detected.
3. The blood pressure and pulse rate are displayed when the measurement is completed.

1-Start

2-Inflating

3-Deflating


4-Result

- The cuff is deflated automatically, and all of the measurement results are stored in the memory with date and time. The IHB symbol will be displayed if irregular heartbeat is detected.
- Press the "⏻" button to turn off the unit. The appliance will automatically turn off after 3 minutes if not used.

NOTE: To stop the inflation or measurement, press the "⏻" button anytime. The unit will stop inflating and then release and turn off.

Classification of Blood Pressure

After each measurement is completed, the LCD display will show your position automatically on the six segments of the bar indicator which corresponds to World Health Organization (WHO) Risk Indicator. Reference Material: Journal of Hypertension 1999, Vol 17 No.2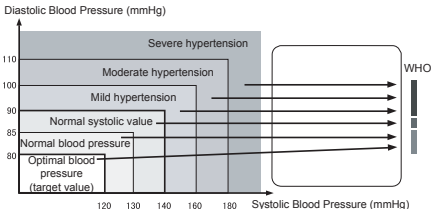

Note:

When a person's systolic and diastolic pressures fall into different categories, the higher category should apply.

Use the Memory Function

After every measurement the appliance memorizes the measured blood pressure and pulse rate with date and time. It can store up to

BP-500: 120 sets of measurements (60 for each of the 2 memory areas).

When all sets of an area are full, the oldest memory will be deleted. The unit also calculates the average of last 3 measurements.

Please select desired memory area before reading the memory.

Last 3 Tests Average


With power off, press the "M" button to activate screen display. The device will display the average of the last 3 readings. The "Ave" symbol will appear.

Press "M" again to cycle through memories. Select the desired memory area first.

To read the memory values


Press "M" again to cycle through memories.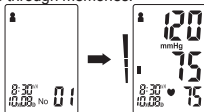

To delete the memory value

Memory for a selected memory area may be deleted while in Memory Check mode. Press and hold the "⏻" button for approximately 3 seconds to delete all memory records from the selected group. The monitor will beep indicating successful deletion and then transfer into testing mode.

If you do not press any button for 3 minutes, the unit will turn off automatically and the memory will not be deleted.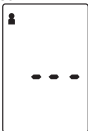

Note: Memory cannot be recovered once it has been deleted.

About Blood Pressure

Blood Circulation

The blood circulation is responsible for supplying the body with oxygen. Blood pressure is the pressure exerted on blood vessels walls.


The systolic blood pressure value (top value) represents the blood pressure produced by contraction of the heart muscle.

The diastolic blood pressure value (lower value) represents the blood pressure produced by relaxation of the heart muscle.

Health and Blood Pressure

The incidence of hypertension increases with age. In addition, a lack of exercise, excess of body fat and high levels of cholesterol (LDL), which sticks to the inside in blood vessels, reduces elasticity of these vessels. Hypertension accelerates arterial sclerosis which can lead to vary serious conditions such as stroke and myocardial infarction. For these reasons it is very important to know wheter the blood pressure is within a healthy range. Blood pressure fluctuates from

minute to minute, throughout the day. Therefore it is essential to take regular measurements to help you identify an average blood pressure.

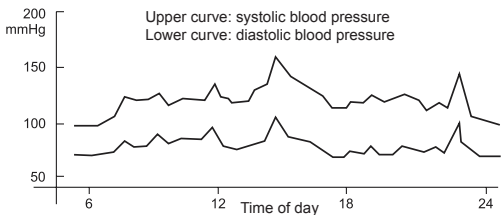Hypertension

High blood pressure can go unnoticed for a long time, since it does not produce noticeable symptoms. If your blood pressure falls in an “hypertension” area in repeated measurements over several days, you should consult your doctor for detailed medical examination.

Self- diagnosis and treatment using measured results may be dangerous. Never initiate, change or suspend a therapy without consulting your doctor.

Hypotension

The definition of hypotension is not universally accepted. Nevertheless, a systolic pressure lower than 100 mmHg is assumed as hypotensive.

Exceptional Situations

Error Indicators

The following symbols will appear on the display in case of malfunction.

Problem	Possible cause	Remedy
	Improper operation	Read user manual carefully and re-test properly.
	Pressurization is over 300mmHg	Read user manual carefully and re-test properly.
	Low battery power	Replace all of the worn batteries with new ones.

Fault Conditions

Problem	Causes and Solutions
No power. No display appears on the display screen.	Check the battery installation for proper placement of the batteries polarities. If necessary replace batteries.
Measurement values appear too high or too low.	Blood pressure varies constantly. Many factors including stress, time of day, how you wrap the cuff, may affect your blood pressure. Review the sections "Proper Way of Measurement". Repeat the measurement

Care

Keep the unit in the storage case when not in use.
Clean the unit with a soft dry cloth. Do not use any abrasive or volatile cleaners.
Never immerse the unit or any components in water.

Avoid subjecting the monitor to strong shocks or vibrations, such as dropping the unit on the floor.
Use the unit consistent with the instructions provided in this manual. Use only authorized parts and accessories.

Maintenance

Do not clean the body and cuff with naphtha, thinner or gasoline, etc.
Do not wet the cuff or attempt to clean the cuff with water.
Store the unit in a clean and dry location. Do not subject the unit to extreme hot or cold temperature, humidity and direct sunlight.
Remove the batteries if the unit will not be used for 3 months or longer. Always replace all of the batteries with new ones at the same time.

Specifications

Model	BP-500	
Display	LCD Digital Display	
Measuring principle	Oscillometric method	
Measurement position	Wrist	
Measurement Range	Pressure	0-300 mmHg
	Pulse	30-180 Pulses/min
Accuracy	Pressure	±3 mmHg
	Pulse	±5% of reading
Inflation and deflation	Automatic	
Memory function	2 areas with 60 memories for each	
Power source	2 x AAA/LR03 Alkaline Batteries	
Automatic turn off	In 3 minutes	
Main unit weight	Approx. 140g. (batteries not included)	
Wrist Cuff	Universal Wrist Cuff, suitable for 13.5-21.5 cm wrist circumference.	
Operating Environment	Temperature	5 ÷ 40°C
	Relative Humidity	15 ÷ 85%RH Noncondensing
	Atmospheric Pressure	700 ÷ 1060 hPa

Specifications are subject to change without notice

Warranty Conditions and Customer Care

The product is guaranteed for 2 years from the purchase unless differently required by local regulations against manufacturer's defect under normal, household use with respect of present manual instructions. The warranty is valid only if the device is sent back to the supplier together with the receipt proving the date of purchase and the warranty card duly filled. The warranty does not cover consumables, expiring components and accessories other than the device itself (ex: batteries, cuff, etc...).

For assistance in obtaining accessories or about the use of the device, please fill the form available in the "assistance" section on the website www.norditalia.biz or contact us using the referrals indicated in the manual. If necessary contact your dealer. Periodically (every 2 years), or in case of doubt about the accuracy of the appliance, compare your results with a different appliance. In case of systematic differences higher than 30 mmHg contact your dealer.

CERTIFICATO DI GARANZIA / WARRANTY CARD / بطاقة الضمان / TARJETA DE GARANTÍA / كارت گارانتي / GARANTNI LIST / KARTA EGÝHSHS / CARD DE GARANȚIE / ΓΑΡΑΝΤΙΟΝΗ ΚΑΡΤΑ

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