SANITAS

SBC 28

- D** **Gebrauchsanleitung**
Blutdruckmessgerät (2–10)
- GB** **Instruction for Use**
Wrist blood pressure monitor (11–17)
- F** **Mode d'emploi**
Tensiomètre au poignet (18–25)
- I** **Istruzioni per l'uso**
Misuratore di pressione (26–33)
- RUS** **Инструкция по применению**
Прибор для измерения давления на запястье (34–42)

Serviceadresse:
Hans Dinslage GmbH
Riedlinger Straße 28
88524 Uttenweiler, GERMANY
Tel.-Nr.: +49 (0) 7374-915766
Fax-Nr.: +49 (0)7374-920723
E-Mail: service@sanitas-online.de

CE 0483

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Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

1. Getting to know your instrument

The wrist blood pressure monitor is used for non-invasive measurement and monitoring of adults' arterial blood pressure.

You can use it to measure your blood pressure quickly and easily, storing the results and displaying the progression of readings. A warning is issued for anyone suffering from cardiac arrhythmia.

The values determined are classified and graphically evaluated according to WHO guidelines.

Keep these instructions carefully for further use and also let other users have access to them.

2. Important information

Advice on use

- Always measure your blood pressure at the same time of day, in order to ensure that values are comparable.
- Relax for approx. 5 minutes before each measurement.
- You should not eat, drink, smoke or do exercise for at least 30 minutes before taking the measurement.
- You should wait five minutes before measuring a second time.
- The readings that you take may only be used for information purposes – they are not a substitute for a medical examination! Discuss your results with the doctor. Never use them to

make medical decisions independently (e.g. regarding medication and dosage thereof).

- If circulation on one arm is restricted as a result of chronic or acute vascular disease (including vasoconstriction), the precision of any wrist measurement is impaired. In this case, use a blood pressure monitor for the upper arm.
- There may be incorrect measurements where there is cardiocirculatory disease, as well as with very low blood pressure, circulatory disorders, dysrhythmia and other preexisting diseases.
- For hygienic reasons the cuff is intended only for the use by one person.
- Only use the instrument on persons whose wrist has the right circumference for the instrument.
- You may only operate the blood pressure monitor with batteries. Note that it is only possible to save data when your blood pressure monitor is receiving power. As soon as the batteries are dead, the date and time on the blood pressure monitor are lost. Any measurement results that have been stored are however retained.
- The automatic switch-off function switches off the blood pressure monitor in order to preserve the batteries, if no button is pressed within one minute.

Storage and Care

- The blood pressure monitor is made up of precision electronic components. Accuracy of readings and the instrument's service life depend on careful handling.

- You should protect the device from impact, moisture, dirt, major temperature fluctuations and direct exposure to the sun's rays.
- Never drop the device.
- Do not use near strong electromagnetic fields, i.e. keep it away from any radio systems and mobile phones.
- Do not press any buttons until the cuff is in position.
- If the instrument is not used for any length of time, we recommend removing the batteries.

Advice on batteries

- Batteries can be fatal if swallowed. You should therefore store the batteries and products where they are inaccessible to small children. If a battery has been swallowed, call a doctor immediately.
- Batteries should not be charged or reactivated with any other means, nor should they be taken apart, thrown in the fire or short-circuited.
- Remove the batteries from the instrument if they are worn out or if you are not going to use the instrument for any length of time. This prevents any damage as a result of leakage. Always replace all the batteries at the same time.
- Never use different types of battery, battery brands or batteries with different capacities. You should preferably use alkaline batteries.

Repair and disposal


- Batteries do not belong in domestic refuse. Used batteries should be disposed of at the collection points provided.

- Never open the instrument. If these instructions are not heeded, the warranty will be null and void.
- Never attempt to repair the instrument or adjust it yourself. We can no longer guarantee perfect functioning if you do.
- Repairs may only be performed by Customer Service or authorized dealers. However, always check the batteries and replace them if necessary prior to making any complaint.
- The appliance should be disposed of according to Regulation 2002/96/EC-WEEE (Waste Electrical and Electronic Equipment). In case of queries, please contact the municipal authorities responsible for waste disposal in your area.


3. Unit description

1. Display
2. START/STOP button 
3. User-Switching Key 
4. Memory button +/M
5. Date/Time Set Key 
6. Wrist cuff
7. Battery cover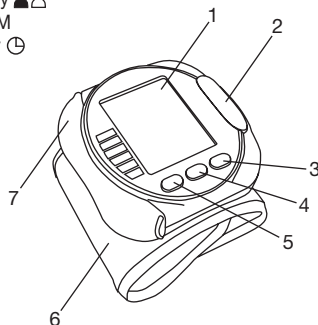

Icons in the display:

1. Battery level indicator
2. Storage space
3. Average value of the 3 most recent measurements - AVG
4. Systolic pressure
5. Diastolic pressure
6. Measured pulse
7. Cardiac arrhythmia icon
8. Pulse icon
9. Memory Zones
10. WHO - classification
11. Pump up, release air (arrow)
12. Time and date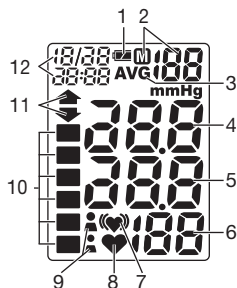

4. Prepare measurement

Inserting battery

- Remove the battery cover (left side of the unit).
- Insert two 1.5 V micro batteries (alkaline, type LR03).
Make absolutely sure that you insert the batteries with the correct polarity as marked. Do not use rechargeable batteries.
- Replace the battery cover carefully.
- The symbol appears if the batteries are new.


If the battery change symbol is flashing and E6 appears, no more measurements are possible and you must replace all batteries.

After inserting the batteries, the year appears on the display and you can then set the date and time.

They are considered to be toxic waste and, as such, should be disposed of in special containers, toxic waste collection points or brought to electrical goods dealers. You are legally obliged to dispose of the batteries.

Note: You will find these markings on batteries containing harmful substances: Pb = battery containing lead, Cd = battery containing cadmium, Hg = battery containing mercury.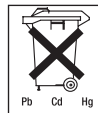

Setting date and time

It is vital to set the date and time. This is the only way that measurements can be saved and recalled with the correct date and time.

Proceed as follows to set the date and time:


1. After inserting the batteries:
 - The year appears on the display. Using the +/M button, you can increase the basic values to be set. Set the year and confirm the entry using the button.
 - Now set the month using the +/M button. Confirm using the button.
 - Set the day/hour/minute and confirm each using the button.
 - To switch off, press the Start/Stop button .

2. At any time:

- Press the  button to display the date/time and current user memory.
- Press and hold the  button again for about 3 seconds to set time/date.
- Set as described above.

5. Measuring blood pressure

Positioning cuff

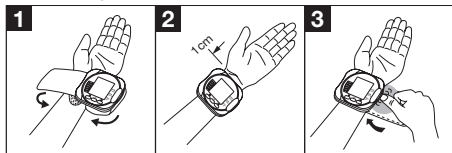- Bare your left wrist, making sure that the circulation in the arm is not restricted by any clothes etc. that are too tight. Position the cuff on the inside of your wrist.
- Fasten the cuff with the Velcro fastening so that the upper edge of the monitor is positioned approx. 1 cm below the ball of your thumb.
- The cuff has to be fitted tightly around the wrist but should not constrict it.

Correct posture

- Rest for approx. 5 minutes before each measurement. Otherwise there may be divergences.
- You can perform the measurement either sitting or lying down. It is essential to support your arm and have it at an angle. Always make sure that the cuff is level with your heart. Otherwise there may be serious divergences. Relax your arm and the palms of your hands.
- In order not to distort the result, it is important to keep still during the measurement and not talk.

Select memory

Press the  button to display the date/time and current user memory. Select the desired user memory by pressing the  function button. Confirm your selection with the  button. You have two memories (60 memory spaces each) in order to save the test results of 2 different people separately, or else save measurements in the morning and evening separately.

Measuring blood pressure

- Put on the cuff as described previously and assume the position in which you want to carry out the measurement.
- Select the user memory using  or . Start the measurement process by pressing button . The cuff pumps itself up automatically. Following the self-test, during which all display elements briefly appear, the measurement begins. During inflation the device already determines measured values used to estimate the required inflation pressure. If this pressure is insufficient, the device automatically inflates by another.

- Then the pressure in the cuff is slowly released and the pulse is measured.
- When the measurement has been completed, the remaining air pressure is released very quickly. The pulse, the systolic and the diastolic blood pressure are displayed.
- You can interrupt measurement at any time by pressing the button .
- The E_u icon appears if it has not been possible to perform the measurement properly. Observe the section in these instructions on error messages/troubleshooting and repeat the measurement.
- The device switches off automatically after 1 minute.

Wait at least 5 minutes before taking another measurement!

Evaluating results

Cardiac arrhythmia:

This instrument can identify possible cardiac arrhythmia disorders during measurement and if necessary indicates the measurement with the flashing icon .

This may be an indicator for arrhythmia. Arrhythmia is a condition where the heart rhythm is abnormal as a result of defects in the bioelectrical system controlling the heart beat. The symptoms (omitted or premature heart beats, slow or excessively fast heart rate) may be caused, among other things, by heart disease, age, physical predisposition, excessive use of stimulants, stress or lack of sleep. Arrhythmia can only be ascertained through examination by your doctor.

Repeat the measurement if the flashing icon  is displayed after the measurement. Please note that you should rest for 5 minutes between measurements and not talk or move during the measurement. If the icon  appears often, please contact your doctor. Any self-diagnosis and treatment based on the test results may be dangerous. It is vital to follow your doctor's instructions.

WHO classification:

According to WHO Guidelines/Definitions and the latest findings, the test results can be classified and evaluated according to the following chart:

Range of blood pressure values	Systolic (in mmHg)	Diastolic (in mmHg)	Measure
Grade 3: Severe hypertension	≥ 180	≥ 110	Seek medical advice
Grade 2: Moderate hypertension	160–179	100–109	Seek medical advice
Grade 1: Mild hypertension	140–159	90–99	Have it checked regularly by doctor
High-normal	130–139	85–89	Have it checked regularly by doctor
Normal	120–129	80–84	Check it yourself
Optimal	< 120	< 80	Check it yourself

Source: WHO, 1999

The bar graph in the display and the scale on the unit indicate the range of the blood pressure which has been recorded.

If the values for systolic and diastolic pressure are in two different WHO ranges (e.g. systolic in the high-normal range and diastolic pressure in the normal range) the graphic WHO classification on the unit indicates the higher range (high-normal in the example described).

6. Saving and displaying measurements

- The results of each successful measurement are stored together with date and time. With more than 60 items of measured data, the earliest items of data measured are lost.
- Select the desired user memory by pressing the  button and then the  button. Pressing the +/M button displays the average value of the 3 most recent measurements stored in the user memory. If you continue pressing the +/M button, the latest individual results are displayed with date and time.

7. Deleting stored values

To delete individual measurements, you must first select them. Press and hold the memory button +/M until CL appears in the display. Then confirm the deletion with the user selection button  (CL and 00 appear in the display).

To delete measurements from a user memory, you must first select them. Press the memory button +/M and the average value of the last 3 saved measurements is displayed. Then press and hold the memory button +/M again until CL appears in the display. Confirm the deletion with the user selection button  (CL and 00 appear in the display).

8. Error messages/troubleshooting

In case of faults, the E_ message appears in the display.

Error messages can occur when

1. the cuff is too tight or too loose (Er1),
2. you move or talk during the measurement (in addition to (E2), the cardiac arrhythmia icon  appears in the display),
3. the cuff is inflated for longer than 25 seconds (E3).
4. errors occur during measuring (E4),
5. inflation pressure is higher than 300 mmHg (E5).
6. the batteries are almost empty  (E6).

In the above cases, you must repeat the measurement. Make sure that the cuff tube is properly inserted and that you do not move or talk. Re-insert the batteries if necessary, or else replace them.

9. Cleaning and Care

- Clean your blood pressure computer carefully using only a slightly moistened cloth.
- Do not use any detergents or solvents.
- Never hold the instrument under water as otherwise liquid can penetrate and damage the instrument.
- Never place any heavy objects on the instrument.

10. Technical details

Model no.	SBC 28
Measurement method	Oscillating, non-invasive blood pressure measurement on the wrist
Measurement range	systolic 50–250 mmHg, diastolic 30–200 mmHg, pulse 40–180 beats/minute

Precision of pressure reading	Systolic ± 3 mmHg/ diastolic ± 3 mmHg/ pulse ± 5 % of displayed value
Measurement uncertainty	Max. admissible standard deviation according to clinical examination. systolic 8 mmHg/diastolic 8 mmHg
Memory	2 x 60 storable data lines
Dimensions	L 75 x W 75 x H 28 mm
Weight	Approx. 116 g
Cuff size	For a wrist circumference of approx. 135 to 195 mm
Admissible operating temperature	+10 °C to +40 °C, 30–85 % relative humidity
Admissible storage temperature	-20 °C to +60 °C, 10–95 % relative humidity
Power supply	2 x 1.5 V Micro (Alkaline Type LR03)
Battery life	For 300 measurements
Accessories	Storage box, instructions for use, 2 x AAA batteries
Classification	Application part type BF
Protection class	internal power supply, IPXO, no AP or APG, continuous operation
Explanation of symbols	Application part type BF  Caution! Read the Instructions for Use. 

- This unit is in line with European Standard EN60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility (EMC). Please note that portable and mobile HF communication systems may interfere with this

instrument. For more details, please contact customer service at the address indicated.

- The instrument is in line with the EU Medical Products Directive 93/42/EC, the German medical products act and European standards EN1060-1 (non-invasive blood pressure monitors, Part 1: general requirements) and EN1060-3 (non-invasive blood pressure monitors, Part 3: Supplementary Requirements for Electromechanical Blood Pressure Measurement Systems).

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