TOPCOM®

Blood Pressure Monitor

BD-4627

USER GUIDE / HANDLEIDING / MANUEL D'UTILISATEUR
GEBRAUCHSANWEISUNG / BRUKSANVISNING / MANUAL DE USUARIO
/ MANUALE D'USO / MANUAL DO UTILIZADOR
UŽIVATELSKÁ PŘÍRUČKA/ INSTRUKCJA OBSŁUGI
UŽÍVATEĽSKÝ MANUÁL

V 1 - 10/12

CE 0197

- UK** The features described in this manual are published with reservation to modifications.
- NL** De in deze handleiding beschreven mogelijkheden worden gepubliceerd onder voorbehoud van wijzigingen.
- F** Les possibilités décrites dans ce manuel sont publiées sous réserve de modifications.
- D** Die in dieser Bedienungsanleitung umschriebenen Möglichkeiten, werden vorbehaltlich Änderungen publiziert.
- S** Funktionerna i denna bruksanvisning publiceras med reservation för ändringar.
- ES** Las características descritas en este manual pueden ser objeto de futuras modificaciones.
- IT** Le caratteristiche descritte nel presente manuale vengono pubblicate con riserva di modifica.
- PT** As características descritas neste manual são publicadas sob reserva de modificação.
- CZ** Možnost úpravy funkcí popsaných v této příručce vyhrazena.
- PL** Właściwości opisane w niniejszej instrukcji obsługi są publikowane z zastrzeżeniem prawa wprowadzenia zmian.
- SK** Vlastnosti popísané v tejto príručke sú publikované s vyhradeným právom na zmenu.

Distributor:

Tristar Europe BV,
Jules Verneweg 87, 5015 BH Tilburg, The Netherlands,

Manufacturer

Health & Life Co., Ltd.
9F No.186, Jian Yi Road, Zhonghe District, New Taipei City, Taiwan
www.healthandlife.com.tw

EMERGO EUROPE

Molenstraat 15, 2513 BH, The Hague, The Netherlands

Keep dry

This Blood Pressure Monitor complies with the EC Directives and bears the CE Mark "CE 0197".

This blood pressure monitor also complies with mainly following standards, (including but not limited).

Safety standards

EN 60601-1 - Medical electrical equipment part 1: General requirements for safety

EMC Standard

EN 60601-1-2 Medical electrical equipment part 1-2: General requirements for safety-
Collateral standard: Electromagnetic compatibility - Requirements and tests

Performance standards:

EN 1060-1 Non-invasive sphygmomanometers - General requirements

EN 1060-3 Non-invasive sphygmomanometers - Supplementary requirements for electromechanical blood pressure measuring systems.

EN 1060-4 Non-invasive sphygmomanometers - Test procedures to determine the overall system accuracy of automated non-invasive sphygmomanometers.

The declaration of conformity can be found on:

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www.tristar.eu

1 Introduction

Congratulations with your purchase of the Topcom . This fully automatic, wrist mounted blood pressure monitor is both easy to use and ideally suited for daily measurements. The large display includes systolic, diastolic and pulse, all clearly shown at the completion of each reading.

In addition, you can store up to 40 measurements per Memory Zone, ideal for users who want to monitor and track their blood pressure on a regular basis. The is compact and portable, making it ideal for home use and travel.

This device uses the oscillometric method to automatically measure systolic and diastolic blood pressure as well as heart rate.

Follow instructions for use.

2 Safety instructions

1. This product is for household use only. It is not intended as a substitute for consultation with your doctor.
2. For users diagnosed with common arrhythmia (atrial or ventricular premature beats or atrial fibrillation), diabetes, poor circulation of blood, kidney problems, or for users suffered from stroke, or for unconscious users, the device might not be suitable for use. Consult your doctor in case of any doubt.
3. The unit should not be operated by children so to avoid hazardous situations.
4. The unit contains high-precision assemblies. Therefore, avoid extreme temperatures, humidity, and direct sunlight. Avoid dropping or strongly shocking the main unit, and protect it from dust. Leaky batteries can damage the unit. Remove the batteries when the unit is not used for a long time.
5. Do not press the start key when the cuff is not placed around the wrist.
6. Do not disassemble the main unit or cuff.
7. Do not store the device in extremely low (less than $-13^{\circ}\text{F}/-25^{\circ}\text{C}$) and high (more than $158^{\circ}\text{F}/70^{\circ}\text{C}$) temperature, nor in a place its humidity exceeds 93% R.H..
8. Clean the blood pressure monitor body and the cuff carefully with a slightly damp, soft cloth. Do not press. Do not bend the pre-formed cuff inside out. Do not wash the cuff or use chemical cleaner on it. Never use thinner, alcohol or petrol (gasoline) as cleaner.
9. If you have one of the circulatory problems as arteriosclerosis, diabetes, liver disease, kidney disease, severe hypertension, peripheral circulation..., please consult your healthcare professional before using the device.
10. This product is not suitable for people with arrhythmias and pregnant women.

To avoid inaccurate results caused by electromagnetic interference between electrical and electronic equipments, do not use the device near a cell phone or microwave oven. At least keep a maximum output power of 2 W yields and a distance 3.3m away from this equipment.

3 Tips

Here are a few helpful tips to help you obtain more accurate readings:

- Do not measure your blood pressure immediately after consuming a large meal. To obtain more accurate readings, please wait one hour before measuring.
- Do not smoke or drink alcohol before measuring your blood pressure.
- You should not be physically tired or exhausted while taking a measurement.
- Rest at least 5~10 minutes before taking a measurement.
- Do not take measurements if you are under stress or under tension.
- Take your blood pressure at normal body temperature. If you are feeling cold or hot, wait a while before taking a measurement.
- If the monitor is stored at very low temperature (near freezing), have it placed at a warm location for at least one hour before using it.
- Relax at least 3~5 minutes in between measurements.
- Wait 30 ~ 45 minutes before measurement if you've just consumed beverages containing caffeine or smoked cigarettes.

4 Disposal of the device

At the end of the product lifecycle, you should not throw this product into the normal household garbage but bring the product to a collection point for the recycling of electrical and electronic equipments. The symbol on the product, user guide and/or box indicate this.

Some of the product materials can be re-used if you bring them to a recycling point. By re-using some parts or raw materials from used products you make an important contribution to the protection of the environment. Please contact your local authorities in case you need more information on the collection points in your area.

5 Blood pressure

5.1 What is blood pressure?

Blood pressure is the pressure exerted on the artery tube while blood flows through the arteries. The pressure measured when the heart contracts and sends blood out of the heart is systolic (highest). The pressure measured when


the heart dilates with blood flowing back into the heart is called diastolic (lowest) blood pressure.

5.2 Why measure your blood pressure?

Among the various health problems afflicting modern people, problems associated with high blood pressure are by far the most common. High blood pressure's dangerously strong correlation with cardiovascular diseases and high morbidity has made measuring blood pressures a necessity of identifying those at risk.

5.3 Blood pressure standard

The World Health Organization (WHO) has developed a Blood pressure standard, according to which areas of low and high-risk blood pressure are identified. This standard however, is a general guideline as individual's blood pressure varies among different people and different groups...etc.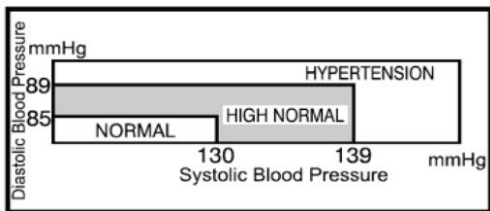


It is important that you consult with your physician regularly. Your physician will tell you your normal blood pressure range as well as the point at which you will be considered at risk.

5.4 Blood pressure fluctuation

Blood pressure fluctuates all the time!

You should not be worried if you encountered two or three measurements at high levels. Blood pressure changes over the month and even throughout the day depending on circumstances (state of mind, temperature, ...)

6 Battery installation

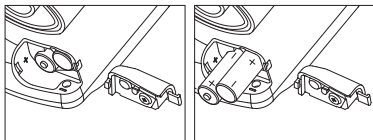
- Open the battery compartment by pulling the cover away
- Insert 2 AAA Alkaline batteries. Follow the polarity as illustrated.
- Re-install the battery cover.


Do not shortcircuit & dispose of in fire. Remove the battery if this device is not going to be used for a long period.

The batteries must be replaced when the battery low icon  is displayed. All batteries used must be the same type. Do not mix alkaline, standard (carbon-zinc) or rechargeable (cadmium) batteries. Do not mix old and new batteries.

7 Buttons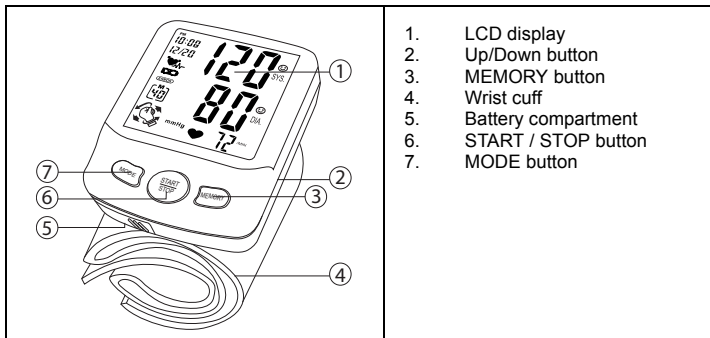

8 Display

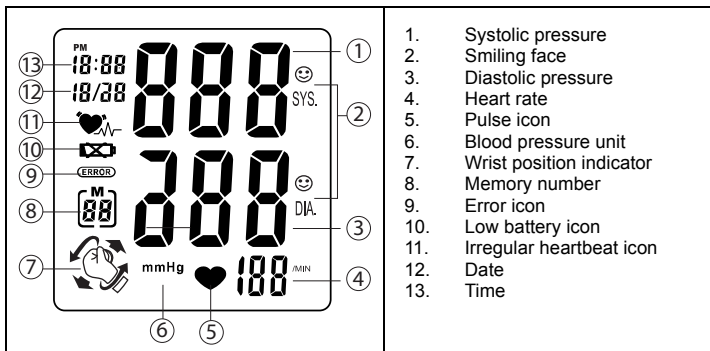
9 Settings

9.1 Time & date adjustment

In stand-by:

- Press the **MODE** button, the month will blink on the display.
- Press the **UP** or **DOWN** button to change the month.
- Press the **MODE** button to confirm the month. The day will blink on the display.
- Change the day, hours and minutes as described above, using the **UP** or **DOWN** button to change and the **MODE** button to confirm every setting.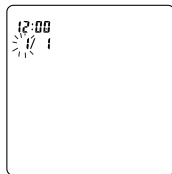

After setting the minutes, the device is ready to use.

9.2 Self-management function

The monitor allows you to pre-set your personalized values of blood pressure which should be recommended or decided by your physician or medical professional.

If the Systolic or Diastolic rate of the measurement is greater than your pre-set personalized level, the digit of the result on the screen will flash to alert.

Besides, either the Systolic or Diastolic rate of the measurement is equal to or less than your pre-set personalized levels, or you don't set the personalized levels, the smiling face 😊 icon will appear and flash on screen after each measurement.

3 different personal levels can be set. The levels are related to the memory zone (see chapter 14)

To set your personal blood pressure levels:

After setting the minutes, the setting of personal Blood pressure level limits is not active. (- -) is displayed.

- Press the **MODE** button, the Systolic pressure limit is flashing.
- Press the **UP** or **DOWN** button to set your personal systolic pressure limit.
- Press the **MODE** button to confirm. The diastolic pressure level is flashing.
- Press the **UP** or **DOWN** button to set your personal diastolic pressure limit.
- Press the **MODE** button to confirm.

Repeat previous steps for setting the levels for memory zone 2 and 3.


After setting you personal blood pressure level, the 3 positioning system can be (de-) activated.

9.3 3D Wrist position system


Keeping the wrist at your heart level while measuring is extremely important to get accurate blood pressure values.

This monitor features a Wrist Position Sensor which guides you to adjust your posture by the WRIST POSITION INDICATOR symbols shown on screen and helps to determine your optimal height of the wrist for measurement.


Wrist position indicator function in measurement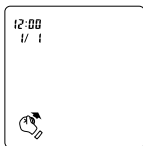


Move wrist upwards

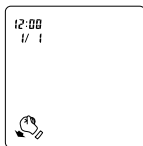Move wrist downwards

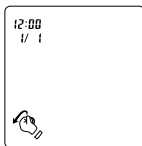Rotate wrist counter-clockwise

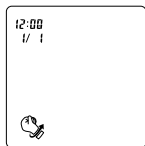Rotate wrist clockwise


To switch the 3d positioning on or off:

After setting your personal blood pressure levels:

- Press the **MODE** button, the '3d On' is displayed.
- Press the **UP** or **DOWN** button to set the function on or off.
- Press the **MODE** button to confirm.
- The monitor is ready to be used.

10 Applying the cuff

1. Remove all watches, jewelry, etc. prior to attaching the wrist monitor. Clothing sleeves should be rolled up and the cuff should be wrapped on bare skin for correct measurements. Apply cuff to left wrist with palm facing up.
2. Make sure the edge of the cuff is about 1 cm from the palm. In order to insure accurate-measurements.
3. Fasten the velcro strap securely around your wrist so there is no extra space between the cuff and the wrist. If the cuff is not wrapped tight enough, the measurement values will be false.
4. If your physician has diagnosed you with poor circulation on your left arm, place the cuff around your right wrist.

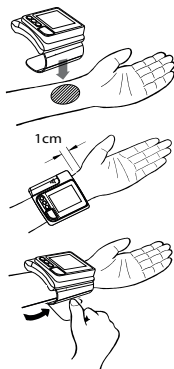


11 Measuring posture

1. Relax your entire body, especially the area between your elbow and fingers. Place your elbow on a table so that the cuff is at the same level as your heart.

Your heart is located slightly below your arm pit.

2. If the cuff is not at the same level as your heart or if you can not keep your arm completely still throughout the reading, use a soft object such as folded towel to support your arm.
3. Turn your palm upwards. Sit upright in a chair, and take 5-6 deep breaths. Avoid leaning back while the measurement is being taken.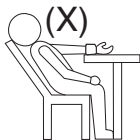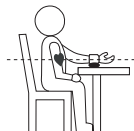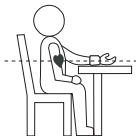


12 Measurement

12.1 Important

1. This monitor automatically switches off 1 minute after the last key operation.
2. To interrupt the measurement, simply press any key. The cuff will deflate immediately after a key is pressed.
3. During the measurement, do not talk or move your arm or hand muscles.
4. The result will be stored automatically in the selected Memory Zone. Select the desired Memory Zone before measurement (see chapter §14)


13 Procedure

- Press the **START/STOP** button to activate the display.
- Press the **UP** or **DOWN** button to select the desired memory zone.
- Press the **START/STOP** button to start measurement.
- The 3d positioning system will help you to position your wrist at the correct level.
- When your wrist is at the correct position, the cuff will inflate to the level that is right for you.
- When the right level is reached, the cuff will deflate.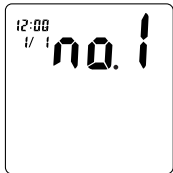

This monitor will re-inflate automatically to approximately 220mmHg if the system detects that your body needs more pressure to measure your blood pressure.

- When the measurement is completed, systolic, diastolic and pulse will be shown simultaneously on the LCD screen.

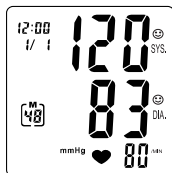13.1 Display icons

- If the Systolic or Diastolic rate of the result is greater than your pre-set personalized level (see chapter 9.3), the digit of Systolic or Diastolic rate will flash.
- If the Systolic or Diastolic rate of the result is equal to or less than your pre-set personalized level, the smiling face icon ☺ will flash.
- When a mistake was detected during measurement the Error icon **ERROR** appears on the display.

13.2 Irregular heartbeat (IHB)

The appearance of this symbol, , signifies that a certain pulse irregularity was detected during measurement.

Talking, moving, shaking or an irregular pulse during the measurement can result in the appearance of this icon.

Usually this is not a cause for concern, however if the symbol appears often, we recommend you seek medical advice.

And please note that the device does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

The irregular heart beat detected record won't be stored in memory.

The pulse display is not suitable for checking the frequency of heart pacemakes. If a certain pulse irregularity is detected during measurement often, we recommend you seek medical advice.

However, as a safeguard, we recommend that if you have arrhythmias such as atrial or ventricular premature beats and atrial fibrillation or any other special conditions you should check with your physician before using your device.

The IHB function is not designed for use by people with arrhythmias nor for diagnosing or treating an arrhythmic problem. In order to filter the unstable status of user and avoid affecting the detection of heart rate from any movement, shaking or talking in the beginning of measurement, the method of averaging heart beat intervals for subject device is calculated with the three proper heart beat pulses detected in the beginning of measurement and that is different from a strict mathematical averaging of all recorded intervals.

At least 3 beats with at least 25% difference from the average heart beat interval will generate the IHB icon on the screen.

14 Memory

The has 3 Memory zones. Each can store up to 40 measurements including the date and time



The result will be stored automatically in the selected Memory Zone after the measurement.

To recall measurement results:

- Press the **START/STOP** button to activate the display.
- Press the **UP** or **DOWN** button to select the desired memory zone.
- Press the **MEMORY** button to see the average of all measurement results **[A]**
- Press the **UP** button repeatedly to see prior stored measurement results.
- To stop reading the data, press the **START/STOP** button to switch back to stand-by.

To erase all data from one memory zone:

- Press the **START/STOP** button to activate the display.
- Press the **UP** or **DOWN** button to select the desired memory zone.
- Press the **MEMORY** button to see the average of all measurement results.
- Press the **MODE** and the **DOWN** button simultaneously to erase all data from the selected memory zone.

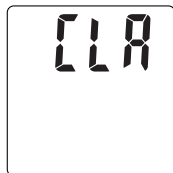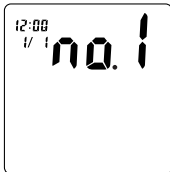15 Technical specifications

Power source	3V DC: Two LR03 (AAA) Alkaline batteries
Measurement method	Oscillometric
Measurement zone	Pressure: 0 ~ 300 mmHg Pulse: 40 ~ 199 beats/minute
Accuracy	Pressure: ± 3 mmHg Pulse: within $\pm 5\%$ of reading
Pressure sensor	Semi-conductor
Inflation	Pump driven
Deflation	Automatic Pressure release valve
Memory capacity	3 Memory zones for each 40 memories
Auto-shut-off	1 minute after last key operation
Operation environment	Temperature $5^{\circ}\text{C} \sim 40^{\circ}\text{C}$ ($41^{\circ}\text{F} \sim 104^{\circ}\text{F}$) Humidity $15\% \sim 93\%$ R.H.
Storage/Transportation environment	Temperature $-25^{\circ}\text{C} \sim 70^{\circ}\text{C}$ ($-13^{\circ}\text{F} \sim 158^{\circ}\text{F}$) Humidity $\leq 93\%$ R.H.
Weight	148g (including batteries)
Wrist circumference	135 ~ 195 mm
Limited patient	Over 18 years old
Dimensions	74(L) x 84(W) x 32(H) mm
Classification 	- Internally powered equipment - Classification of type BF (applied part) - IPX0 - Not suitable for use in presence of flammable anaesthetic mixture with air or with oxygen or nitrous oxide - Continuous operation with short-time loading

*Specifications are subject to change without notice.

The CE symbol indicates that the unit complies with the essential requirements of the directive 93/42/EEC.

16 Warranty

16.1 Warranty period

The Topcom units have a 24-month warranty period. The warranty period starts on the day the new unit is purchased. There is no warranty on Alkaline batteries (AA/AAA type).

Consumables or defects causing a negligible effect on operation or value of the equipment are not covered.

The warranty has to be proven by presentation of the original or copy of the purchase receipt, on which the date of purchase and the unit-model are indicated.

16.2 Warranty exclusions

Damage or defects caused by incorrect treatment or operation and damage resulting from use of non-original parts or accessories are not covered by the warranty.

The warranty does not cover damage caused by outside factors, such as lightning, water and fire, nor any damage caused during transportation.

No warranty can be claimed if the serial number on the units has been changed, removed or rendered illegible.

Any warranty claims will be invalid if the unit has been repaired, altered or modified by the buyer.

To ensure precise measurement reading, re-calibration of the device is recommended after 2 years from the date of purchase. Shipping plus handling cost and recalibration service fee shall be charged accordingly. For more information, check www.tristar.eu

Troubleshooting

SYMBOLS/ SYMPTOMS	CONDITIONS/ CAUSES	INDICATION/ CORRECTION
Unit does not turn on when START/STOP button is pushed.	Batteries have run down.	Replace them with 2 new AAA alkaline batteries.
	Battery polarities have been positioned incorrectly.	Re-insert the batteries in the correct positions.
EE Measuring Error Symbol shown on display when blood pressure value displayed is excessively low or high.	Cuff has been placed incorrectly.	Wrap the cuff properly so that it is positioned correctly.
	Did you talk or move during measurement?	Measure again. Keep wrist steady during measurement.
	Shaking of the wrist with the cuff on.	
E1 Measuring Error Symbol	Air circuit abnormality.	Measure again.
E2 Measuring Error Symbol	Inflation pressure exceeding 300 mmHg.	Switch the unit off, then measure again.
E3 Measuring Error Symbol	Error determining measurement data	Measure again.
Note: if "EP" appears on the display, just return the device to your local distributor or importer.		

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
7 Boutons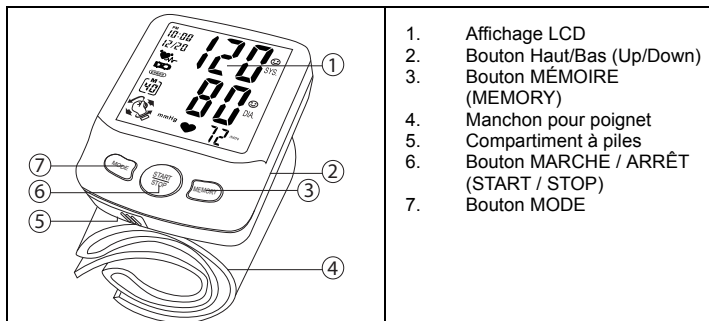

8 Écran

The diagram shows the device's LCD screen with various icons and numbers. Numbered callouts point to specific elements:

- 1. Systolic blood pressure (top right large number)
- 2. Smiling face icon (top right)
- 3. Diastolic blood pressure (middle right large number)
- 4. Heart rate (bottom right large number)
- 5. Heart icon (bottom center)
- 6. mmHg unit (bottom center)
- 7. Hand icon (bottom left)
- 8. Memory number (middle left large number)
- 9. Error icon (middle left)
- 10. Low battery icon (middle left)
- 11. Irregular heart rate icon (middle left)
- 12. Date (top left)
- 13. Time (top left)

Additional labels on the screen include: PM, 18:88, 18/28, SYS., DIA., /MIN, and mmHg.

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
7 Przyciski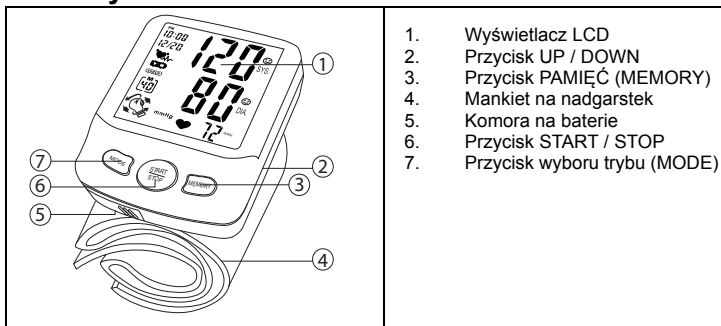

8 Wyświetlacz

The diagram shows the LCD display with the following elements labeled by numbers:

- 1. Systolic blood pressure (top right)
- 2. Smiley face icon (top right)
- 3. Diastolic blood pressure (middle right)
- 4. Heart rate (bottom right)
- 5. Heart icon (bottom center)
- 6. mmHg unit (bottom center)
- 7. Hand icon pointing to a button (bottom left)
- 8. Memory icon (middle left)
- 9. Error icon (middle left)
- 10. Battery level icon (middle left)
- 11. Heart rate icon (middle left)
- 12. Date (top left)
- 13. Time (top left)

Additional labels on the display include: PM, 18:88, 18/88, SYS., DIA., and /MIN.

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Appendix

Guidance and manufacturer's declaration – electromagnetic emissions		
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	

Guidance and manufacturer's declaration – electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact	±6 kV contact	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
	±8 kV air	±8 kV air	
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Guidance and manufacturer's declaration – electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	Not Applicable	Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1,2 \sqrt{P}$ 80 MHz to 800 MHz $d = 1,2 \sqrt{P}$ 800 MHz to 2,5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2,5 GHz	3 V/m	

			Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^a Interference may occur in the vicinity of equipment marked with the following symbol: 
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people. ^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the device. ^a Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.			

Recommended separation distances between portable and mobile RF communications equipment and the device			
The device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz Not applicable	80 MHz to 800 MHz $d = 1,2 \cdot \sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3 \cdot \sqrt{P}$
0,01	Not applicable	0,12	0,23
0,1	Not applicable	0,38	0,73
1	Not applicable	1,2	2,3
10	Not applicable	3,8	7,3
100	Not applicable	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

TOPCOM®

**visit our website
www.tristar.eu**